

# M7

-001 thru -010

Self-Contained Steam Sterilizer



## Service and Parts Manual

Serial Number Prefixes:  
RB, CR, CP, CS, FM, FN

**M7** -001  
thru  
-010



**NO LONGER IN  
PRODUCTION**  
Some service parts may not  
be available for this product!

FOR USE BY MIDMARK TRAINED TECHNICIANS ONLY



# TABLE OF CONTENTS

| Section/Paragraph                                                             | Page | Section/Paragraph                                                             | Page   |
|-------------------------------------------------------------------------------|------|-------------------------------------------------------------------------------|--------|
| IMPORTANT INSTRUCTIONS                                                        |      |                                                                               |        |
| General Safety Instructions .....                                             | ii   | 4.14 Bellows Assembly Removal /<br>Installation (Specify Serial Number) ..... | 4-14   |
| Warnings .....                                                                | ii   | 4.15 Bellows Assembly Removal /<br>Installation (Specify Serial Number) ..... | 4-15   |
| Warranty Instructions.....                                                    | ii   | 4.16 Bellows Assembly Removal /<br>Installation (Specify Serial Number) ..... | 4-16   |
| SECTION I GENERAL INFORMATION                                                 |      | 4.17 Fill / Vent Valve Removal / Installation .....                           | 4-17   |
| 1.1 Scope of Manual .....                                                     | 1-1  | 4.18 Door Assembly Removal / Installation .....                               | 4-18   |
| 1.2 How to Use Manual .....                                                   | 1-1  | 4.19 Door Gasket Removal / Installation .....                                 | 4-19   |
| 1.3 Description of M7 Sterilizer .....                                        | 1-1  | 4.20 Wire Rack And Trays Removal /<br>Installation .....                      | 4-20   |
| 1.4 Specifications .....                                                      | 1-3  | 4.21 Condensing Tank Removal /<br>Installation .....                          | 4-20   |
| 1.5 Parts Replacement Ordering .....                                          | 1-4  | 4.22 Chamber Assembly Removal /<br>Installation .....                         | 4-22   |
| 1.6 Special Tools .....                                                       | 1-4  |                                                                               |        |
| SECTION II TESTING AND TROUBLESHOOTING                                        |      | SECTION V SCHEMATICS, DIAGRAMS, AND<br>CHARTS                                 |        |
| 2.1 Operational Test .....                                                    | 2-1  | 5.1 Electrical Schematics / Wiring Diagrams ....                              | 5-1    |
| 2.2 Troubleshooting Procedures .....                                          | 2-2  | 5.2 Flow Diagrams .....                                                       | 5-7    |
| SECTION III SCHEDULED MAINTENANCE                                             |      | 5.3 Suggested Times And Temperatures<br>Charts.....                           | 5-11   |
| 3.1 Scheduled Maintenance .....                                               | 3-1  |                                                                               |        |
| SECTION IV MAINTENANCE/SERVICE<br>INSTRUCTIONS                                |      | SECTION VI PARTS LIST                                                         |        |
| 4.1 Introduction .....                                                        | 4-1  | 6.1 Introduction .....                                                        | 6-1    |
| 4.2 Cover Assembly Removal / Installation .....                               | 4-1  | 6.2 Description of Columns .....                                              | 6-1    |
| 4.3 Timer Assembly Removal / Installation (New<br>Style Timer Assembly) ..... | 4-1  | M7 Sterilizer .....                                                           | 6-2    |
| 4.4 Timer Assembly Removal / Installation (Old<br>Style Timer Assembly) ..... | 4-2  | Sterilizer Cover .....                                                        | 6-3    |
| 4.5 Timer Buzzer Removal / Installation .....                                 | 4-3  | Sterilizer Cover Components .....                                             | 6-4    |
| 4.6 Pilot Light Removal / Installation .....                                  | 4-4  | Sterilizer Main Components .....                                              | 6-5.*  |
| 4.7 Heating Element Removal / Installation .....                              | 4-4  | Front Panel Components .....                                                  | 6-6.*  |
| 4.8 Overheat Thermostat Removal /<br>Installation .....                       | 4-6  | Door Assembly .....                                                           | 6-7.*  |
| 4.9 Temperature Regulator Relay Removal /<br>Installation .....               | 4-8  | Base Components .....                                                         | 6-8.*  |
| 4.10 Temperature Regulator Relay<br>Adjustment .....                          | 4-10 | Chamber Components - Side .....                                               | 6-9    |
| 4.11 Diaphragm Cup Removal / Installation .....                               | 4-12 | Chamber Components - Lower .....                                              | 6-10.* |
| 4.12 Temperature Gauge Removal /<br>Installation .....                        | 4-12 | Tank Assembly .....                                                           | 6-11.* |
| 4.13 Pressure Relief Valve Removal /<br>Installation .....                    | 4-13 | Valve Assembly .....                                                          | 6-12.* |
|                                                                               |      | Rack And Trays .....                                                          | 6-13   |
|                                                                               |      | Packaging .....                                                               | 6-14   |
|                                                                               |      | COMMENTS .....                                                                | 7-1    |
|                                                                               |      | FAX ORDERING FORM.....                                                        | 7-2    |

(\*) Indicates that there has been a serial number break for the illustration and that there are additional point page(s) following the original page.

# IMPORTANT INSTRUCTIONS

## General Safety Instructions

**Safety First:** The primary concern of Midmark Corporation is that this sterilizer is maintained with the safety of the staff in mind. To assure that services and repairs are completed safely and correctly, proceed as follows:

- (1) Read this entire manual before performing any services or repairs on this sterilizer.
- (2) Be sure you understand the instructions contained in this manual before attempting to service or repair this sterilizer.

## Warnings

Throughout this manual are Note, Caution, and Danger paragraphs that call attention to particular procedures. These items are used as follows:

### NOTE

A note is used to amplify an operating procedure, practice, or condition.



### CAUTION

A CAUTION is used for an operating procedure, practice, or condition which, if not correctly followed, could result in equipment damage.



### DANGER

A DANGER is used for an operating procedure, practice, or condition which, if not correctly followed, could result in loss of life or serious personal injury.

## Warranty Instructions

Refer to the Midmark "Limited Warranty" printed on the back cover of the Installation and Operation Manual for warranty information. Failure to follow the guidelines listed below will void the warranty and/or render the sterilizer unsafe for operation.

- In the event of a malfunction, do not attempt to operate the sterilizer until necessary repairs have been made.
- Do not attempt to disassemble sterilizer, replace malfunctioning or damaged components, or perform adjustments unless you are one of Midmark's authorized service technicians.
- Do not substitute parts of another manufacturer when replacing inoperative or damaged components. Use only Midmark replacement parts.

## **SECTION I**

### **GENERAL INFORMATION**

#### **1.1 Scope of Manual**

This manual contains detailed troubleshooting, scheduled maintenance, maintenance, and service instructions for the M7 SpeedClave® Sterilizer. This manual is intended to be used by Midmark's authorized service technicians.

#### **1.2 How to Use Manual**

- A. Manual Use When Performing Scheduled Maintenance.
  - (1) Perform inspections and services listed in Scheduled Maintenance Chart (Refer to para 3.1).
  - (2) If a component is discovered to be faulty or out of adjustment, replace or adjust component in accordance with Maintenance / Service Instructions (Refer to para 4.1).
- B. Manual Use When Sterilizer Is Malfunctioning And Cause Is Unknown.
  - (1) Perform an operational test on sterilizer (Refer to para 2.1).
  - (2) Perform troubleshooting procedures listed in Troubleshooting Guide (Refer to para 2.2).
  - (3) If a component is discovered to be faulty or out of adjustment, replace or adjust component in accordance with Maintenance/Service Instructions (Refer to para 4.1).
- C. Manual Use When Damaged Component Is Known.
  - (1) Replace or adjust component in accordance with Maintenance/Service Instructions (Refer to para 4.1).

#### **1.3 Description Of M7 Sterilizer**

- A. General Description (See Figure 1-1).

The M7 SpeedClave® Sterilizer is a self generating steam sterilizer designed to sterilize instruments and other sterilizable goods. The major components of the


sterilizer consist of a chamber, condensing tank assembly, heating element, temperature regulator relay, diaphragm cup, timer assembly, timer buzzer, manual reset overheat thermostat (low water), auto-reset overheat thermostat (low water) (auto-reset overheat thermostat is only on newer units with CS or RB serial number prefixes), pilot light, fill / vent valve, bellows assembly, pressure relief valve, temperature gauge, and reset button linkage assembly.

- B. Theory of Operation (See Figures 5-1 thru 5-3).

##### Electrical Operation

Current flows thru the normally closed contacts of the overheat thermostat(s) (also known as the low water thermostat) to one side of the contacts on the timer switch. When the timer assembly is not set to a time, the timer switch contacts are not closed and no path to the heating element is completed. When the operator sets the timer assembly to any time setting, the timer switch contacts are closed allowing current to flow across the normally closed contacts of the temperature regulator relay and be applied to the heating element, causing it to heat up. Current is also applied across the pilot light causing it to illuminate, thus indicating that the heating element is energized. The diaphragm cup is pressure sensitive and expands outward as the pressure in the chamber increases. The normally closed temperature regulator relay is adjusted so the diaphragm cup will push open the contacts of the relay when the selected temperature / pressure is reached during a sterilizing cycle, de-energizing the heating element. When the temperature / pressure lowers slightly, the diaphragm cup contracts allowing the temperature regulator relay to close its contacts which causes current to be applied to the heating element and pilot light again. The temperature in the chamber is regulated to within +1° to +2° of the selected temperature during the cycle by the continuous opening and closing of the temperature regulator relay contacts by the diaphragm cup. The timer assembly contains a timer motor which runs the timer assembly. When the timer is run down to a setting of 0 minutes, the timer switch contacts move, breaking the circuit to the heating element and pilot light. A circuit to the timer buzzer is now completed, causing it to sound. The timer assembly timer motor continues to run for one minute. Then the timer switch contacts are opened, causing the timer buzzer to stop sounding.

**SECTION I**  
**GENERAL INFORMATION**

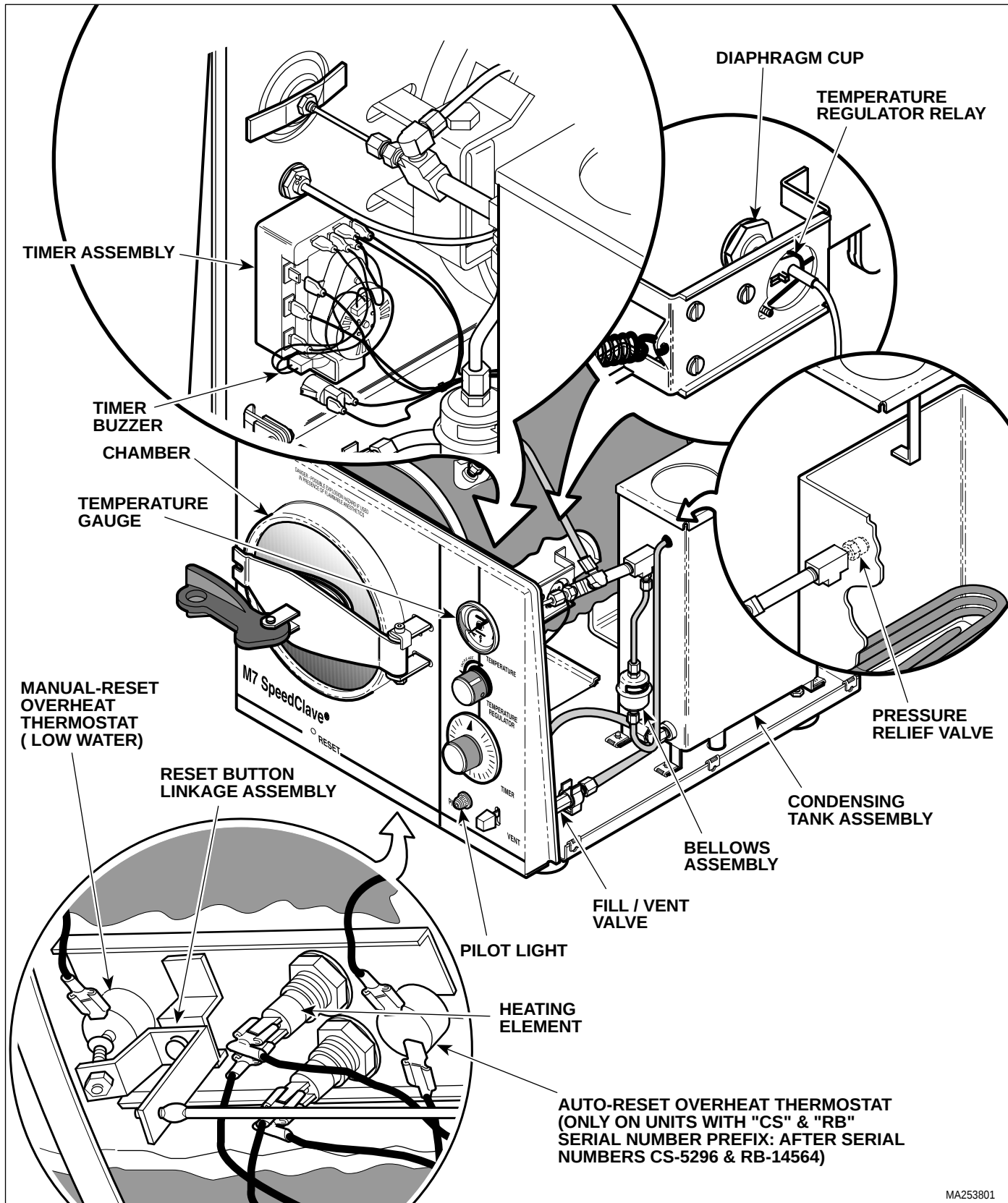

**Figure 1-1. Major Components**

## SECTION I GENERAL INFORMATION

During the cycle, if the temperature inside the chamber rises up higher than 285°F (141°C), the normally closed manual reset overhear thermostat (also known as low water thermostat) contacts open, breaking the circuit to the timer switch, heating element, and pilot light. The most frequent cause of activation of the overhear thermostat is a low water condition in the chamber. The N.C. auto-reset overhear thermostat is a safety backup for the manual reset overhear thermost and opens at 295°F (146°C). The manual reset overhear thermostat is reset by pressing the RESET button located on front panel of the sterilizer and the auto-reset overhear thermostat automatically resets after approximately six minutes.

### Water / Air / Steam Flow

The M7 sterilization cycle has four phases; filling, heat up, sterilizing, and venting. During the fill portion of the cycle, the operator depresses the FILL / VENT lever which opens the fill / vent valve. Water flows from the condensing tank thru the fill / vent valve and into the chamber. When the operator visually observes that the water level in the chamber is within 1/2 in. to 5/8 in. (13 to 16 mm) from the front rim of the chamber, the operator releases the FILL / VENT lever, closing the fill / vent valve. Now the operator begins the heat up portion of the cycle by turning on the timer assembly. During the heat up portion of the cycle, the water is heated by the heating element. As the water begins to boil, air is bled off thru the bellows assembly into the condensing tank. When the bellows assembly senses pure steam flowing thru it, the valve in the bellows assembly closes, allowing pressure in the chamber to build. The sterilizing portion of the cycle begins when the bellows assembly is completely closed, not allowing steam to flow thru it, and the desired temperature in the chamber for the selected cycle is reached. The operator sets the timer assembly for the desired length of the cycle and the cycle is run. When the timer assembly counts down to 0 minutes and shuts off the heating element, the vent portion of the cycle begins. After the door handle has been moved to the vent position, the operator depresses the FILL / VENT lever which opens the fill / vent valve. Steam and water flow thru the fill / vent valve and into the condensing tank, venting the chamber. There is coiled tubing under water in the condensing tank which serves to turn the steam back into water.

There is a pressure relief valve which opens if the pressure in the pressure valve reaches 31 PSI (214 kPa) during a cycle. This provides a safety relief for the chamber so that unsafe pressures cannot build.

There is a temperature gauge on the front control panel which indicates the temperature inside the chamber during a cycle.

## 1.4 SPECIFICATIONS

Factual data for the sterilizer is provided in Table 1-1.

**Table 1-1. Specifications**

| Description                                                                                                                                                                                                                       | Data                                                                  |
|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------|
| Dimensions (overall):                                                                                                                                                                                                             |                                                                       |
| Length .....                                                                                                                                                                                                                      | 19 in (48.3 cm)                                                       |
| Width .....                                                                                                                                                                                                                       | 14 in (35.6 cm)                                                       |
| Height .....                                                                                                                                                                                                                      | 13 in (33 cm)                                                         |
| Shipping Carton .....                                                                                                                                                                                                             | 24 in x 16 in x 16 in<br>(61 cm x 40.6 cm x 40.6 cm)                  |
| Door Opening .....                                                                                                                                                                                                                | 6 5/8 in. (16.8 cm)                                                   |
| Weight:                                                                                                                                                                                                                           |                                                                       |
| With Reservoir Empty .....                                                                                                                                                                                                        | 30 lb (13.6 kg)                                                       |
| With Reservoir Full .....                                                                                                                                                                                                         | 39 lb (17.7 kg)                                                       |
| With Shipping Carton .....                                                                                                                                                                                                        | 39 lb (17.7 kg)                                                       |
| Water Reservoir Capacity .....                                                                                                                                                                                                    | Approx. 3/4 gallon<br>(2.75 Liters) to full mark                      |
| Electrical Requirements:                                                                                                                                                                                                          |                                                                       |
| 100 VAC Unit (M7-003) .....                                                                                                                                                                                                       | 100 VAC 50 - 60 HZ,<br>15 amp, single phase                           |
| 115 VAC Unit (M7-001, .....                                                                                                                                                                                                       | 110 - 120 VAC 50 - 60 HZ,<br>M7-004, and M7-005) 10 amp, single phase |
| 230 VAC Unit (M7-002) .....                                                                                                                                                                                                       | 220 - 240 VAC 50 - 60 HZ,<br>5 amp, single phase                      |
| Power Consumption:                                                                                                                                                                                                                |                                                                       |
| 100 VAC Unit .....                                                                                                                                                                                                                | 1150 WATTS,<br>12 amps @ 100 VAC                                      |
| 115 VAC Unit .....                                                                                                                                                                                                                | 1150 WATTS,<br>10 amps @ 120 VAC                                      |
| 230 VAC Unit .....                                                                                                                                                                                                                | 1150 WATTS,<br>5 amps @ 240 VAC                                       |
| Recommended Circuit:                                                                                                                                                                                                              |                                                                       |
| A separate (dedicated) circuit is recommended for this sterilizer. The sterilizer <i>should not</i> be connected to an electrical circuit with other appliances or equipment unless the circuit is rated for the additional load. |                                                                       |

SECTION I  
GENERAL INFORMATION

Table 1-1. Specifications - Continued

| Description                                     | Data                                                        |
|-------------------------------------------------|-------------------------------------------------------------|
| Chamber Pressure:                               |                                                             |
| Operating .....                                 | 27 psi (175 kPa)                                            |
| Maximum Before Pressure Relief Valve Opens..... | 31 psi (214 kPa)                                            |
| Chamber Temperature: .....                      | See Tables 5-1 and 5-2 for Suggested Times And Temperatures |

1.5 Parts Replacement Ordering

If a part replacement is required, order the part directly from the factory as follows:

- (1) Refer to Figure 1-2 to determine the location of the model number and serial number of the sterilizer and record this data. There are different letter prefixes which proceed the serial number, depending on the configuration of the unit. These prefixes are very important and are needed to order the proper parts.
- (2) Refer to the Parts List to determine the item numbers of the parts, part numbers of the parts, descriptions of the parts, and quantities of parts needed and record this data (Refer to para 6.1).

NOTE

Ask the Purchasing Department of the company that owns the sterilizer for this information. Otherwise, this information may be obtained from the dealer that sold the sterilizer.

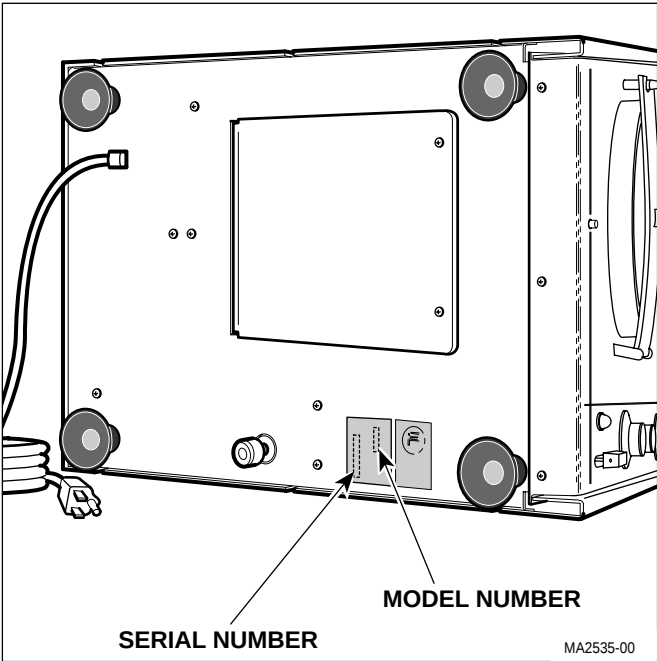


Figure 1-2. Model Number / Serial Number Location

- (3) Determine the installation date of the sterilizer and record this data.
- (4) Call Midmark with the recorded information and ask for the Medical Services Department. See back cover of this manual for the phone number or use the Fax Order Form (see page 7-2 for Fax Order Form).

1.6 Special Tools

Table 1-2 lists all the special tools needed to repair the sterilizer, describes how to obtain the special tools, and describes the purpose of each special tool.

Table 1-2. Special Tool List

| Description of Special Tool     | Manufacturer's Name / Address / Phone                                                    | Manufacturer's Part Number | Purpose of Special Tool                                                                                                                                                                    |
|---------------------------------|------------------------------------------------------------------------------------------|----------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Maximum Registering Thermometer | Taylor Precision Products<br>280 Cane Creek Road<br>Fletcher, NC 28732<br>(828) 684-5178 | 21466-1                    | Used to check the maximum temperature reached inside the pressure vessel during a cycle to diagnose malfunctions and/or adjust the maximum temperature potentiometer to a correct setting. |
| Multimeter                      | Commercially Available                                                                   | Any Type                   | Used to check probes, switches, and connections for proper function by performing continuity checks.                                                                                       |

**SECTION II**  
**TESTING AND TROUBLESHOOTING**

**2.1 Operational Test**

In order to effectively diagnose the malfunction of the sterilizer, it is necessary to perform an operational test as follows:



**DANGER**

**Refer to the operator manual for complete instructions on operating the sterilizer. Failure to do so could result in severe personal injury.**

- (1) Place the sterilizer on a level surface.
- (2) Plug the sterilizer into a properly grounded receptacle, capable of supplying correct and adequate power to operate this sterilizer.
- (3) Remove the fill cap from the condensing tank.

**NOTE**

Overfilling will cause sterilizer to malfunction.

- (4) Fill the condensing tank to the FULL mark indicator tab with distilled or demineralized water. Replace filler cap.
- (5) Swing the door handle to the unlatched position and then pull the door out of the chamber.



**CAUTION**

**Ensure that the Maximum Registering Thermometer has been shaken down to its lowest setting. Failure to do so could result in an incorrect reading.**

- (6) Place a Maximum Registering Thermometer in the middle tray toward the door of the sterilizer (Refer to Table 1-2 for special tool).
- (7) Depress the FILL / VENT lever and allow the water to fill the chamber until the water level is within 1/2 to 5/8 in. (13 to 16 mm) from the front rim of the chamber; then release the FILL / VENT lever.
- (8) Close and latch the door in the chamber as follows: Insert the right edge of the door into the chamber and then push the rest of the door

into the chamber. Latch the door by swinging the door handle all the way to the latched position.

- (9) Set the TIMER knob to its maximum setting of 30 minutes. Set the TEMPERATURE REGULATOR knob to its maximum setting of 270°F (132°C) by turning the knob counter-clockwise as far as possible.
- (10) Observe. The PILOT light will illuminate indicating that the heating element is energized. Should begin to hear water boiling in the chamber and steam and air releasing thru the bellows assembly. Should hear the bellows assembly close completely - hissing sound should almost completely stop. The PILOT light should flash on and off during the cycle indicating the heating element is being energized and deenergized to regulate the temperature in the chamber.
- (11) Record the highest temperature reached during the cycle from the TEMPERATURE gauge on the front panel of the sterilizer.

**NOTE**


Earlier units do not have a timer buzzer.

- (12) Observe. Wait for the TIMER knob to run down to 0 minutes. When 0 minutes is reached, the sterilization cycle is over and power to the heating element and pilot light is removed. Proceed immediately to step 13.

On later units, the timer buzzer will sound when the timer assembly has run down to 0 minutes. This is to alert the operator that the sterilization cycle is completed. After 1 minute the timer buzzer will stop sounding. Proceed immediately to step 13.

- (13) Swing the door handle to the vent position.
- (14) Depress the FILL / VENT lever to vent the chamber; then release the lever once the chamber is vented. Allow chamber to cool.
- (15) Observe. Water and steam will flow into the condensing tank, making a bubbling sound.

## SECTION II TESTING AND TROUBLESHOOTING

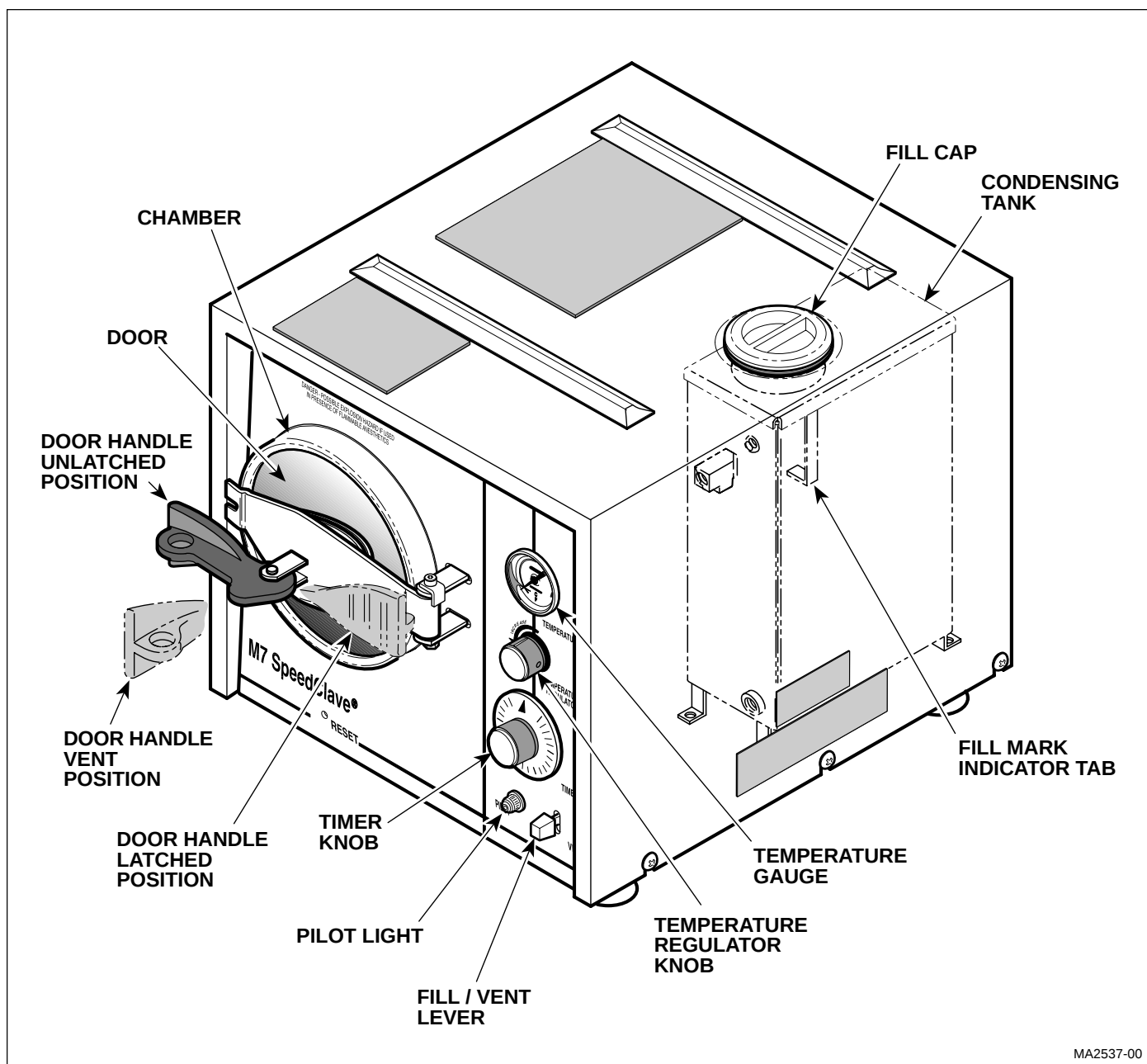

**Figure 2-1. Operational Test**

When the bubbling sound stops and the door pops open, the chamber will be vented.

- (16) Allow the Maximum Registering Thermometer to cool and then remove it from the chamber.

### NOTE

The Maximum Registering Thermometer has an accuracy of  $\pm 4^{\circ}\text{F}$  ( $\pm 2.2^{\circ}\text{C}$ ).

- (17) Read the temperature on the Maximum Registering Thermometer. The reading should be  $270^{\circ}\text{F}$  -  $272^{\circ}\text{F}$  ( $132.2^{\circ}\text{C}$  -  $133.4^{\circ}\text{C}$ ) and should match the temperature reading recorded from the TEMPERATURE gauge during the cycle.

## 2.2 Troubleshooting Procedures (see next page)

Table 2-1 is a troubleshooting guide which is used to determine the cause of the malfunction.

## SECTION II

### TESTING AND TROUBLESHOOTING

**Table 2-1. Troubleshooting Guide**

| Problem                                              | Symptom                                                                                                            | Probable Cause                                                                                                                                                                       | Check                                                                                                                                                                          | Correction                                                                                                                                 |
|------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------|
| Unit seems powerless.                                | Pilot light does not illuminate when timer is set and unit does not start to heat up.                              | Power cord is not plugged in to wall outlet.                                                                                                                                         | Check to see if power cord is plugged in.                                                                                                                                      | Plug power cord into facility wall outlet.                                                                                                 |
|                                                      |                                                                                                                    | Facility circuit breaker providing power to unit is tripped.                                                                                                                         | Check to see if facility circuit breaker is tripped. One way of checking this is to plug a lamp into wall outlet that unit was plugged into.                                   | If circuit breaker is tripped, determine what caused circuit breaker to trip, correct the problem, and then reset/replace circuit breaker. |
|                                                      |                                                                                                                    | Manual reset overhear thermostat is tripped.                                                                                                                                         | If the unit is hot, allow the unit to cool for 15 to 20 minutes. Press the RESET button on the front panel. Now, the PILOT light should illuminate when the TIMER knob is set. | If necessary, add distilled water to condensing tank; then press the RESET button on the front panel.                                      |
|                                                      |                                                                                                                    | Auto-reset overhear thermostat is tripped (Applies only to units with Serial Numbers CS5297 and RB14565 Thru Present).                                                               | If the unit is hot, allow the unit to cool for 15 to 20 minutes. Now, the PILOT light should illuminate when the TIMER knob is set.                                            | Wait 15 to 20 minutes to allow thermostat to reset itself. Also, replace manual reset overhear thermostat. Refer to para 4.8.              |
|                                                      |                                                                                                                    | Manual reset or auto-reset overhear thermostat is malfunctioning - stuck open (Auto-reset overhear thermostat is only on units with Serial Numbers CS5297 and RB14565 Thru Present). | Perform continuity check on N.C. overhear thermostat (cool = closed).                                                                                                          | If open, replace overhear thermostat. Refer to para 4.8.                                                                                   |
|                                                      |                                                                                                                    | Timer assembly is malfunctioning.                                                                                                                                                    | Replace suspect timer assembly with known working timer assembly.                                                                                                              | Replace timer assembly. Refer to para 4.3 or 4.4.                                                                                          |
|                                                      |                                                                                                                    | Temperature regulator relay is malfunctioning - stuck open.                                                                                                                          | Perform continuity check on temperature regulator relay.                                                                                                                       | Replace temperature regulator relay. Refer to para 4.9.                                                                                    |
|                                                      |                                                                                                                    | Wiring connections loose.                                                                                                                                                            | Check all wiring connections.                                                                                                                                                  | Clean any dirty connections. Tighten any loose connections. Replace any damaged connections.                                               |
| Unit does not complete sterilization cycle properly. | PILOT light does illuminate but chamber does not reach required temperature / pressure during sterilization cycle. | Flexible shaft assembly is malfunctioning.                                                                                                                                           | Check flexible shaft assembly for wear or damage.                                                                                                                              | Replace flexible shaft assembly. Refer to para 4.9.                                                                                        |
|                                                      |                                                                                                                    | Temperature regulator relay is out of adjustment.                                                                                                                                    | Check adjustment of temperature regulator relay.                                                                                                                               | Adjust temperature regulator relay. Refer to para 4.10.                                                                                    |
|                                                      |                                                                                                                    | Temperature regulator relay is malfunctioning.                                                                                                                                       | Check if temperature regulator relay is stuck open or contacts not touching properly.                                                                                          | Replace temperature regulator relay. Refer to para 4.9.                                                                                    |

## SECTION II

### TESTING AND TROUBLESHOOTING

**Table 2-1. Troubleshooting Guide - Continued**

| Problem                                                          | Symptom                                                                                                                        | Probable Cause                                                                                                                                                                             | Check                                                                                                                                                                          | Correction                                                                                                                    |
|------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------|
| Unit does not complete sterilization cycle properly - Continued. | PILOT light does illuminate but chamber does not reach required temperature / pressure during sterilization cycle - Continued. | Heating element is malfunctioning.                                                                                                                                                         | Check resistance of heating element (in ohms):<br>100 V, 50 Hz-8.08 to 8.94<br>115 V, 60 Hz-11.31 to 12.5<br>230 V, 50 Hz-45.65 to 50.45                                       | If heating element resistance is not within range, replace heating element. Refer to para 4.7.                                |
|                                                                  |                                                                                                                                | Bellows assembly is closing prematurely or stuck in open position.                                                                                                                         | Replace suspect bellows assembly with known working bellows assembly.                                                                                                          | Replace bellows assembly. Refer to para 4.14, 4.15, or 4.16.                                                                  |
|                                                                  |                                                                                                                                | Pressure relief valve is malfunctioning - stuck open.                                                                                                                                      | Run cycle and observe pressure relief valve; it should not dump pressure until temperature gauge reads 275 degrees F (135 degrees C) +/- 2 degrees.                            | Replace pressure relief valve. Refer to para 4.13.                                                                            |
|                                                                  |                                                                                                                                | Diaphragm cup is malfunctioning.                                                                                                                                                           | Replace suspect diaphragm cup with known working diaphragm cup.                                                                                                                | Replace diaphragm cup. Refer to para 4.11.                                                                                    |
|                                                                  |                                                                                                                                | Fill / vent valve is malfunctioning - stuck open or leaking past o-rings.                                                                                                                  | Check fill / vent valve for dirt and o-rings for wear or damage.                                                                                                               | Clean or replace fill / vent valve. Refer to para 4.17.                                                                       |
|                                                                  |                                                                                                                                | Door gasket and / or door assembly is leaking.                                                                                                                                             | Run cycle and observe for leaks around edges of door assembly.                                                                                                                 | Replace door gasket. Refer to para 4.19. Replace door assembly if necessary. Refer to para 4.18.                              |
|                                                                  | Temperature reaches 210 - 212 degrees F (98.8 - 100 degrees C) during cycle and stays there.                                   | Bellows assembly is stuck in open position.                                                                                                                                                | Replace suspect bellows assembly with known working bellows assembly.                                                                                                          | Replace bellows assembly. Refer to para 4.14, 4.15, or 4.16..                                                                 |
|                                                                  | Time set on timer assembly does not expire.                                                                                    | Timer motor is malfunctioning.                                                                                                                                                             | Replace suspect timer assembly with known working timer assembly.                                                                                                              | Replace timer assembly. Refer to para 4.3 or 4.4.                                                                             |
|                                                                  |                                                                                                                                | Manual reset overhear thermostat is tripped.                                                                                                                                               | If the unit is hot, allow the unit to cool for 15 to 20 minutes. Press the RESET button on the front panel. Now, the PILOT light should illuminate when the TIMER knob is set. | If necessary, add distilled water to condensing tank; then press the RESET button on the front panel.                         |
|                                                                  |                                                                                                                                | Auto-reset overhear thermostat is tripped (Applies only to units with Serial Numbers CS5297 and RB14565 Thru Present).                                                                     | If the unit is hot, allow the unit to cool for 15 to 20 minutes. Now, the PILOT light should illuminate when the TIMER knob is set.                                            | Wait 15 to 20 minutes to allow thermostat to reset itself. Also, replace manual reset overhear thermostat. Refer to para 4.8. |
|                                                                  |                                                                                                                                | Manual reset or auto-reset overhear thermostat is malfunctioning - stuck open (Auto-reset overhear thermostat is only on units with Serial Numbers CS5297 and RB14565 Thru Present).       | Perform continuity check on N.C. overhear thermostat (cool = closed).                                                                                                          | If open, replace overhear thermostat. Refer to para 4.8.                                                                      |
|                                                                  |                                                                                                                                | Diaphragm cup is malfunctioning - is failing to open contacts of temperature regulator relay when maximum temperature (270 degrees F [132 degrees C]) is reached resulting in overheating. | Replace suspect diaphragm cup with known working diaphragm cup.                                                                                                                | Replace diaphragm cup. Refer to para 4.11.                                                                                    |

## SECTION II TESTING AND TROUBLESHOOTING

**Table 2-1. Troubleshooting Guide - Continued**

| Problem                                                          | Symptom                                                                               | Probable Cause                                                | Check                                                                                                                                        | Correction                                                                                                                   |
|------------------------------------------------------------------|---------------------------------------------------------------------------------------|---------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------|
| Unit does not complete sterilization cycle properly - Continued. | Chamber does not vent at completion of cycle when the FILL / VENT lever is depressed. | Fill / vent valve is clogged with foreign matter.             | Check fill / vent valve for foreign matter.                                                                                                  | Clean or replace fill / vent valve. Refer to para 4.17.                                                                      |
|                                                                  |                                                                                       | Tubing is clogged.                                            | Check tubing for foreign matter.                                                                                                             | Clean out tubing.                                                                                                            |
|                                                                  | Sterility Indicators show sterilization standards were not met.                       | Improper loading of unit.                                     | Check operator's manual for recommended loading procedures.                                                                                  | Load unit properly and repeat cycle.                                                                                         |
|                                                                  |                                                                                       | Temperature regulator relay is out of adjustment.             | Check adjustment of temperature regulator relay using a lag thermometer - TEMPERATURE gauge on front panel may be giving inaccurate reading. | Adjust temperature regulator relay. Refer to para 4.10.                                                                      |
|                                                                  |                                                                                       | Bellows assembly is stuck in open position.                   | Replace suspect bellows assembly with known working bellows assembly.                                                                        | Replace bellows assembly. Refer to para 4.14, 4.15, or 4.16.                                                                 |
|                                                                  | Timer does not buzz at end of cycle.                                                  | Timer buzzer is malfunctioning.                               | Replace suspect timer buzzer with known working timer buzzer.                                                                                | Replace timer buzzer. Refer to para 4.5.                                                                                     |
| FILL / VENT function does not work properly                      | When FILL / VENT lever is depressed, water does not flow into chamber.                | Fill / vent valve is clogged with foreign matter.             | Check fill / vent valve for foreign matter.                                                                                                  | Clean or replace fill / vent valve. Refer to para 4.17.                                                                      |
|                                                                  |                                                                                       | Fill / Vent valve is not adjusted to open and close properly. | Check adjustment of Fill / Vent valve.                                                                                                       | Adjust closing bracket on fill vent valve. Refer to para 4.17.                                                               |
|                                                                  |                                                                                       | Condensing tank is low on distilled or demineralized water.   | Check water level in condensing tank.                                                                                                        | Fill condensing tank with water up to FULL mark.                                                                             |
|                                                                  |                                                                                       | Tubing is clogged.                                            | Check condensing tank and tubing for foreign matter.                                                                                         | Clean out condensing tank and tubing. Replace condensing tank if necessary. Refer to para 4.21. Replace tubing if necessary. |
|                                                                  | When FILL / VENT lever is released, water continues to enter the chamber.             | Fill / vent valve is clogged with foreign matter.             | Check fill / vent valve for dirt.                                                                                                            | Clean or replace fill / vent valve. Refer to para 4.17.                                                                      |
|                                                                  |                                                                                       | O-rings in Fill / Vent valve are worn or damaged.             | Check o-rings in Fill / Vent valve for wear or damage.                                                                                       | Repair Fill / Vent Valve. Refer to para 4.17.                                                                                |
|                                                                  |                                                                                       | Fill / Vent valve is not adjusted to open and close properly. | Check adjustment of Fill / Vent valve.                                                                                                       | Adjust closing bracket on fill vent valve. Refer to para 4.17.                                                               |
|                                                                  | When FILL / VENT lever is depressed, water does not vent from chamber.                | Fill / vent valve is clogged with foreign matter.             | Check fill / vent valve for dirt.                                                                                                            | Clean or replace fill / vent valve. Refer to para 4.17.                                                                      |
|                                                                  |                                                                                       | Fill / Vent valve is not adjusted to open and close properly. | Check adjustment of Fill / Vent valve.                                                                                                       | Adjust closing bracket on fill vent valve. Refer to para 4.17.                                                               |
|                                                                  |                                                                                       | Tubing is clogged.                                            | Check condensing tank and tubing for foreign matter.                                                                                         | Clean out condensing tank and tubing. Replace condensing tank if necessary. Refer to para 4.21. Replace tubing if necessary. |

## SECTION II

### TESTING AND TROUBLESHOOTING

**Table 2-1. Troubleshooting Guide - Continued**

| Problem                                   | Symptom                               | Probable Cause                                                               | Check                                                       | Correction                                                                                  |
|-------------------------------------------|---------------------------------------|------------------------------------------------------------------------------|-------------------------------------------------------------|---------------------------------------------------------------------------------------------|
| Non operational problems with sterilizer. | Sterilizer slides on counter top.     | Film or grease on counter top.                                               | Check for film or grease on counter top.                    | Clean counter top.                                                                          |
|                                           |                                       | Damaged "suction" foot.                                                      | Check if "suction" foot works.                              | Replace damaged "suction" foot.                                                             |
|                                           | Water leaks around sterilizer.        | Loose fittings.                                                              | Check all fittings for leaks.                               | Remove fitting, coat threads of fitting with Teflon tape or sealant, and reinstall fitting. |
|                                           |                                       | Compression nut and/ or compression sleeve is loose or excessively deformed. | Check all compression nuts for leaks.                       | Loosen compression nut. Replace compression sleeve if necessary.                            |
|                                           |                                       | Tubing is damaged.                                                           | Check for holes, kinks, or other types of damage to tubing. | Replace tubing.                                                                             |
|                                           | Door handle is hard to operate.       | Handle cam is dry.                                                           | Check for lubricant on cam of door handle.                  | Lubricate cam part of handle with high temperature grease.                                  |
|                                           | Water leaks around door during cycle. | Door gasket is worn or damaged.                                              | Check door gasket for wear or damage.                       | Replace door gasket. Refer to para 4.19.                                                    |

## SECTION III SCHEDULED MAINTENANCE

### 3.1 Scheduled Maintenance

Table 3-1 is a Scheduled Maintenance Chart which lists the inspections and services that should be performed

periodically on the sterilizer. These inspections and services should be performed as often as indicated in the chart.

**Table 3-1. Scheduled Maintenance Chart**

| Interval      | Inspection or Service            | What to Do                                                                                                                                                                                                                                                                                                       |
|---------------|----------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Semi-annually | Obvious damage                   | Visually check condition of sterilizer for obvious damage such as: cracks in components, missing components, dents in components, leaks, or any other visible damage which would cause sterilizer to be unsafe to operate or would compromise the performance of the sterilizer. Repair sterilizer if necessary. |
|               | Fasteners/hardware               | Check sterilizer for missing or loose fasteners/hardware. Replace any missing hardware and tighten any loose hardware as necessary using Loctite 242 if necessary.                                                                                                                                               |
|               | Moving parts                     | All moving parts should be lubricated with high temperature grease.                                                                                                                                                                                                                                              |
|               | Warning and instructional decals | Check for missing or illegible decals. Replace decals as necessary.                                                                                                                                                                                                                                              |
|               | Wiring connections               | Check the integrity of all wiring connections. Clean all dirty connections. Tighten any loose connections. Replace any damaged connections.                                                                                                                                                                      |
|               | Easy movement of door handle     | Lubricate cam of door assembly with high temperature grease.                                                                                                                                                                                                                                                     |
|               | Door gasket                      | Remove door gasket and check it for dirt, voids, or deterioration. Clean gasket using a mild soap and water solution. Replace gasket if necessary. Refer to para 4.19.                                                                                                                                           |
|               | Door assembly                    | Check door assembly for any damage which would make it unsafe or compromise the performance of the sterilizer. Make sure long and short springs of door assembly have enough power left to open door. Replace springs or door assembly if necessary. Refer to para 4.18.                                         |
|               | Door assembly hinge screws.      | Make sure nut on hinge screws are secure. If nut is loose, remove nut, coat threads of nut with removable threadlocking adhesive (Loctite 242), and reinstall nut.                                                                                                                                               |
|               | PILOT light                      | With power to unit, set timer. PILOT light should illuminate.                                                                                                                                                                                                                                                    |
|               | Condensing tank                  | Remove tank cover and inspect for foreign matter and buildup. Clean out condensing tank and remove buildup from walls of tank.                                                                                                                                                                                   |
|               | Tubing                           | Remove tubing and inspect for buildup. Clean, drain, and flush tubing using Speed-Clean (directions for use are on bottle). Replace tubing if necessary. See Operator's manual.                                                                                                                                  |
|               | Chamber                          | Check for cracks or other signs of stress or metal fatigue. Clean chamber with Speed-Clean (directions for use are on bottle). Check chamber for excessive rust or deterioration. Replace chamber if necessary. Refer to para 4.22.                                                                              |
|               | Wire rack and trays              | Check wire rack and trays for excessive rust or deterioration. Replace wire rack and trays if necessary. Refer to para 4.20.                                                                                                                                                                                     |
|               | Suction feet                     | Check all four suction feet for wear or deterioration. Make sure suction feet fasten securely to counter top. Clean counter top if necessary. Replace suction feet if necessary.                                                                                                                                 |
|               | Operational test                 | Perform an operational test to determine if the sterilizer is operating within its specifications (Refer to para 2.1). Adjust or replace any malfunctioning components                                                                                                                                           |



## SECTION IV MAINTENANCE / SERVICE INSTRUCTIONS

### 4.1 Introduction



#### **DANGER**

Refer to the Operator Manual for complete instructions on operating the sterilizer. Failure to do so could result in personal injury.

#### **NOTE**

Perform an operational test on the sterilizer after the repair is completed to confirm the repair was properly made and that all malfunctions were repaired.

The following paragraphs contain replacement, repair, and adjustment procedures for the sterilizer.

### 4.2 Cover Assembly Removal / Installation

#### A. Removal




#### **DANGER**

Always unplug the power cord from the wall outlet before removing any of the sterilizers cover / panels or making any repairs to prevent the possibility of electrical shock. Failure to comply with these instructions could result in severe personal injury or death.

- (1) Disconnect power cord from wall outlet.
- (2) Loosen six screws (1, Figure 4-1), three on each side, and remove cover assembly (2) from base assembly (3).

#### B. Installation

- (1) Align channel of cover assembly (2) with rail of base assembly (3) and then slide cover assembly onto base assembly. Secure in place by tightening six screws (1).
- (2) Connect power cord to wall outlet.

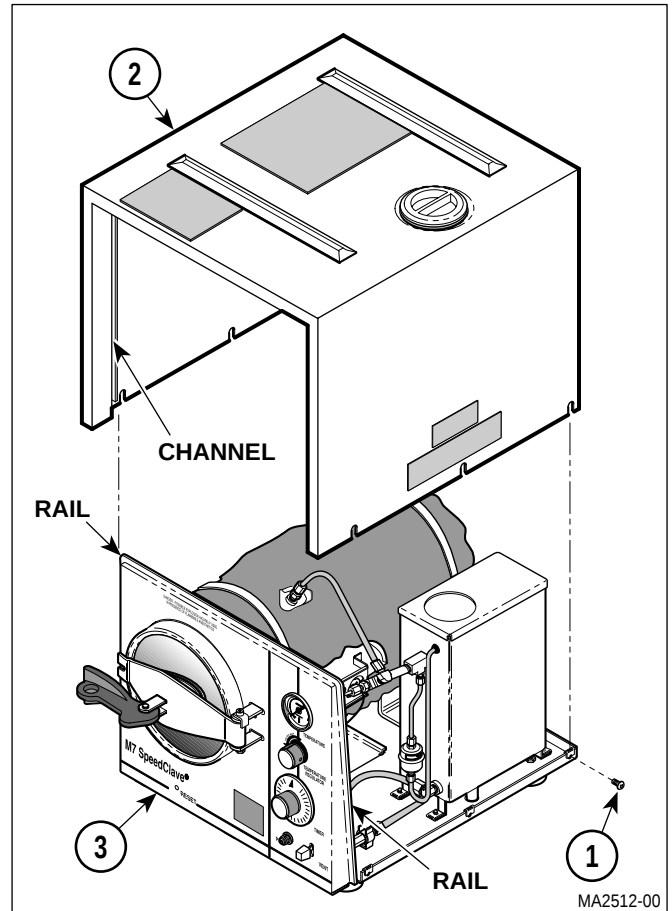


**Figure 4-1. Cover Assembly Removal / Installation**

### 4.3 Timer Assembly Removal / Installation (New Style Timer Assembly)

#### A. Removal

- (1) Remove cover assembly (Refer to para 4.2).
- (2) Remove the timer knob (1, Figure 4-2) from shaft of timer assembly (2) by pulling straight out on timer knob.
- (3) Tag and disconnect four wires (3) from terminals of timer assembly (2).

## SECTION IV MAINTENANCE / SERVICE

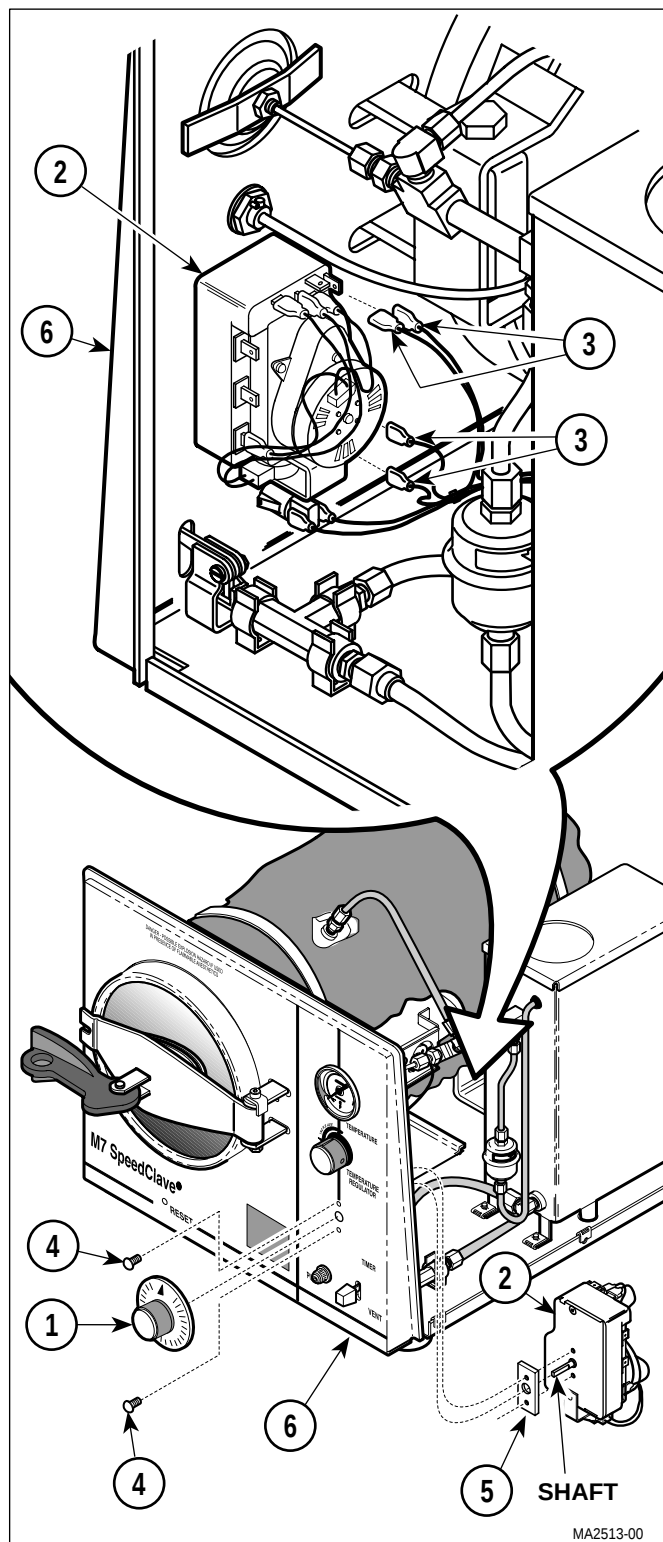

**Figure 4-2. Timer Assembly Removal / Installation**

### NOTE

Spacer is only used on newer units.

- (4) Remove two screws (4), timer assembly (2), and spacer (5) from front panel (6).

### B. Installation

### NOTE

Spacer is only used on newer units

- (1) Install spacer (5) and timer assembly (2) on front panel (6) and secure with two screws (4).
- (2) Connect four wires (3) to terminals of timer assembly (2).
- (3) Install timer knob (1) on shaft of timer assembly (2).
- (4) Install cover assembly (Refer to para 4.2).

## 4.4 Timer Assembly Removal / Installation (Old Style Timer Assembly)

### A. Removal

- (1) Remove cover assembly (Refer to para 4.2).
- (2) Remove the timer knob (1, Figure 4-3) from shaft of timer assembly (2) by pulling straight out on timer knob.
- (3) Tag and disconnect four wires (3) from terminals of timer assembly (2).
- (4) Remove nut (4) and timer assembly (2) from front panel (5).

### B. Installation

- (1) Align locator pin with locating hole; then install timer assembly (2) on front panel (5) and secure with nut (4).
- (2) Connect four wires (3) to terminals of timer assembly (2).
- (3) Install timer knob (1) on shaft of timer assembly (2).
- (4) Install cover assembly (Refer to para 4.2).


#### 4.5 Timer Buzzer Removal / Installation (Only On Units With New Style Timer Assembly)

##### A. Removal


- (1) Remove cover assembly (Refer to para 4.2).
- (2) Remove timer assembly (Refer to para 4.3).
- (3) Tag and disconnect two wires (1, Figure 4-4) from timer assembly (2).
- (4) Remove screw (3) and timer buzzer (4) from timer assembly (2).

##### B. Installation

- (1) Install timer buzzer (4) on timer assembly (2) and secure with screw (3).
- (2) Connect two wires (1) to timer assembly (2).
- (3) Install timer assembly (Refer to para 4.3).
- (4) Install cover assembly (Refer to para 4.2).

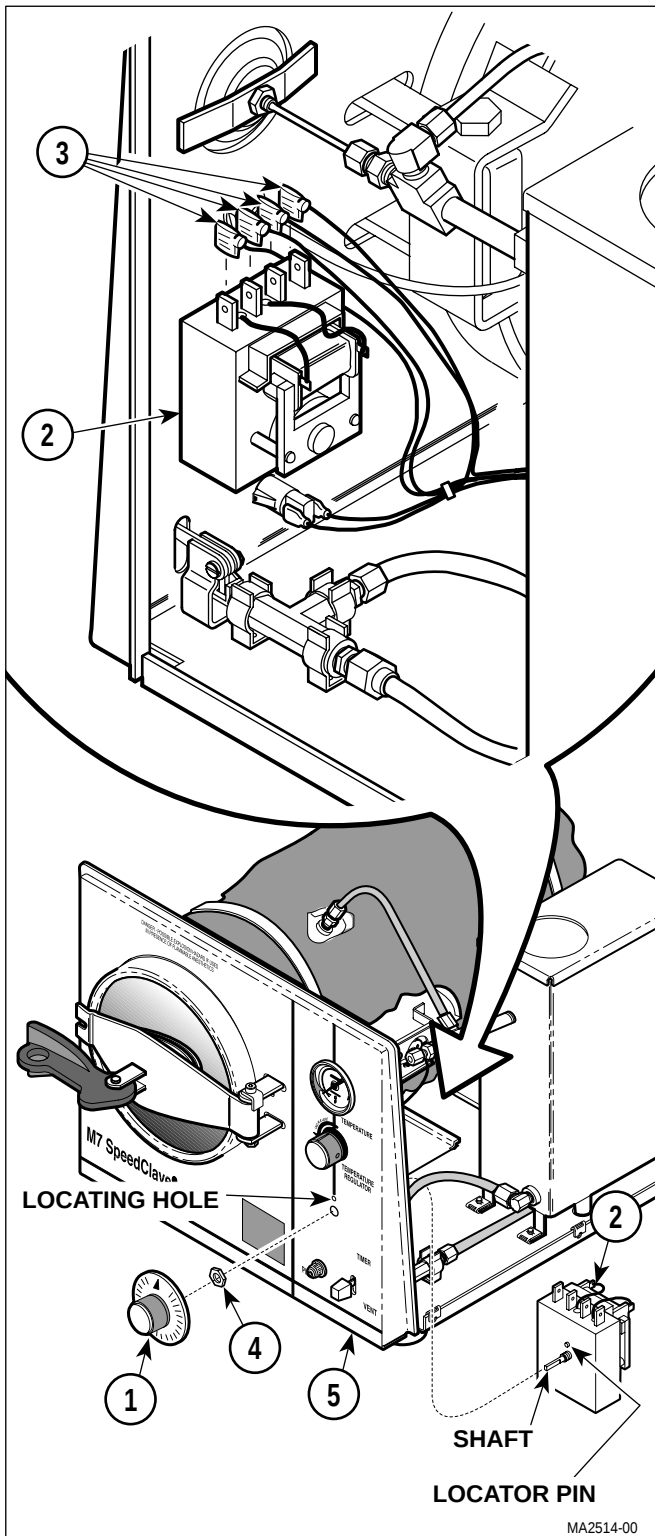

**Figure 4-3. Timer Assembly Removal / Installation**

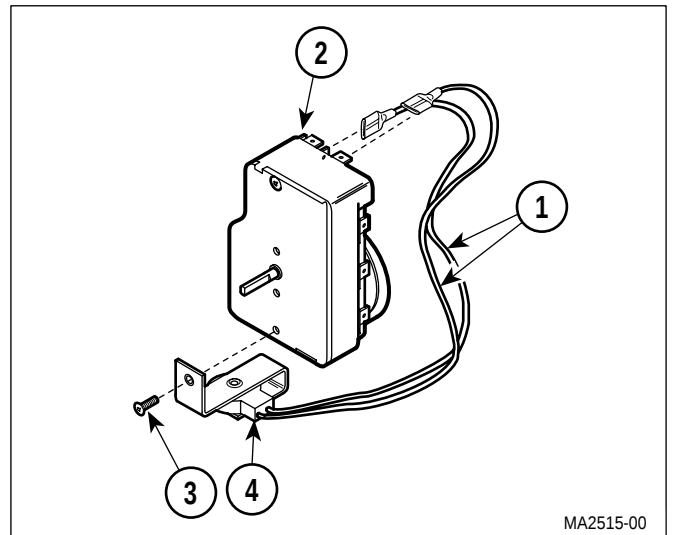

**Figure 4-4. Timer Buzzer Removal / Installation**

## SECTION IV MAINTENANCE / SERVICE

### 4.6 Pilot Light Removal / Installation

#### A. Removal

- (1) Remove cover assembly (Refer to para 4.2).
- (2) Disconnect two wires (1, Figure 4-5) from terminals of pilot light (2).

#### NOTE

Pilot lights on older units do not have the four tabs. These older pilot lights are held in by friction only or by a retaining ring.

- (3) Press in on four tabs of pilot light (2) while simultaneously removing pilot light from front panel (3).

#### B. Installation

#### NOTE

Pilot lights on older units do not have the four tabs. These older pilot lights are held in by friction only or by a retaining ring.

- (1) Push pilot light (2) into front panel (3) until it pops into place.
- (2) Connect two wires (1) to terminals of pilot light (2).
- (3) Install cover assembly (Refer to para 4.2).

### 4.7 Heating Element Removal / Installation

#### A. Removal



#### DANGER

Always unplug the power cord from the wall outlet before removing any of the sterilizers cover / panels or making any repairs to prevent the possibility of electrical shock. Failure to comply with these instructions could result in severe personal injury or death.

- (1) Disconnect power cord from wall outlet.
- (2) Unscrew drain (1, Figure 4-6) and drain all water from condensing tank.
- (3) Remove wire rack and trays (Refer to para 4.20).


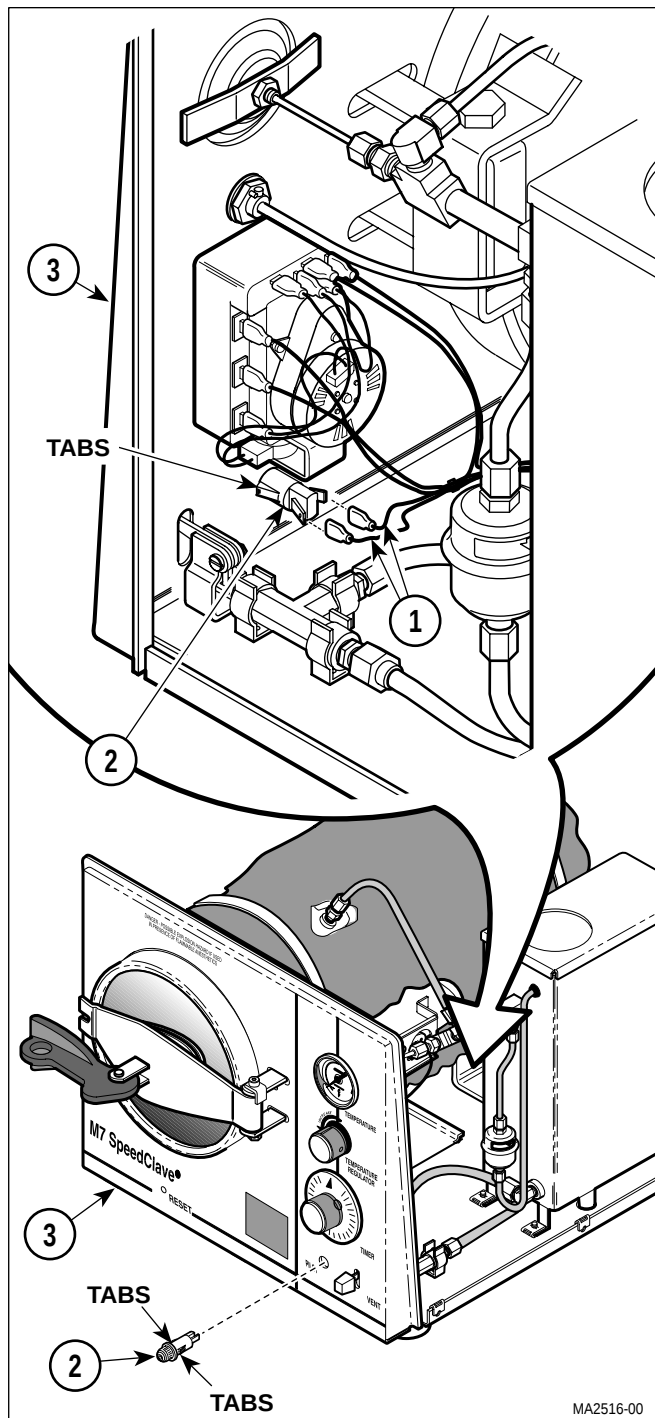


Figure 4-5. Pilot Light Removal / Installation

- (4) Lay unit onto its side.
- (5) Remove two screws (2) and inspection cover (3) from base (4).
- (6) Taking care not to kink rod (5), flex rod and remove it from rocker bracket (6).

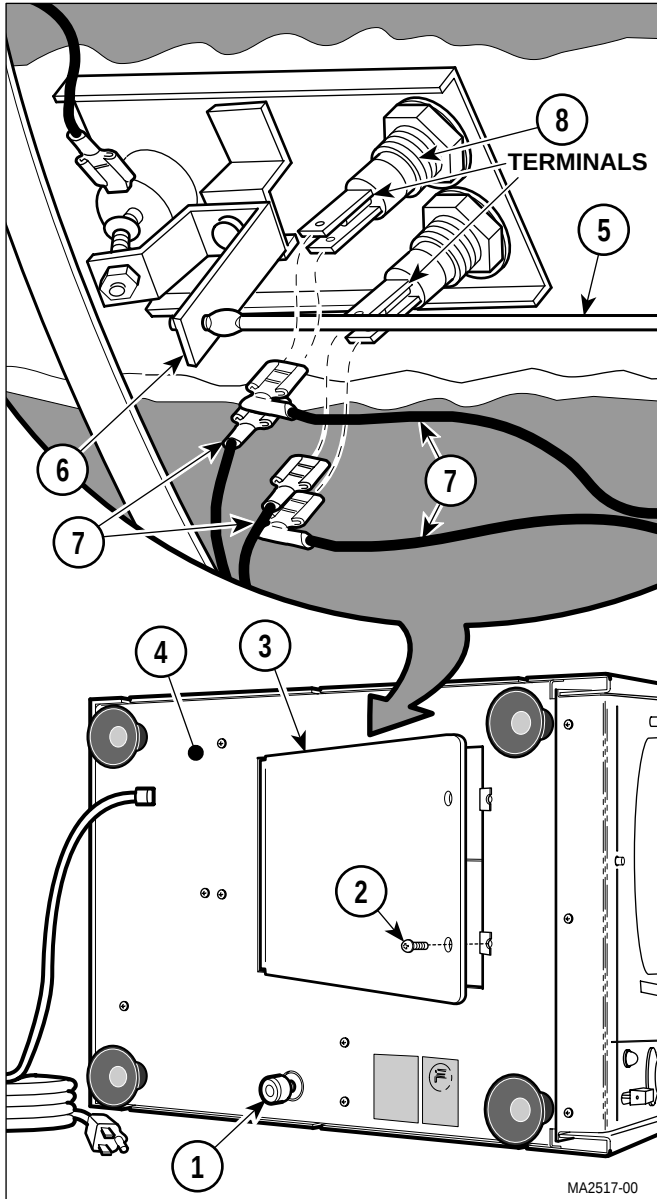

**Figure 4-6. Inspection Cover Removal / Installation**


- (7) Tag and disconnect four wires (7) from terminals of heating element (8).

**NOTE**

Step 8 applies only to units which have an old style heating element. Units with a new style heating element have built in terminals.

- (8) If unit has an old style heating element, remove nut (1, Figure 4-7), two terminals (2), and nut (3) from each terminal post of heating element (4). Discard nuts and terminals - will receive new style heating element.

- (9) Remove two nuts (1, Figure 4-8), lockwashers (2), and washers (3) from terminal posts of heating element (4).
- (10) Pull bracket (5) off of terminal posts of heating element (4).
- (11) Remove heating element (4) and spacer (6) from inside of chamber.
- (12) Remove one gasket (7) from each terminal post of heating element (4).

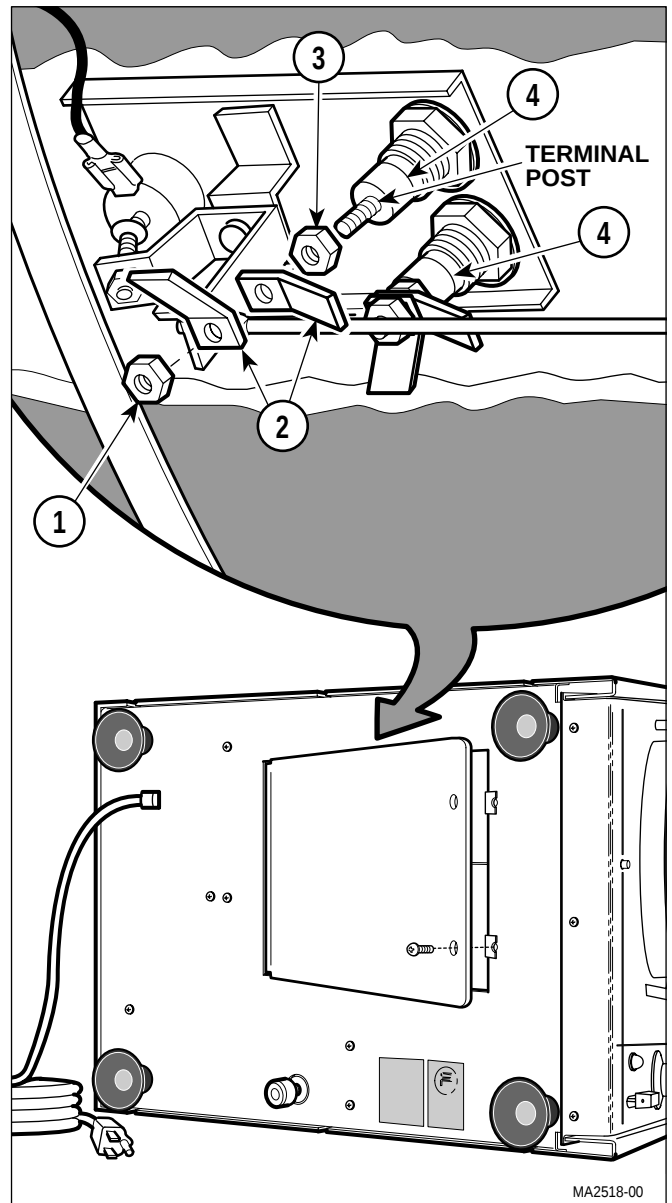

**Figure 4-7. Removal Of Terminals From Old Style Heating Element**

## SECTION IV MAINTENANCE / SERVICE

## B. Installation

- (1) Install one gasket (7, Figure 4-8) on each terminal post of heating element (4).



## CAUTION

 Hold heating element firmly in position while tightening nuts. Otherwise, damage to heating element or improper positioning of heating element may result. Also, spacer must remain above gaskets. If spacer is installed under gasket, leaking will result.

## NOTE

Flat side of nut should be facing lockwasher.

- (2) Install spacer (6) and heating element (4) on chamber wall and secure with bracket (5), two washers (3), lockwashers (2), and nuts (1).
- (3) Connect four wires (7, Figure 4-6) to terminals of heating element (8).
- (4) Taking care not to kink rod (5), flex rod and insert it into rocker bracket (6).
- (5) Install inspection cover (3) on base (4) and secure with two screws (2).
- (6) Turn unit upright.
- (7) Install wire rack and trays (Refer to para 4.20).
- (8) Tighten drain (1).
- (9) Connect power cord to wall outlet.
- (10) Fill condensing tank with distilled or demineralized water.


## 4.8 Overheat Thermostat Removal / Installation

### A. Removal




# DANGER

 **Always unplug the power cord from the wall outlet before removing any of the sterilizers cover / panels or making any repairs to prevent the possibility of electrical shock. Failure to comply with these instructions could result in severe personal injury or death.**

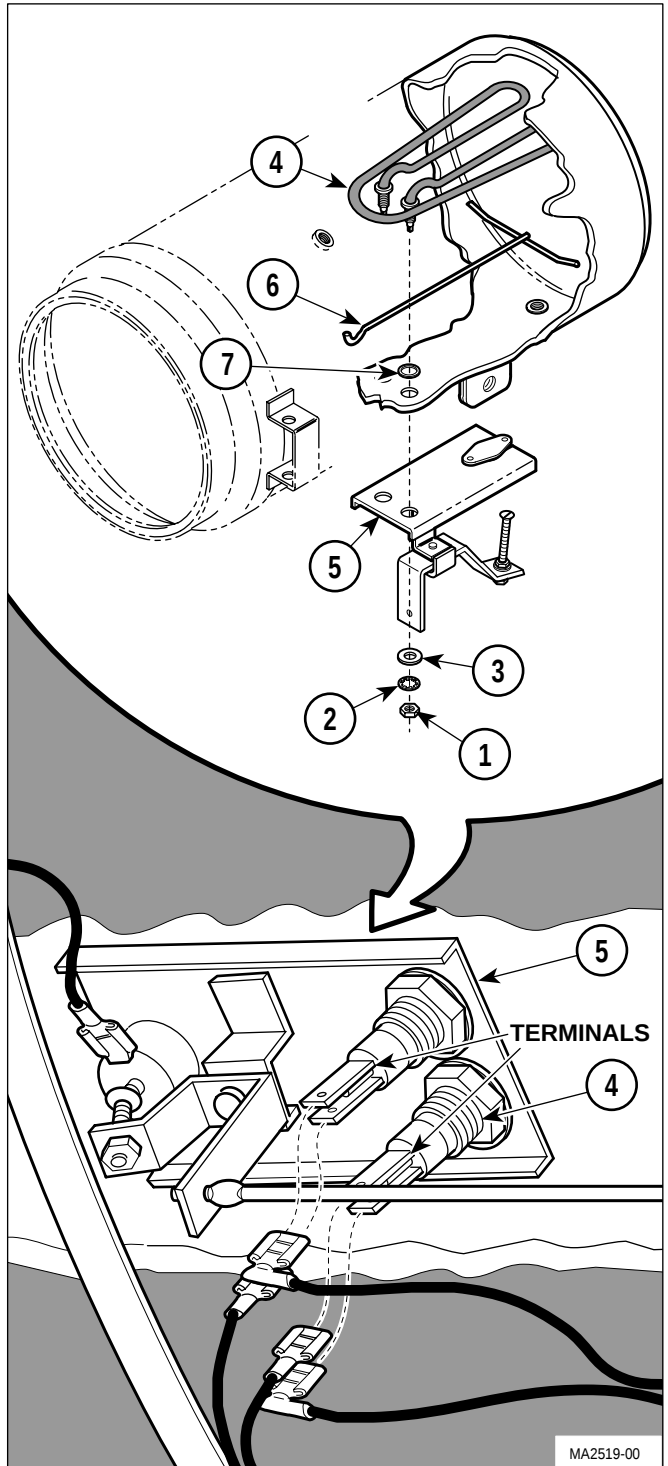

### Figure 4-8. Heating Element Removal / Installation

- (1) Disconnect power cord from wall outlet.
- (2) Unscrew drain (1, Figure 4-9) and drain all water from condensing tank.

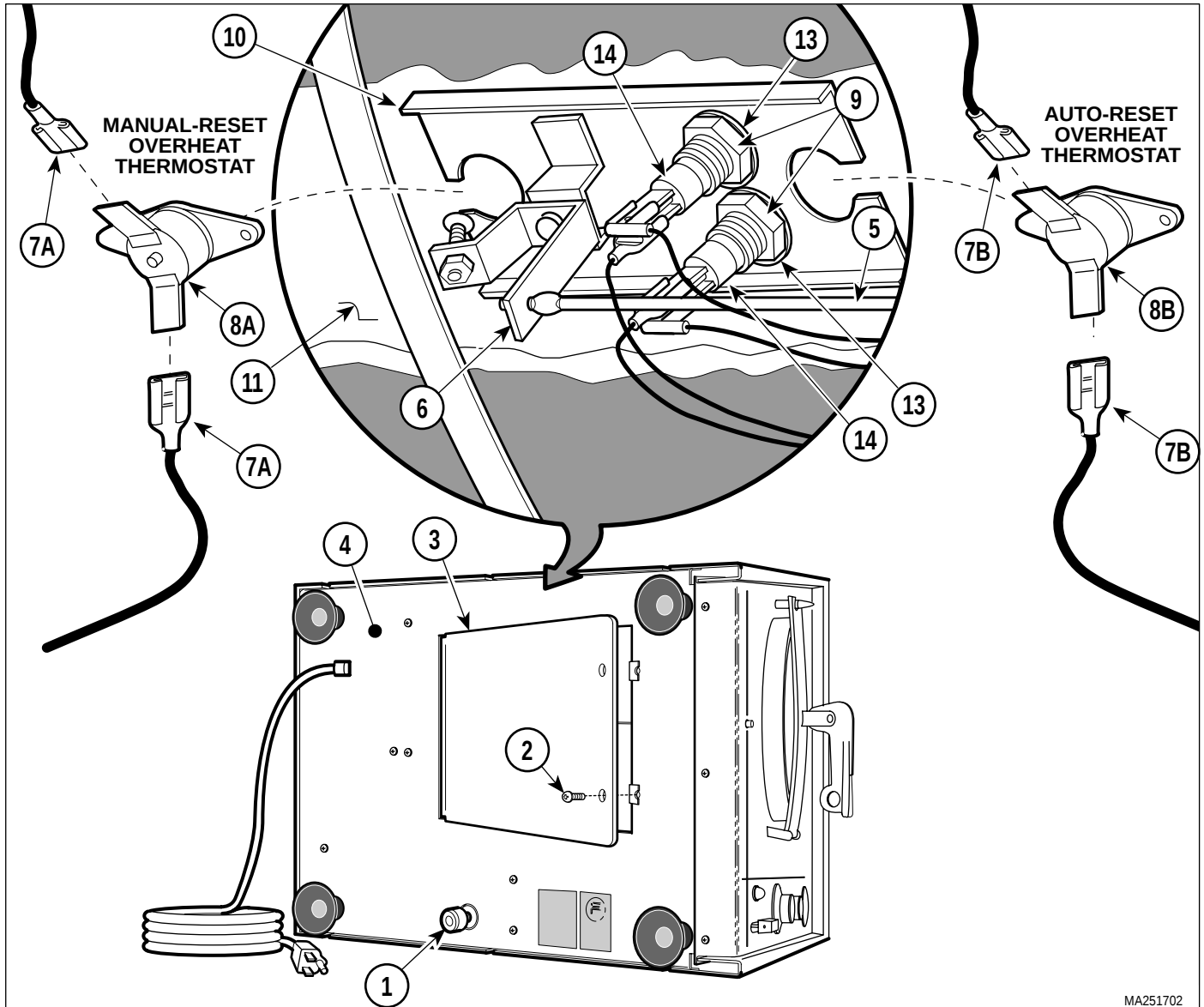

MA251702

**Figure 4-9. Overheat Thermostat Removal / Installation**

- (3) Remove wire rack and trays (Refer to para 4.20).
- (4) Lay unit onto its side.
- (5) Remove two screws (2) and inspection cover (3) from base (4).
- (6) Taking care not to kink rod (5), flex rod and remove it from rocker bracket (6).
- (7) If removing manual reset overheat thermostat (8A), tag and disconnect two wires (7A) from terminals of overheat thermostat (8A).  
  
If removing auto-reset overheat thermostat (8B), tag and disconnect two wires (7B) from terminals of overheat thermostat (8B).
- (8) Loosen two nuts (9) and partially separate bracket (10) from chamber wall (11).
- (9) Remove overheat thermostat (8A or 8B) from bracket (10).

**NOTE**

Only units with Serial Numbers CS5297 & RB14565 Thru Present also have auto-reset overheat thermostats.

## SECTION IV MAINTENANCE / SERVICE

### B. Installation

- (1) Install manual reset overhear thermostat (8A) or auto-reset overhear thermostat (8B) on bracket (10).



#### CAUTION

Hold heating element firmly in position while tightening nuts. Otherwise, damage to heating element or improper positioning of heating element may result. Also, spacer must remain above gaskets. If spacer is installed under gasket, leaking will result.

- (2) Position bracket (10) on chamber wall (11) and secure by tightening two nuts (9), making sure spacer (inside of chamber) is positioned under heating element properly.
- (3) Connect two wires (7A) to terminals of manual reset overhear thermostat (8A) **or** connect two wires (7B) to terminals of auto-reset overhear thermostat (8B).
- (4) Taking care not to kink rod (5), flex rod and insert it into rocker bracket (6).
- (5) Install inspection cover (3) on base (4) and secure with two screws (2).
- (6) Turn unit upright.
- (7) Install wire rack and trays (Refer to para 4.20).
- (8) Tighten drain (1).
- (9) Connect power cord to wall outlet.
- (10) Fill condensing tank with distilled or demineralized water.

### 4.9 Temperature Regulator Relay Removal / Installation

#### A. Removal

- (1) Remove cover assembly (Refer to para 4.2).
- (2) Remove setscrew (1, Figure 4-10) from hub of flexible shaft assembly (2).
- (3) Rotate TEMPERATURE REGULATOR knob in a counterclockwise direction until the flexible

shaft assembly (2) is unscrewed from temperature regulator relay (3).

- (4) Tag and disconnect one wire (4) from the timer assembly (5) and one wire (6) from the heating element (7).

#### NOTE

Relay spacer is not used on older 220 VAC units.

- (5) Remove two screws (8A), one screw (8B), temperature regulator relay (3), and relay spacer (9) from relay bracket (10).


### B. Installation

#### NOTE

Relay spacer is not used on older 220 VAC units.

Screw (8B) is used for adjustment and will be installed later.

- (1) Install relay spacer (9) and temperature regulator relay (3) on relay bracket (10) and secure with two screws (8A). Do not install screw (8B) at this time.
- (2) Connect one wire (6) to terminal of heating element (7) and one wire (4) to terminal of timer assembly (5).
- (3) Adjust setscrew (11) until it is flush with relay bracket (10).
- (4) Coat threaded end of flexible shaft assembly (2) with high temperature grease.
- (5) Screw threaded end of flexible shaft assembly (2) into temperature regulator relay (3) by turning TEMPERATURE REGULATOR knob in clockwise direction. Keep rotating TEMPERATURE REGULATOR knob in clockwise direction until flexible shaft assembly is screwed in as far as possible.
- (6) Rotate TEMPERATURE REGULATOR knob  $1\frac{1}{2}$  turns in a counterclockwise direction.
- (7) Install setscrew (1) in hub of flexible shaft assembly (2).
- (8) Rotate TEMPERATURE REGULATOR knob in a counterclockwise direction until it hits the setscrew stop.

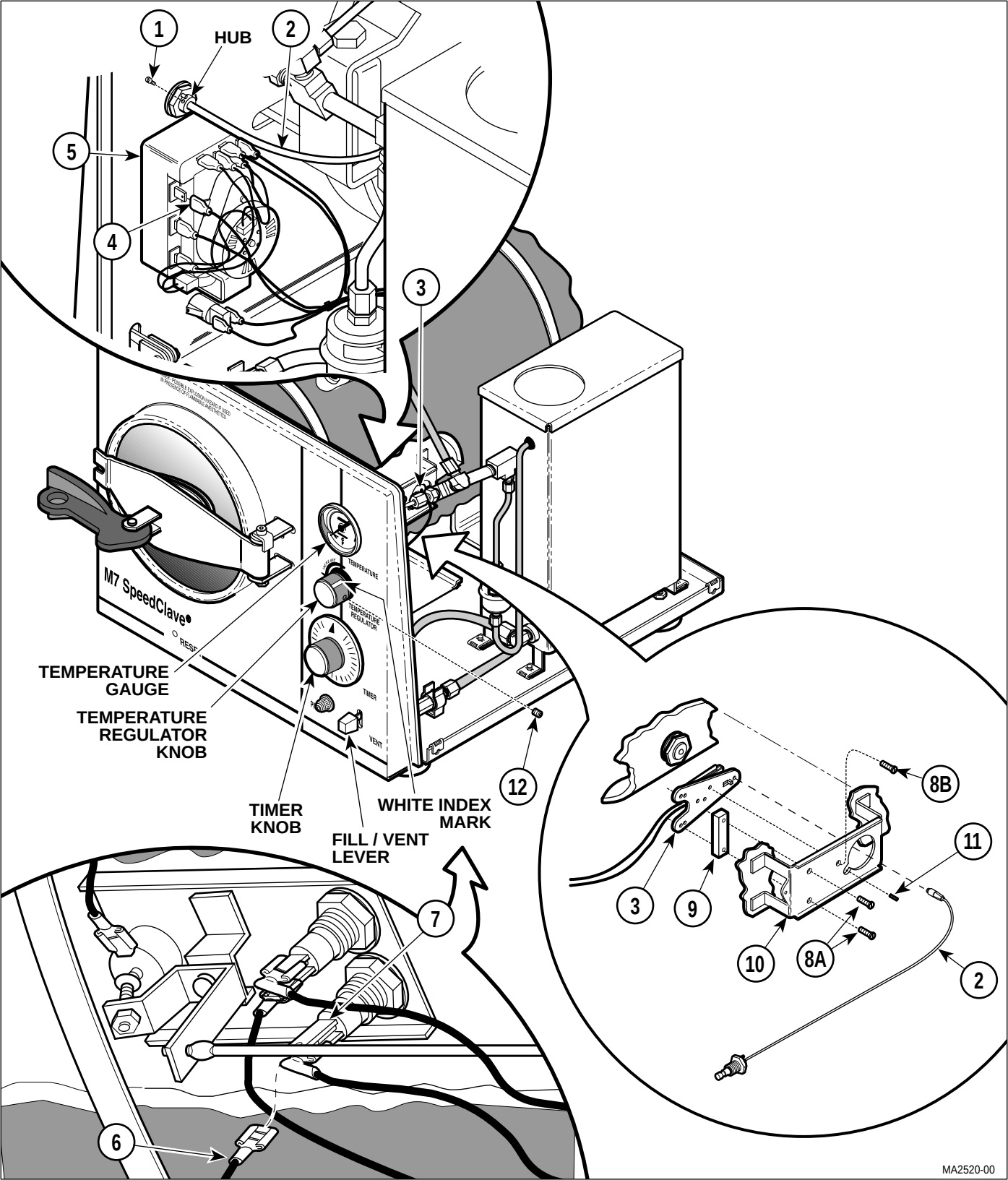

MA2520-00

**Figure 4-10. Temperature Regulator Relay  
Removal / Installation**

## SECTION IV MAINTENANCE / SERVICE

### NOTE

Earlier units have only one setscrew securing the TEMPERATURE REGULATOR knob while later units have two.

- (9) Making sure TEMPERATURE REGULATOR knob is held against setscrew stop, loosen one/two setscrews (12) in TEMPERATURE REGULATOR knob. Rotate TEMPERATURE REGULATOR knob until white index mark is at the 9:00 o'clock position, then tighten one/two setscrews (12).
- (10) Depress the FILL / VENT lever and allow the water to fill the chamber until the water level is within 1/2 - 5/8 in. (13 - 16 mm) from the front rim of the chamber; then release the lever.



### DANGER

The following steps require the unit to be powered up with the cover assembly removed. Use extreme care to prevent contact with exposed terminals and chamber components. Failure to do so could result in electrical shock or burns which could cause serious personal injury or death.

- (11) Plug the power cord into wall outlet.
- (12) Close and latch the door of the chamber.
- (13) Set the TIMER knob to its maximum setting of 30 minutes.
- (14) Unscrew setscrew (11) 2 to 3 turns.

### NOTE

Wait until chamber temperature reaches its maximum temperature before attempting to adjust screw.

- (15) Install screw (8B). Adjust screw until TEMPERATURE gauge reads slightly above 270 °F (132 °C).

### NOTE

Setscrew provides fine tuning of temperature adjustment.

- (16) Adjust setscrew (11) until TEMPERATURE gauge reads 270 - 271 °F (131.6 - 132.8 °C).

- (17) After adjustment is satisfactory, turn TIMER knob to off and vent the chamber.

- (18) Install cover assembly (Refer to para 4.2).

## 4.10 Temperature Regulator Relay Adjustment

### A. Removal

- (1) Remove cover assembly (Refer to para 4.2).
- (2) Remove setscrew (1, Figure 4-11) from hub of flexible shaft assembly (2).

### NOTE


Relay spacer is not used on older 220 VAC units.

- (3) Remove screw (3).
- (4) Adjust setscrew (4) until it is flush with relay bracket (5).
- (5) Rotate TEMPERATURE REGULATOR knob in clockwise direction until flexible shaft assembly (2) is screwed in as far as possible.
- (6) Rotate TEMPERATURE REGULATOR knob 1½ turns in a counterclockwise direction.
- (7) Install setscrew (1) in hub of flexible shaft assembly (2).
- (8) Rotate TEMPERATURE REGULATOR knob in a counterclockwise direction until it hits the setscrew stop.

### NOTE

Earlier units have only one setscrew securing the TEMPERATURE REGULATOR knob while later units have two.

- (9) Making sure TEMPERATURE REGULATOR knob is held against stop, loosen one/two setscrew(s) (6) in TEMPERATURE REGULATOR knob. Rotate TEMPERATURE REGULATOR knob until white index mark is at the 9:00 o'clock position, then tighten one/two setscrew(s) (6).

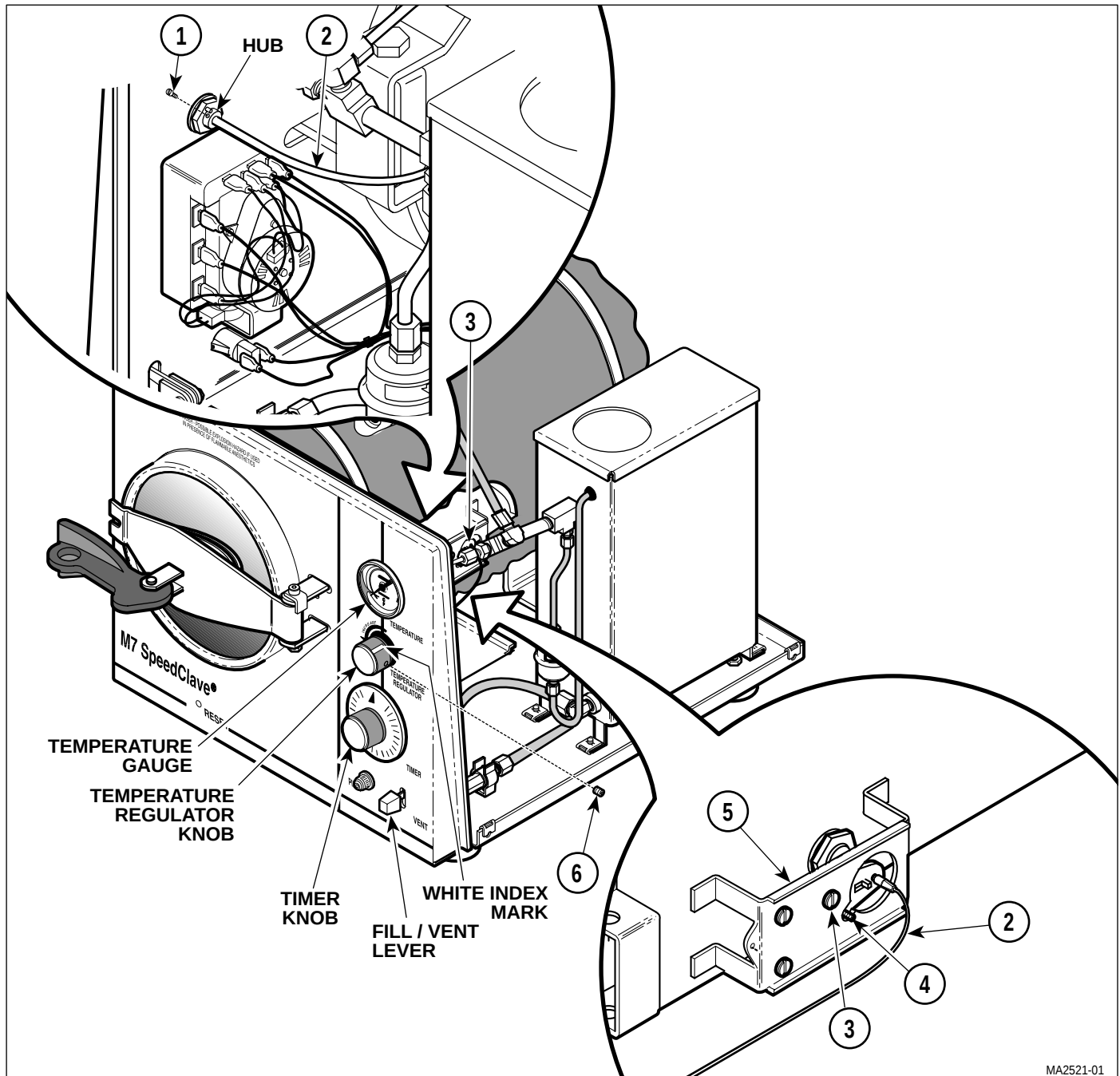

MA2521-01

**Figure 4-11. Temperature Regulator  
Relay Adjustment**

- (10) Depress the FILL / VENT lever and allow the water to fill the chamber until the water level is within 1/2 - 5/8 in. (13 - 16 mm) from the front rim of the chamber; then release the lever.

## SECTION IV MAINTENANCE / SERVICE



### DANGER

The following steps require the unit to be powered up with the cover assembly removed. Use extreme care to prevent contact with exposed terminals and chamber components. Failure to do so could result in electrical shock or burns which could cause serious personal injury or death.

- (11) Plug the power cord into wall outlet.
- (12) Close and latch the door of the chamber.
- (13) Set the TIMER knob to its maximum setting of 30 minutes.
- (14) Unscrew setscrew (4) 2 to 3 turns.

### NOTE

Wait until chamber temperature reaches as high as it is going to go before attempting to adjust.

- (15) Install screw (3). Adjust screw until TEMPERATURE gauge reads slightly above 270 °F (132 °C).

### NOTE

Setscrew provides fine tuning of temperature adjustment.

- (16) Adjust setscrew (4) until TEMPERATURE gauge reads 270 - 271 °F (131.6 - 132.8 °C).
- (17) After adjustment is satisfactory, turn TIMER knob to off and vent the chamber.
- (18) Install cover assembly (Refer to para 4.2).

## 4.11 Diaphragm Cup Removal / Installation

### A. Removal

- (1) Remove cover assembly (Refer to para 4.2).
- (2) Remove wire rack and trays (Refer to para 4.20).
- (3) Remove temperature regulator relay (Refer to para 4.9).

- (4) Remove nut (1, Figure 4-12), lockwasher (2), and diaphragm cup (3) from chamber wall (4).
- (5) Remove gasket (5) from diaphragm cup (3).

### B. Installation

- (1) Install gasket (5) on diaphragm cup (3).



### CAUTION

The lip of the nut must face toward the lockwasher.

- (2) Install diaphragm cup (3) on chamber wall (4) and secure with lockwasher (2) and nut (1).
- (3) Install wire rack and trays (Refer to para 4.20).
- (4) Install temperature regulator relay (Refer to para 4.9).
- (5) Install cover assembly (Refer to para 4.2).

## 4.12 Temperature Gauge Removal / Installation

### A. Removal

- (1) Remove cover assembly (Refer to para 4.2).
- (2) Loosen nut (1, Figure 4-13) and slide retaining bar (2) away from front panel (3).


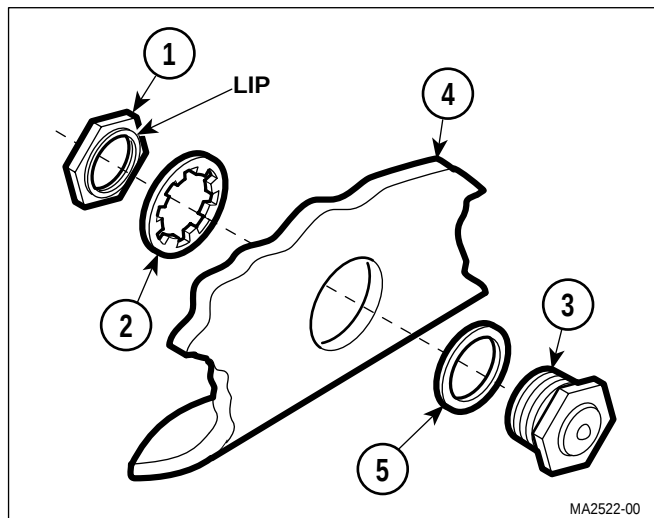


Figure 4-12. Diaphragm Cup Removal / Installation

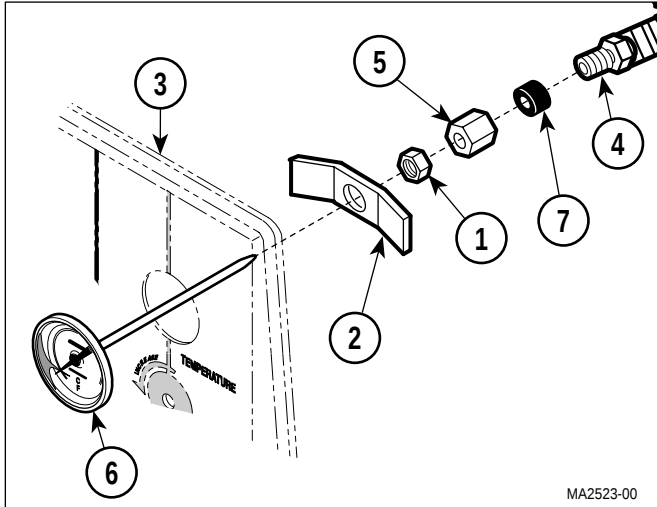

**Figure 4-13. Temperature Gauge Removal / Installation**

- (3) While holding connector (4), loosen compression nut (5).
- (4) Pull temperature gauge (6) out of connector (4).
- (5) Remove thermometer sleeve (7), compression nut (5), nut (1), and retaining bar (2) from temperature gauge (6).

**B. Installation**

- (1) Install retaining bar (2), nut (1), compression nut (5), and thermometer sleeve (7) on temperature gauge (6).
- (2) Making sure temperature gauge is oriented properly so face of gauge can be read by operator, position temperature gauge (6) on front panel (3) and secure with retaining bar (2) and nut (1).
- (3) Slide thermometer sleeve (7) against connector (4).
- (4) Install compression nut (5) on connector (4). Tighten compression nut until thermometer sleeve just starts to protrude out back side of compression nut. Do not tighten any further.
- (5) Install cover assembly (Refer to para 4.2).

### 4.13 Pressure Relief Valve Removal / Installation

**A. Removal**

- (1) Remove cover assembly (Refer to para 4.2).

**NOTE**

There may be a strip of tape holding the tank cover on to prevent the tank cover from vibrating and making noise.

- (2) Remove tank cover (1, Figure 4-14) from tank assembly (2).

**NOTE**

The pressure relief valve can be mounted to tee (4) in two different locations depending if it is an old style or new style unit.

- (3) Remove pressure relief valve (3) from tee (4).

**B. Installation**


- (1) Coat threads of pressure relief valve (3) with Teflon tape / sealant.
- (2) Install pressure relief valve (3) on tee (4).
- (3) Install tank cover (1) on tank assembly (2).
- (4) Install cover assembly (Refer to para 4.2).

**NOTE**

The pressure relief valve is designed to open at 31 psi / 275 °F (135 °C).

- (5) With the TEMPERATURE REGULATOR knob set to its maximum setting, run a cycle. The temperature should reach normal maximum pressure / temperature of 27 psi / 270 °F (132 °C). If not, it indicates that the pressure relief valve is opening too early.

## SECTION IV MAINTENANCE / SERVICE

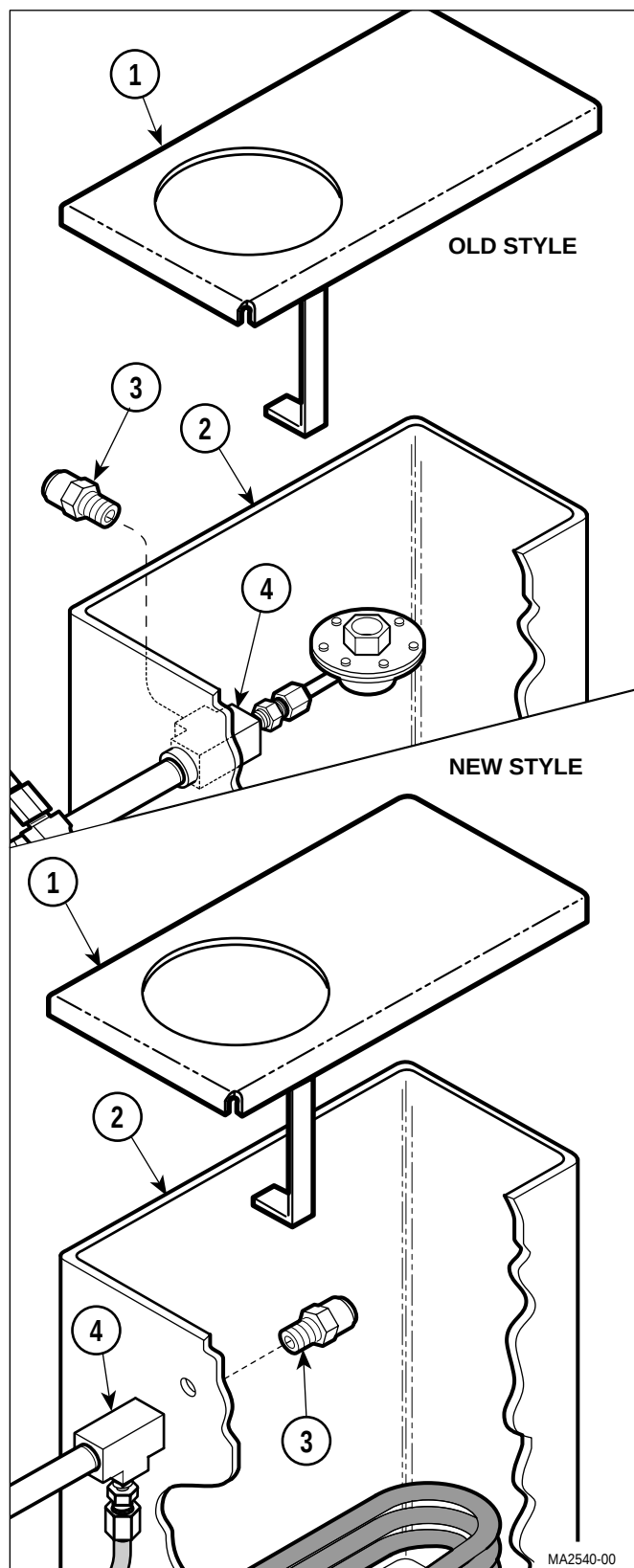

**Figure 4-14. Pressure Relief Valve  
Removal / Installation**

### 4.14 Bellows Assembly Removal Installation (Applies to Units With Serial Numbers CR-1114, RB-7194, CS-1570, And CP-1062 Thru Present)

#### A. Removal

- (1) Remove cover assembly (Refer to para 4.2).




#### **DANGER**

**Make sure unit is vented and cool before attempting to make repairs. Failure to do so could result in serious injury or burns.**

- (2) Using a wrench on bellows nuts to hold bellows assembly (1, Figure 4-15) stationary, loosen and disconnect compression nuts (2 and 3).
- (3) Remove bellows assembly (1) from tubes (4).

#### B. Installation

- (1) Position bellows assembly (1) on tubes (4), making sure flow arrow is pointing toward compression nut (3).

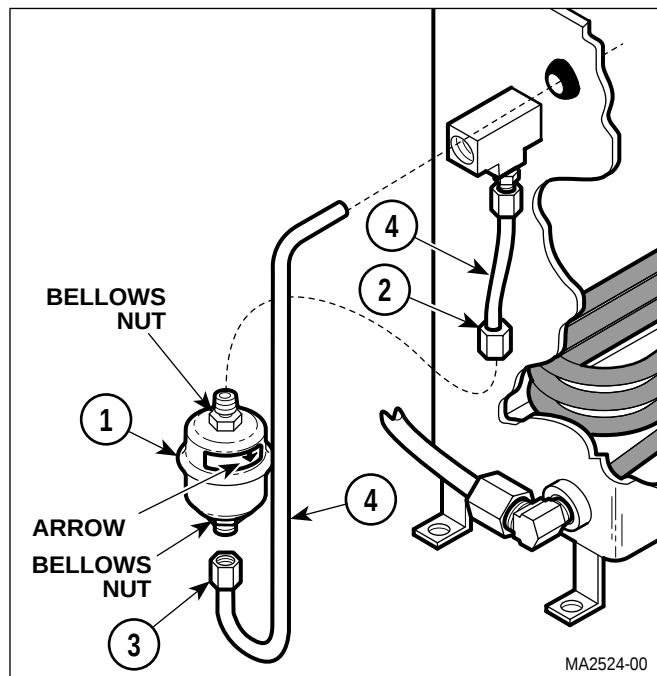

**Figure 4-15. Bellows Assembly  
Removal / Installation**

- (2) Using a wrench on bellows nuts to hold bellows assembly (1) stationary, screw compression nuts (2 and 3) onto bellows assembly.
- (3) Install cover assembly (Refer to para 4.2).

#### **4.15 Bellows Assembly Removal / Installation (Applies To Units With Serial Numbers CP-1000 Thru CP-1061, CR-1000 Thru CR-1113, CS-1000 Thru CS-1569, And RB-1000 Thru RB-7193)**

##### **A. Removal**

- (1) Remove cover assembly (Refer to para 4.2).



#### **DANGER**

**Make sure unit is vented and cool before attempting to make repairs.**

**Failure to do so could result in serious injury or burns.**

#### **NOTE**


There may be a strip of tape holding the tank cover on to prevent the tank cover from vibrating and making noise.

- (2) Remove tank cover (1, Figure 4-16) from tank assembly (2).
- (3) Using a wrench on bellows nut to hold bellows assembly (3) stationary, loosen and disconnect compression nut (4).
- (4) Remove bellows assembly (3) from tube (5).

##### **B. Installation**

- (1) Position bellows assembly (3) on tube (5), making sure flow arrow is pointing away from compression nut (4).
- (2) Using a wrench on bellows nut to hold bellows assembly (3) stationary, screw compression nut (4) onto bellows assembly.

- (3) Install tank cover (1) on tank assembly (2).
- (4) Install cover assembly (Refer to para 4.2).

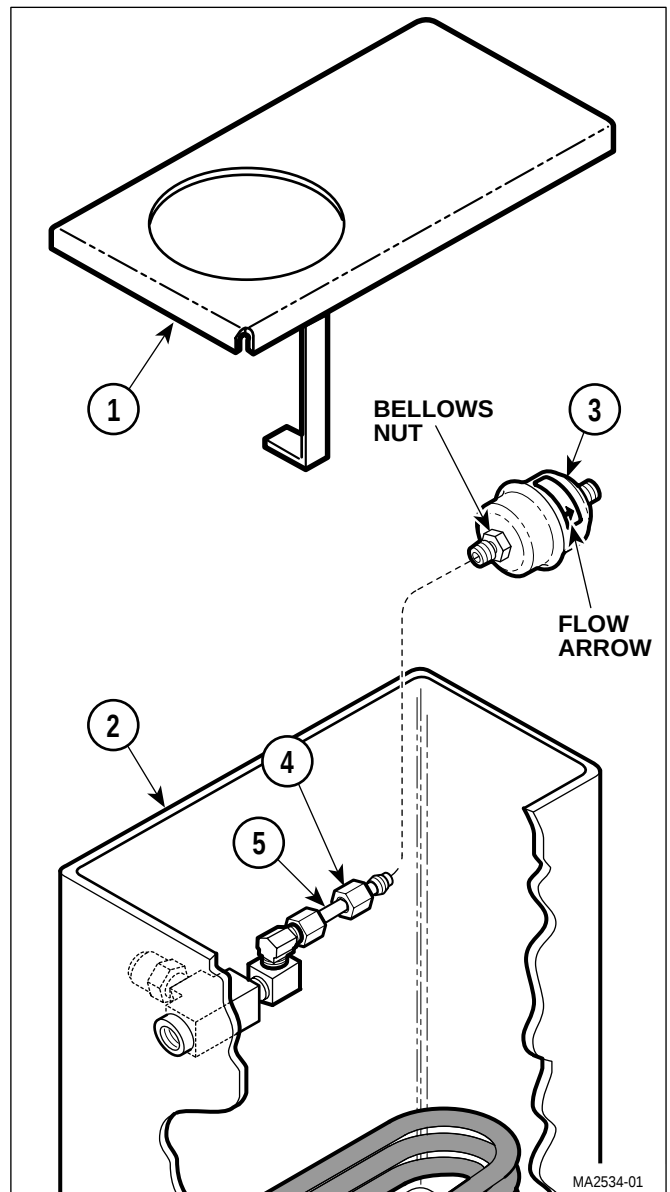

**Figure 4-16. Bellows Assembly Removal / Installation**

## SECTION IV MAINTENANCE / SERVICE

### 4.16 Bellows Assembly Removal / Installation (Sybron Units)

#### NOTE

Older units which do not have a Serial Number prefix of CP-xxxx, CR-xxxx, CS-xxxx, or RB-xxxx were manufactured by Sybron Corp and not by Midmark. However, a retrofit kit for the bellows assembly is available. If the unit has already been retrofitted with a Midmark style bellows, perform para 4.15.

#### A. Removal

- (1) Remove cover assembly (Refer to para 4.2).



#### DANGER


**Make sure unit is vented and cool before attempting to make repairs.**

**Failure to do so could result in serious injury or burns.**

- (2) Remove tank cover (1, Figure 4-17) from tank assembly (2).
- (3) Loosen compression nut (3) and remove bellows assembly (4) from male connector (5).
- (4) Remove male connector (5) from tee (6).

#### B. Installation

- (1) Coat threads of largest elbow in elbow assembly (7) with Teflon tape / sealant.
- (2) Install elbow assembly (7) on tee (6).
- (3) Install compression nut (8) and compression sleeve (9) on tube of bellows assembly (10).
- (4) Connect tube of bellows assembly (10) to elbow assembly (7) by screwing compression nut (9) onto elbow assembly (7).
- (5) Install tank cover (1) on tank assembly (2).
- (6) Install cover assembly (Refer to para 4.2).

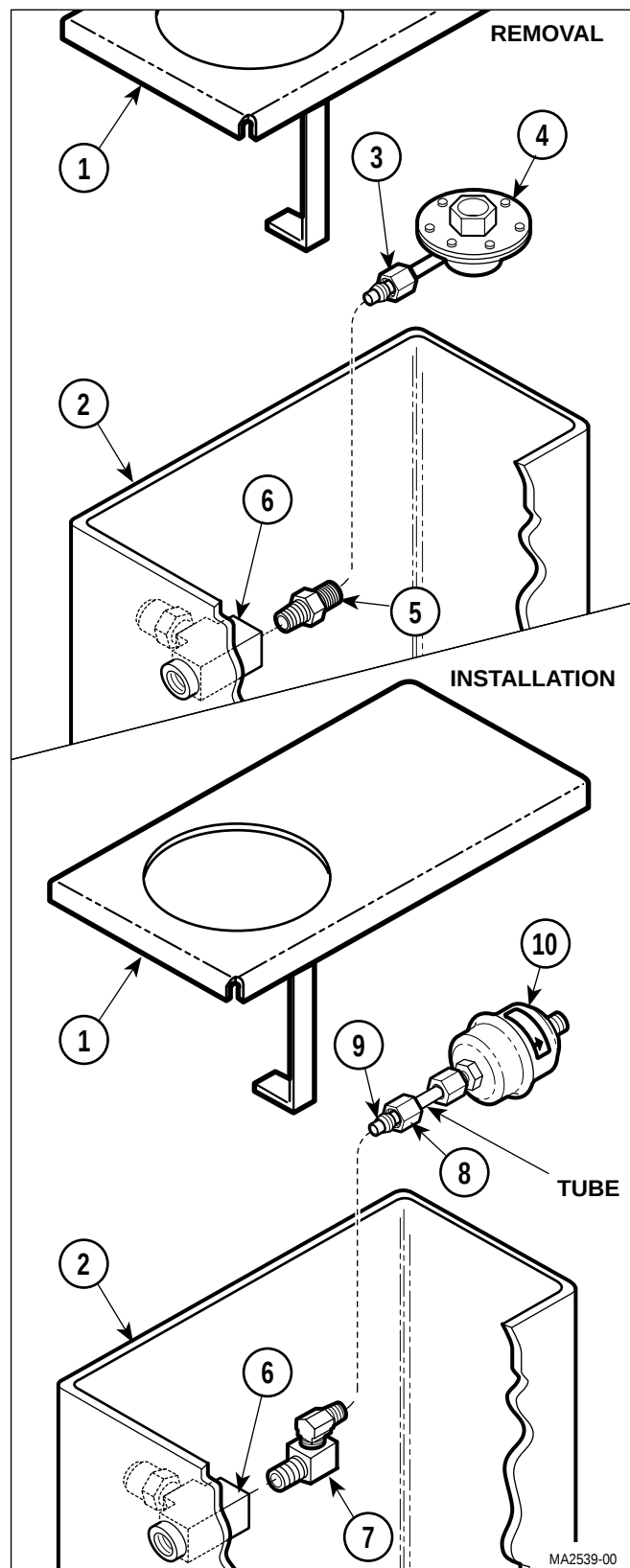

**Figure 4-17. Bellows Assembly  
Removal / Installation**

## 4.17 Fill / Vent Valve Removal / Installation

### A. Removal

- (1) Remove cover assembly (Refer to para 4.2).




#### **DANGER**

**Make sure unit is vented and cool before attempting to make repairs. Failure to do so could result in serious injury or burns.**


- (2) Loosen and disconnect two compression nuts (1, Figure 4-18) from fittings of fill / vent valve assembly (2).
- (3) Disengage closing bracket of fill / vent valve assembly (2) from fill / vent lever (3) by lifting up on back end of fill / vent valve assembly first and then pulling it out of brackets (4).

### B. Disassembly

- (1) Remove fitting (1, Figure 4-19) from valve body (2).

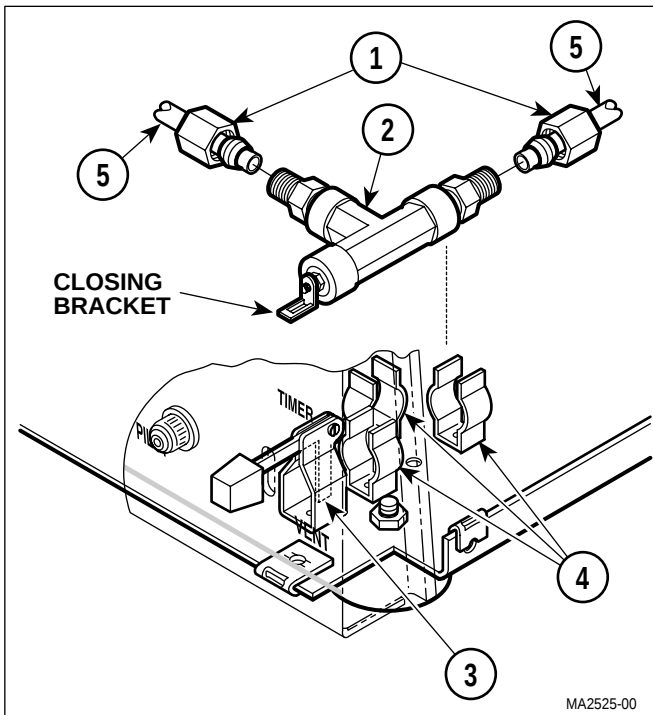

**Figure 4-18. Fill / Vent Valve Removal / Installation**

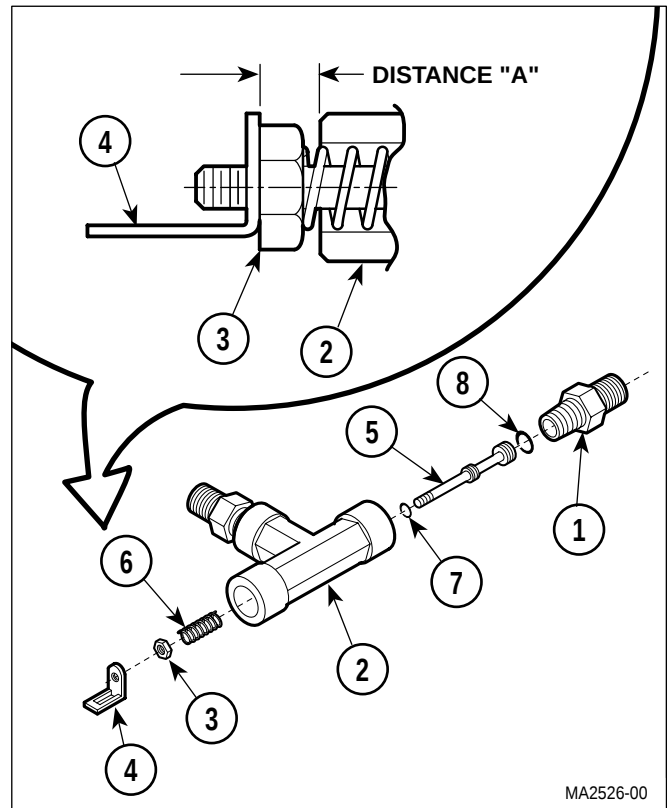

**Figure 4-19. Fill / Vent Valve Assembly Disassembly / Assembly**

- (2) While holding nut (3) stationary, unscrew closing bracket (4) from plunger (5).
- (3) Remove nut (3) and spring (6) from plunger (5).
- (4) Push plunger (5) out of valve body (2).
- (5) Remove two o-rings (7 and 8) from plunger (5).

### C. Assembly

- (1) Flush all foreign matter out of valve body (2) and off of all components with water.
- (2) Coat o-rings (7 and 8) with high temperature lubricant.
- (3) Install o-rings (7 and 8) on plunger (5).
- (4) Insert plunger (5) into valve body (2).

## SECTION IV MAINTENANCE / SERVICE

- (5) Install spring (6), nut (3), and closing bracket (4) on plunger (5).

### NOTE

The back side of the plunger has a screwdriver slot / allen wrench to assist in adjusting the closing bracket and nut.

- (6) Adjust closing bracket (4) until Distance A is approximately 3/16 to 1/4 in. (0.47 to 0.63 cm). Secure closing bracket in position by tightening nut (3) against closing bracket.
- (7) Coat threads of fitting (1) with Teflon tape / sealant.
- (8) Install fitting (1) on valve body (2).

### D. Installation

- (1) Install fill / vent valve assembly (2, Figure 4-18) in brackets (4), making sure fill / vent lever (3) is inserted in slot of closing bracket.
- (2) Position tubes (5) and screw compression nuts (1) onto fittings of fill / vent valve assembly (2).
- (3) Install cover assembly (Refer to para 4.2).
- (4) Test operation of fill / vent valve. If water will not flow or flows very slowly when fill / vent lever is depressed, lengthen Distance A slightly. See Figure 4-19.

## 4.18 Door Assembly Removal / Installation

### A. Removal




### DANGER

**Make sure unit is vented and cool before attempting to make repairs.**

**Failure to do so could result in serious injury or burns.**

- (1) Open door handle (1, Figure 4-20) to the unlatched position.
- (2) Remove torque nut (2), screw (3), door stop (4), and door assembly (5) from hinge (6).

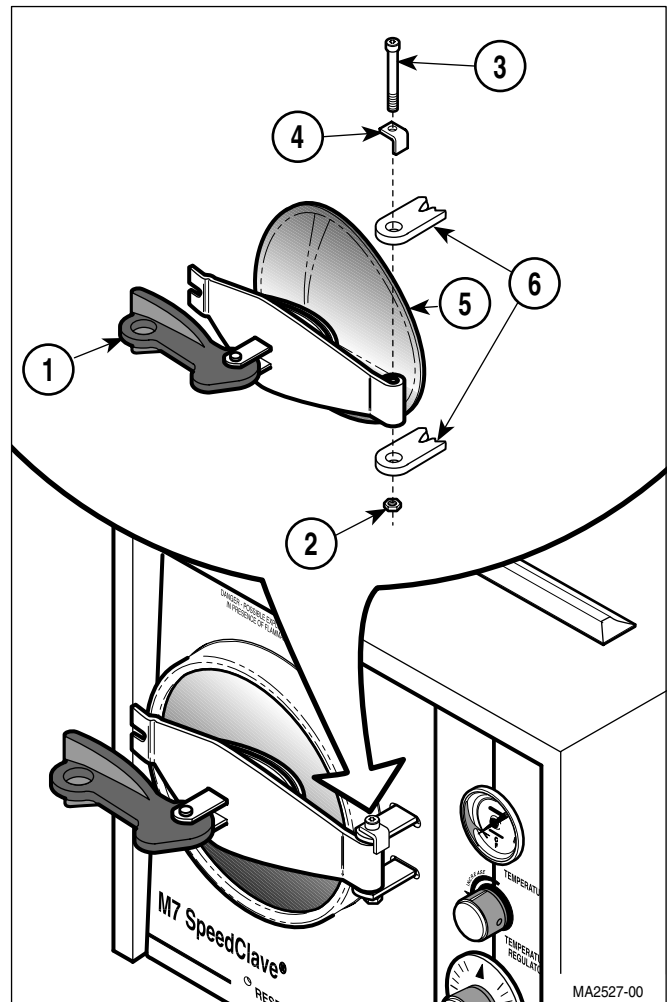


**Figure 4-20. Door Assembly Removal / Installation**

### B. Disassembly

### NOTE

Washer (2) is held on by removable threadlocking adhesive (Loctite 242) and may be difficult to remove. Pull on door to dislodge washer.

- (1) Remove cap nut (1, Figure 4-21), washer (2), and door (3) from T-bolt (4).
- (2) Remove short spring (5) and long spring (6) from T-bolt (4).
- (3) Remove retaining ring (7), pin (8), handle (9), and door cross arm (10) from U-bracket (11).
- (4) Remove T-bolt (4) from U-bracket (11).

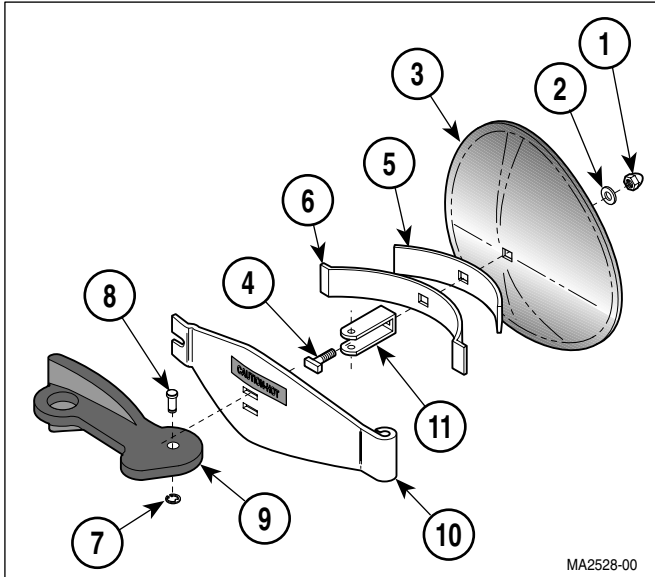

**Figure 4-21. Door Assembly Disassembly / Assembly**

**C. Assembly**

- (1) Install T-bolt (4) into U-bracket (11).
- (2) Coat bottom face of washer (2) and threads of T-bolt (4) with removable threadlocking adhesive.

**NOTE**

Door must be installed so that bowed portions of door are on top and bottom; not on the sides.

It may be necessary to place U-bracket in vise in order to tighten cap nut.

- (3) Install long spring (6), short spring (5), and door (3) on T-bolt (4) and secure with washer (2) and cap nut (1).
- (4) Install door cross arm (10) and handle (9) on U-bracket (11) and secure with pin (8) and retaining ring (7).

**D. Installation**

- (1) Coat threads of torque nut (2) with removable threadlocking adhesive (Loctite 242).

**NOTE**

Do not overtighten torque nut or door assembly may be hard to open and close.

- (2) Install door assembly (5) on hinge (6) and secure with door stop (4), screw (3), and torque nut (2).

**4.19 Door Gasket Removal / Installation**


**A. Removal**



**DANGER**

**Make sure unit is vented and cool before attempting to make repairs. Failure to do so could result in serious injury or burns.**

- (1) Open the door assembly.
- (2) Pry door gasket (1, Figure 4-22) out of gasket channel of chamber (2).

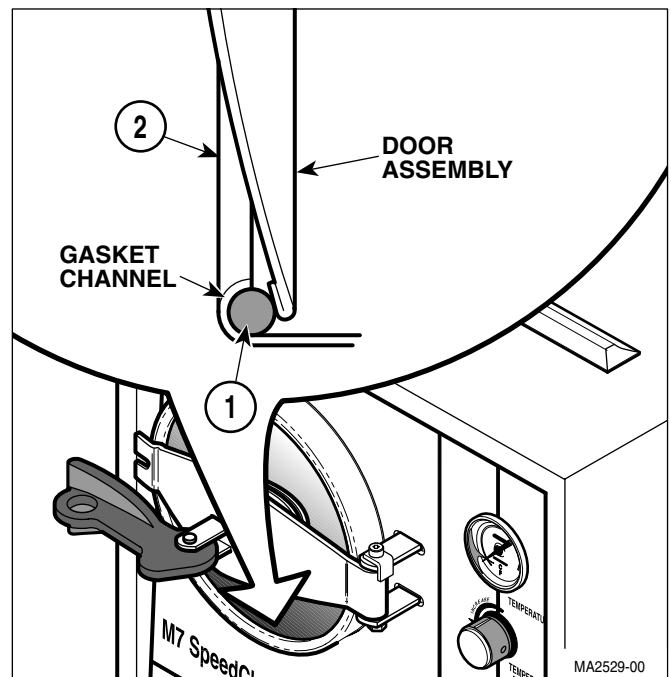

**Figure 4-22. Door Gasket Removal / Installation**

## SECTION IV MAINTENANCE / SERVICE

### B. Installation

- (1) Using a hard bristle brush, clean all foreign matter from gasket channel of chamber (2).
- (2) Lubricate door gasket (1) and gasket channel of chamber with soapy water or liquid detergent.
- (3) Using fingers, push door gasket (1) into gasket channel of chamber (2).
- (4) Run a cycle to seat door gasket. Refer to Operator Manual if necessary.

### 4.20 Wire Rack And Trays Removal / Installation

#### A. Removal



#### **DANGER**

**Make sure unit is vented and cool before attempting to make repairs. Failure to do so could result in serious injury or burns.**

- (1) Remove three trays (1, Figure 4-23) from wire rack (2).
- (2) Lift up on left edge of rack base plate (3) until rack base plate "pops" free of wire rack (2).
- (3) While holding rack base plate (3) in a vertical position, squeeze bottom of wire rack (2) together and remove it from chamber.


#### B. Installation

- (1) While holding rack base plate (3) in a vertical position, squeeze the bottom of the wire rack (2) together and insert it into chamber.

#### **NOTE**

There are two slots on the left edge of the rack base plate. These slots must be aligned with wire rack before the rack base plate can be pushed down into locked position.

- (2) Press down on left edge of rack base plate (3) until rack base plate "pops" into locked position.

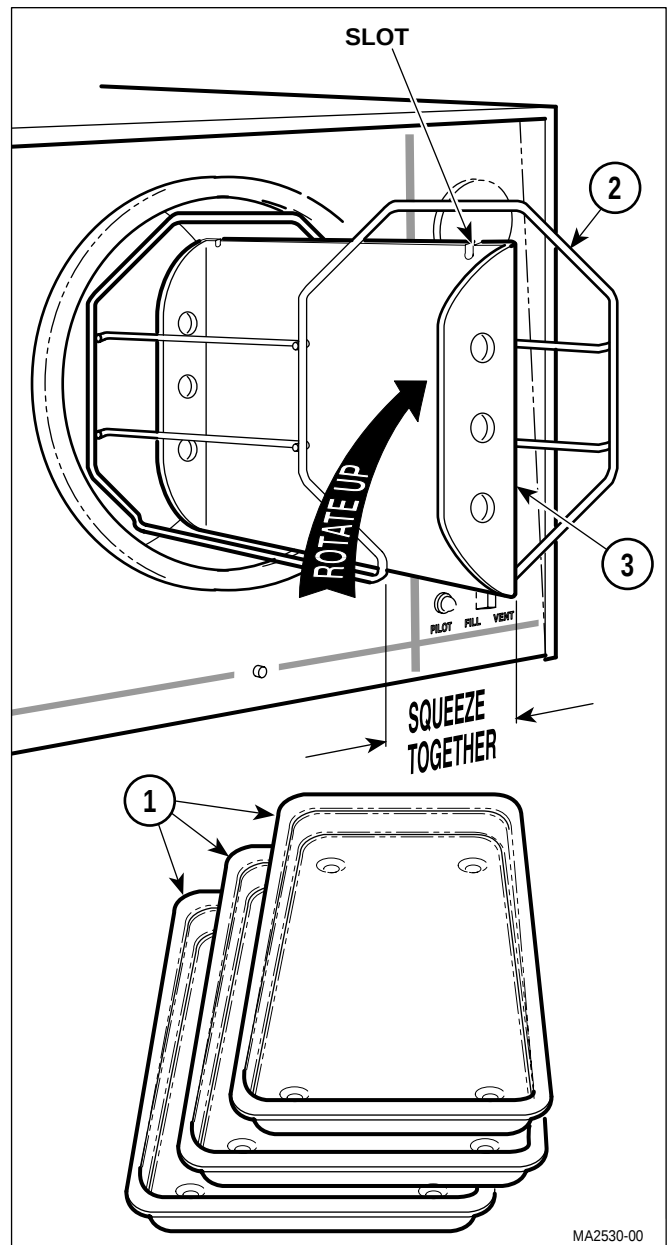

**Figure 4-23. Wire Rack And Trays Removal / Installation**

### 4.21 Condensing Tank Removal / Installation

#### **NOTE**

This procedure is based on removing a tank assembly with a new style bellows setup. Removing a tank assembly with an old style bellows setup is similar - the bellows is removed from inside of the tank assembly instead of from the outside on an old style bellows setup.

## A. Removal

- (1) Remove cover assembly (Refer to para 4.2).




### **DANGER**

**Make sure unit is vented and cool before attempting to make repairs.**

**Failure to do so could result in serious injury or burns.**

- (2) Remove drain (1, Figure 4-24) and drain water from tank assembly (2).
- (3) Remove temperature gauge (Refer to para 4.12).

- (4) Loosen and disconnect compression nut (3) from elbow (4).
- (5) Loosen and disconnect compression nut (5) from elbow (6).
- (6) Loosen and disconnect compression nut (7) and remove bellows assembly (8) and grommet (9) from tank assembly (2).
- (7) Remove pipe-T assembly (10) from tee of tank assembly (2).
- (8) Lay unit onto its side.

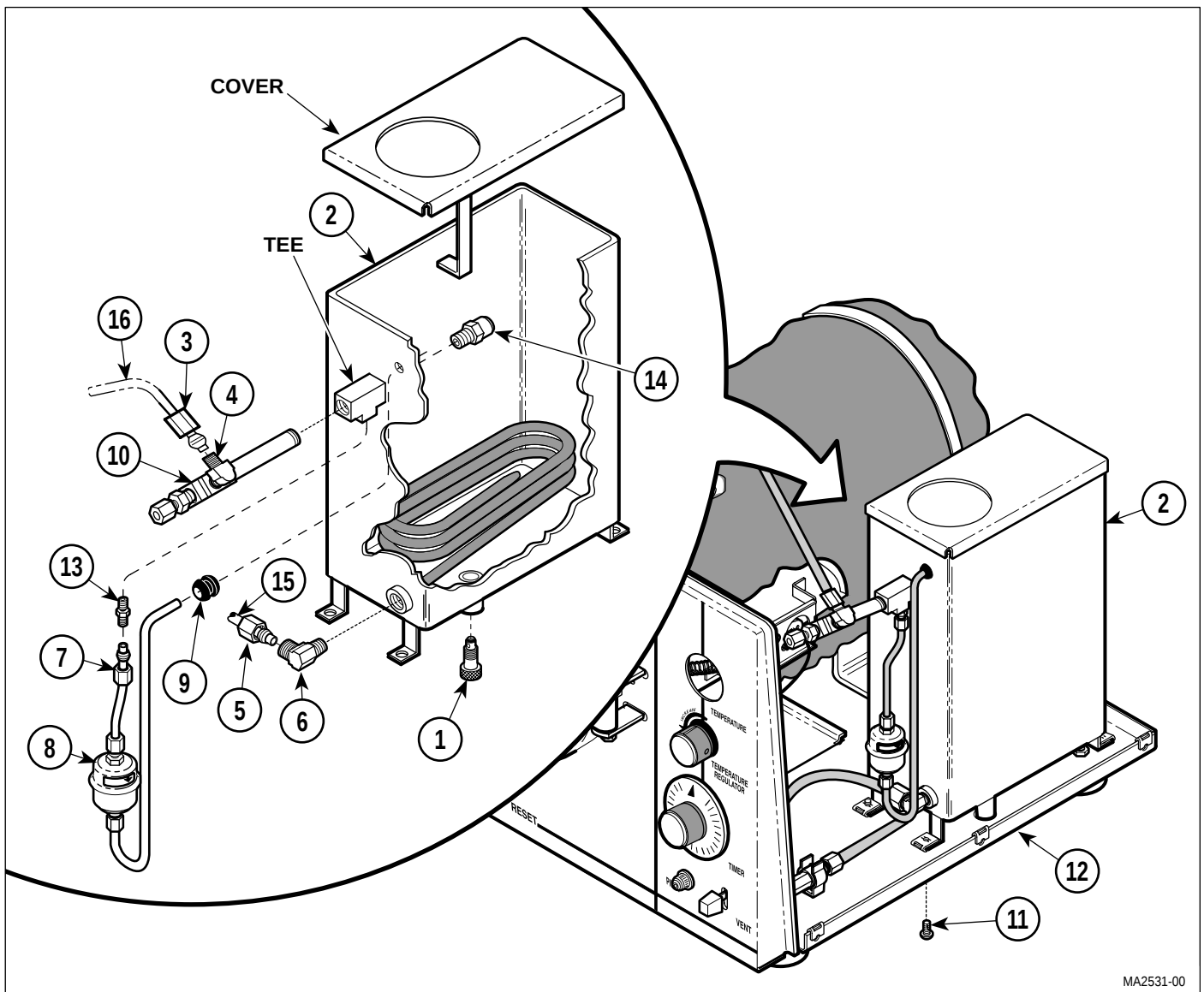

**Figure 4-24. Condensing Tank Removal / Installation**

## SECTION IV MAINTENANCE / SERVICE

- (9) Remove four screws (11) and tank assembly (2) from base (12).
- (10) Remove male connector (13) from tee of tank assembly (2).
- (11) Remove elbow (6) from tank assembly (2).
- (12) Remove pressure relief valve (14) from tee of tank assembly (2).


### B. Installation

- (1) Coat threads of pressure relief valve (14), elbow (6), and male connector (13) with Teflon tape / sealant.
- (2) Install pressure relief valve (14) on tee of tank assembly (2).
- (3) Install elbow (6) on tank assembly (2).
- (4) Install male connector (13) on tee of tank assembly (2).
- (5) Install tank assembly (2) on base (12) and secure with four screws (11).
- (6) Turn unit to upright position.
- (7) Install pipe-T assembly (10) on tee of tank assembly (2).
- (8) Install grommet (9) in tank assembly (2).
- (9) Slide tube of bellows assembly (8) thru grommet. Connect other end of bellows assembly to male connector (13) with compression nut (7).
- (10) Connect tube (15) to elbow (6) with compression nut (5).
- (11) Connect tube (16) to elbow (4) with compression nut (3).
- (12) Install temperature gauge (Refer to para 4.12).
- (13) Install drain (1) in tank assembly (2).
- (14) Install cover assembly (Refer to para 4.2).
- (15) Fill tank assembly with distilled or demineralized water.

## 4.22 Chamber Assembly Removal / Installation


### A. Removal

- (1) Remove cover assembly (Refer to para 4.2).
- (2) Remove door assembly (Refer to para 4.18).
- (3) Remove door gasket (Refer to para 4.19).
- (4) Open drain (1, Figure 4-25) and drain water from condensing tank.
- (5) Remove wire rack and trays (Refer to para 4.20).
- (6) Remove temperature regulator relay (Refer to para 4.9).
- (7) Remove diaphragm cup (Refer to para 4.11).
- (8) Remove heating element (Refer to para 4.7).
- (9) Remove overheat thermostat(s) (Refer to para 4.8).
- (10) Loosen and disconnect compression nut (2) from male fitting (3).
- (11) Loosen and disconnect compression nut (4) from elbow fitting (5).
- (12) Remove two screws (6) and chamber assembly (7) from base (8).
- (13) Disconnect and remove one banding strap (1, Figure 4-26) and two banding straps (2) from insulation (3).
- (14) Remove insulation (3) from chamber (4).
- (15) Remove nut (5), screw (6), and chamber leg bracket (7) from chamber (4).
- (16) Remove elbow fitting (8) from chamber (4).
- (17) Remove male fitting (9) from chamber (4).

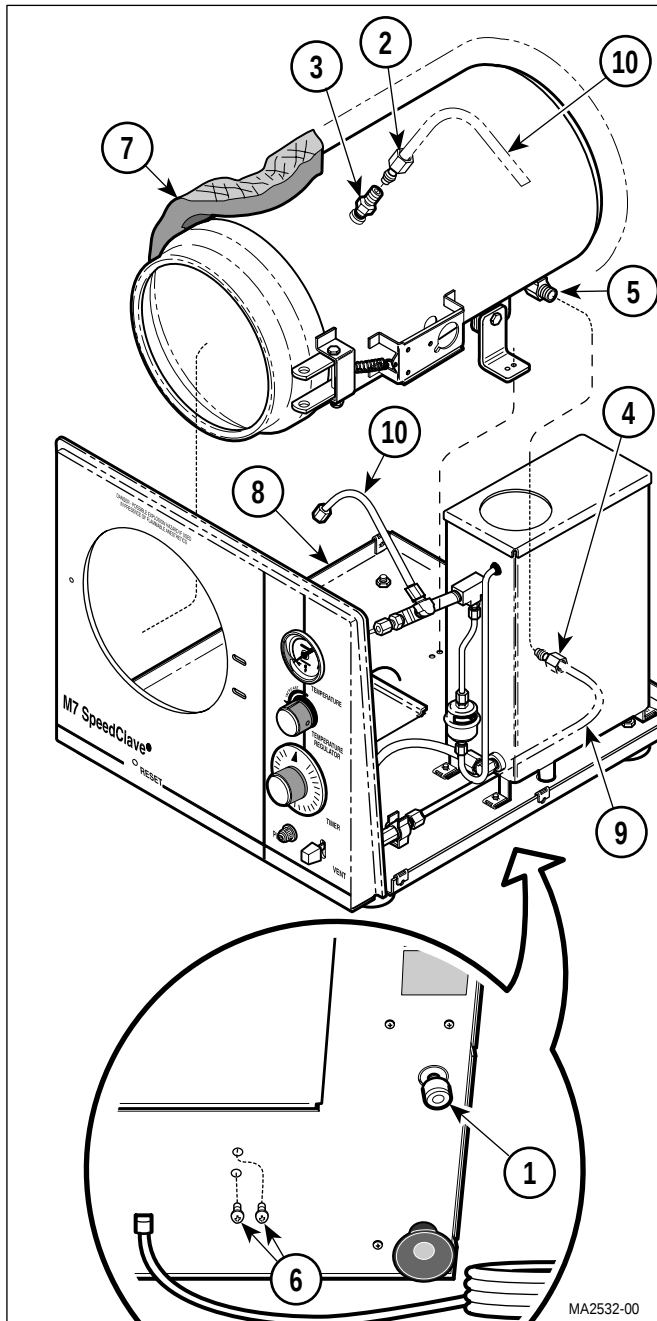

**Figure 4-25. Chamber Assembly  
Removal / Installation**

- (18) Disconnect spring (10) from spring holes.
- (19) Remove nut (11), screw (12), hinge (13), and washer (14) from door bracket of chamber (4).

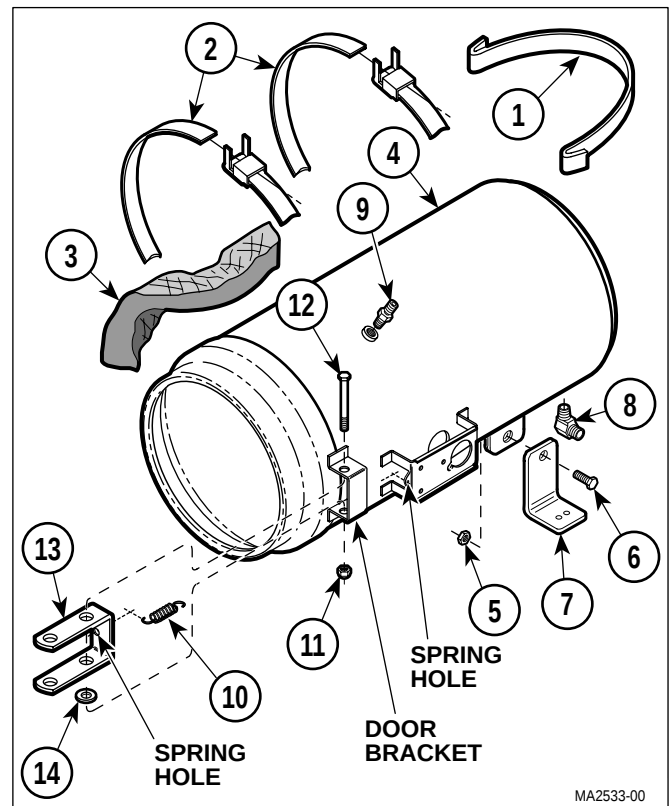

**Figure 4-26. Chamber Assembly  
Disassembly / Assembly**

**B. Installation**

- (1) Coat threads of nut (11, Figure 4-26) with removable threadlocking adhesive (Loctite 242).
- (2) Install washer (14), and hinge (13) on door bracket of chamber (4) and secure with screw (12) and nut (11).
- (3) Insert ends of spring (10) in spring holes.
- (4) Coat threads of fittings (8 and 9) with Teflon tape / sealant.
- (5) Install male fitting (9) and elbow fitting (8) on chamber (4).
- (6) Install chamber leg bracket (7) on chamber (4) and secure with screw (6) and nut (5).
- (7) Install insulation (3) on chamber (4). Secure insulation on chamber with two banding straps (2) and banding strap (1).

## SECTION IV MAINTENANCE / SERVICE

- (8) Install chamber assembly (7, Figure 4-25) on base (8) and secure with two screws (6).

### NOTE

If compression sleeves are too deformed to be reused, remove old compression sleeves and replace with new ones.

- (9) Connect tube (9) to elbow fitting (5) and secure with compression nut (4).
- (10) Connect tube (10) to male fitting (3) and secure with compression nut (2).
- (11) Install overheat thermostat (Refer to para 4.8).


- (12) Install heating element (Refer to para 4.7).
- (13) Install diaphragm cup (Refer to para 4.11).
- (14) Install wire rack and trays (Refer to para 4.20).
- (15) Install door assembly (Refer to para 4.18).
- (16) Install door gasket (Refer to para 4.19).
- (17) Close drain (1) and fill condensing tank with distilled or demineralized water.
- (18) Install temperature regulator relay (Refer to para 4.9).

**SECTION V**  
**SCHEMATICS AND DIAGRAMS**

**5.1 Electrical Schematics / Wiring Diagrams**

Figure 5-1, sheets 1 thru 3 illustrate the current flow


between electrical components in the M7. Figure 5-2, sheets 1 thru 3 illustrate the wiring connections for the sterilizer.

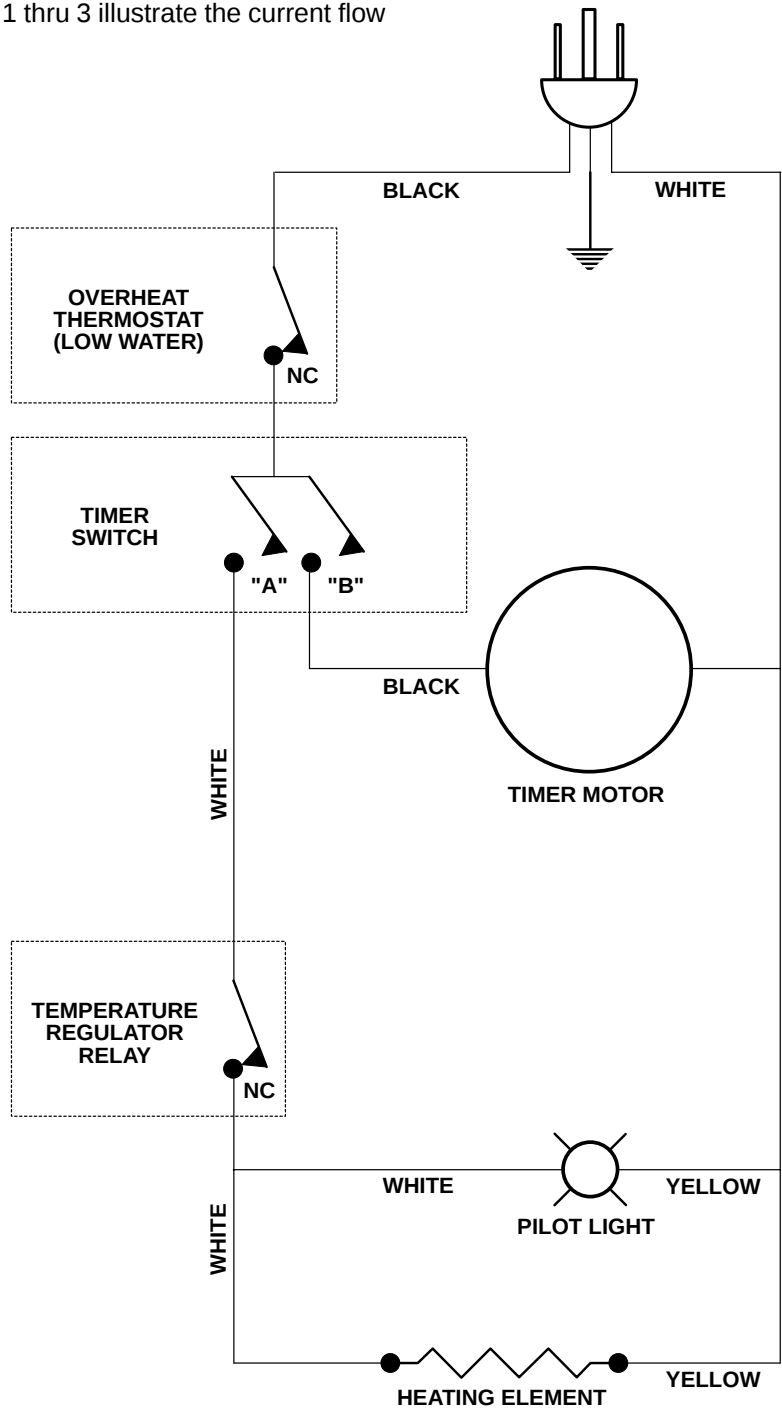

MA2562-01

**Figure 5-1 (Sheet 1 of 3). Electrical Schematic - Units With Serial Numbers CP-1000 Thru CP1069, CS-1000 Thru CS1694, CR-1000 Thru CR-1125, RB-1000 Thru RB-7499, And RB-8000 Thru RB-8106**


**SECTION V**  
**SCHEMATICS AND DIAGRAMS**

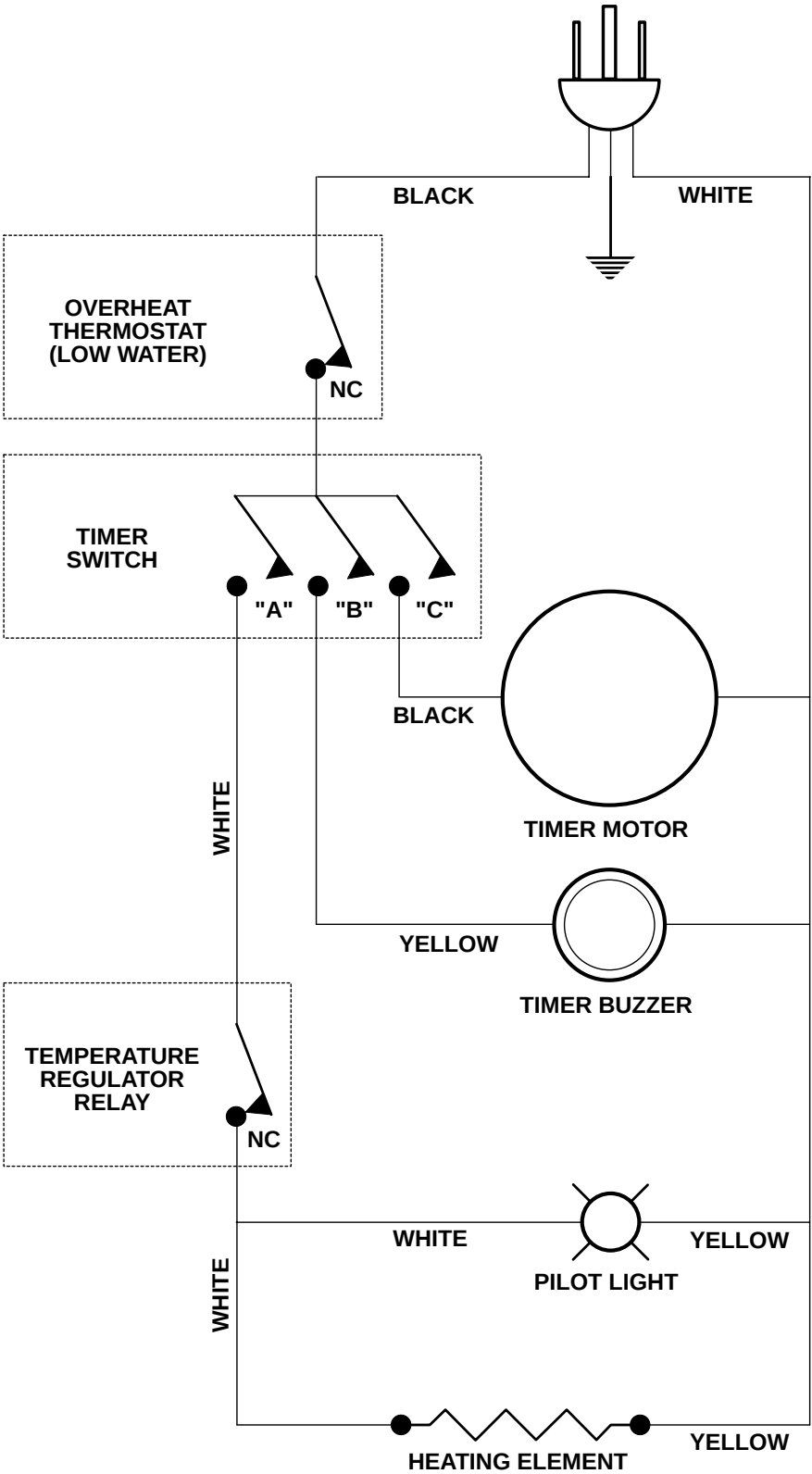

MA2562-00

**Figure 5-1 (Sheet 2 of 3). Electrical Schematic - Units With Serial Numbers CP-1070 Thru Present, CS-1695 Thru CS-5296, CR-1126 Thru Present, DR-1015 Thru Present, RB-7500 Thru RB-7999, And RB-8107 Thru RB-14564**

**SECTION V**  
**SCHEMATICS AND DIAGRAMS**

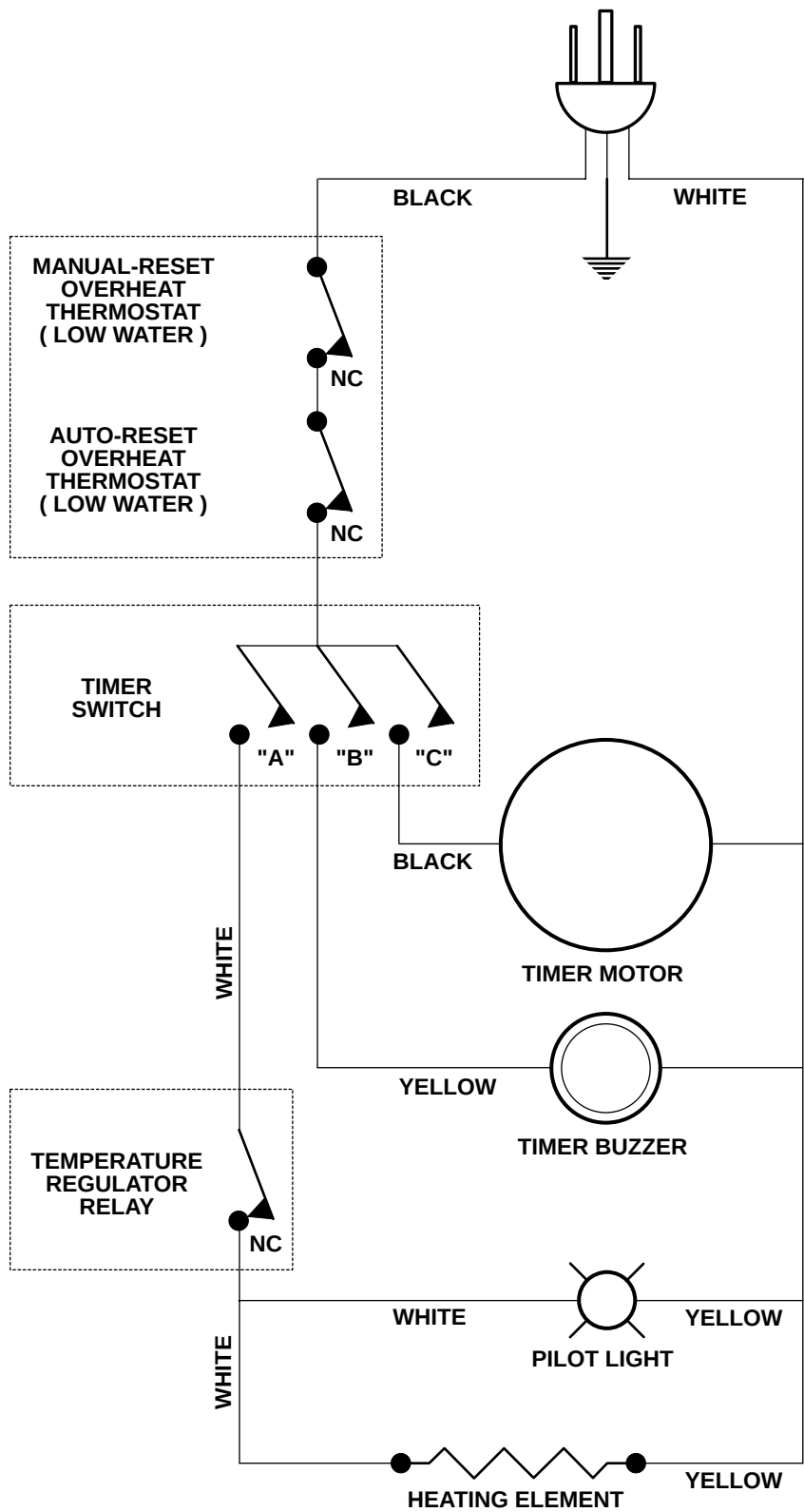

MA2562-02


**Figure 5-1 (Sheet 3 of 3). Electrical Schematic - Units With Serial Numbers  
CS-5297 Thru Present and RB-14565 Thru Present**

## SCHEMATICS AND DIAGRAMS



**Figure 5-2 (Sheet 1 of 3). Wiring Diagram - Units With Serial Numbers CP-1000 Thru CP1069, CS-1000 Thru CS1694, CR-1000 Thru CR-1125, RB-1000 Thru RB-7499, And RB-8000 Thru RB-8106**


## SECTION V SCHEMATICS AND DIAGRAMS

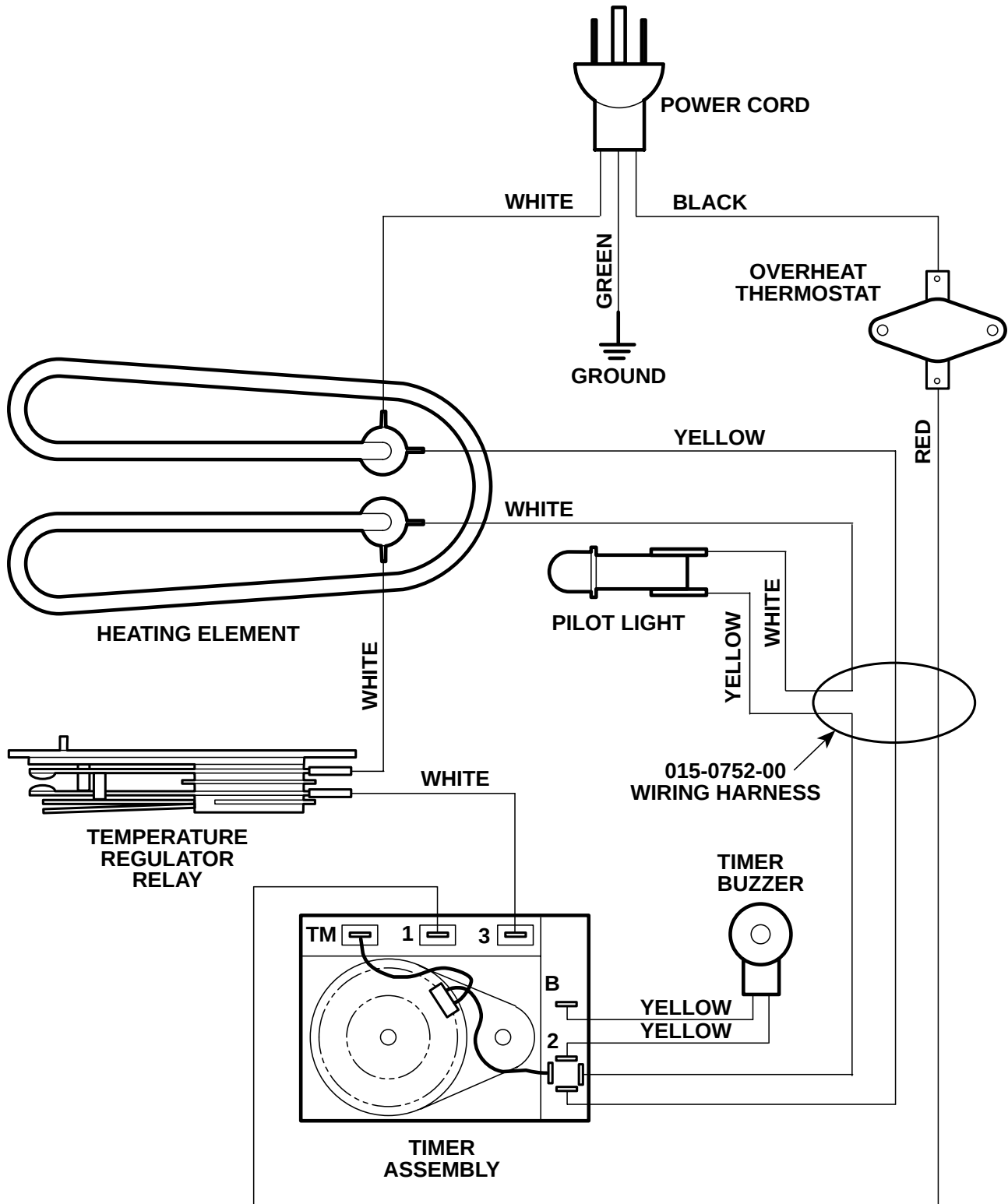

MA253601

Figure 5-2 (Sheet 2 of 3). Wiring Diagram - Units With Serial Numbers CP-1070 Thru Present, CS-1695 Thru CS-5296, CR-1126 Thru Present, DR-1015 Thru Present, RB-7500 Thru RB-7999, And RB-8107 Thru RB-14564

## SCHEMATICS AND DIAGRAMS

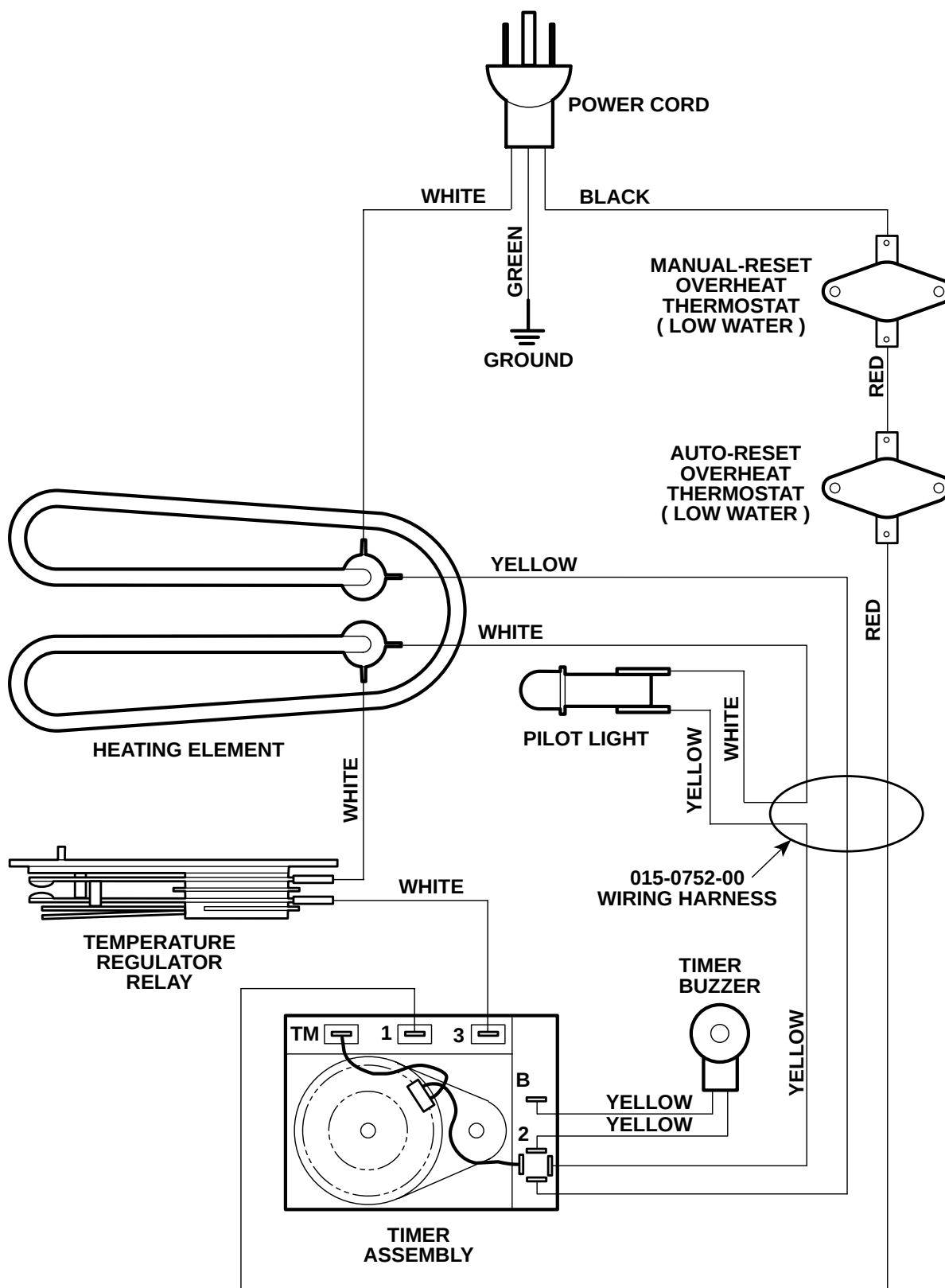

**Figure 5-2 (Sheet 3 of 3). Wiring Diagram - Units With Serial Numbers  
CS-5297 Thru Present And RB-14565 Thru Present**

MA253602

5.2 Flow Diagrams

Figure 5-3, sheets 1 thru 4, illustrate the water, heated water, steam, and vented steam/water flow throughout the sterilizer during a cycle.


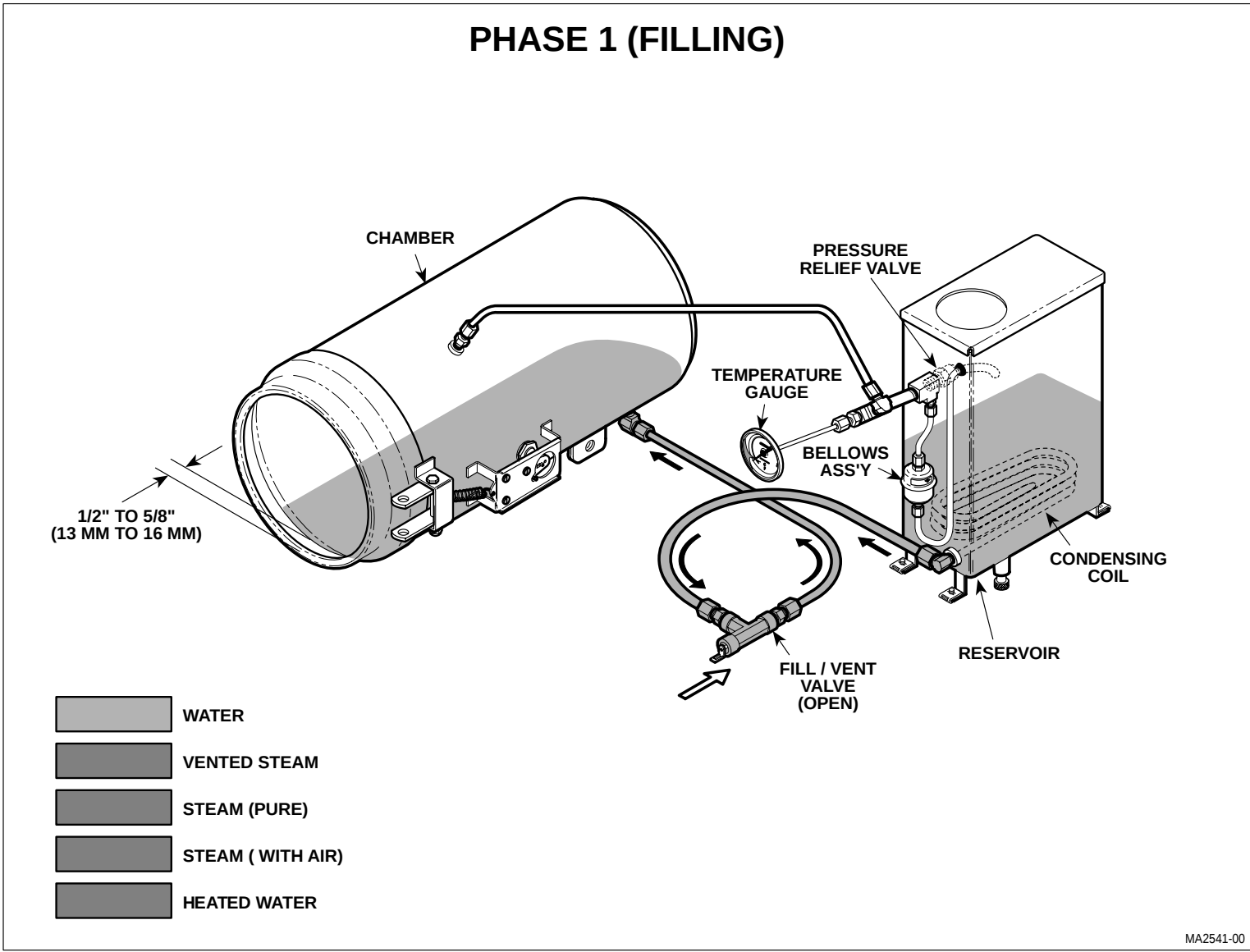


Figure 5-3 (Sheet 1 of 4). Flow Diagram


## PHASE 2 (HEAT-UP)

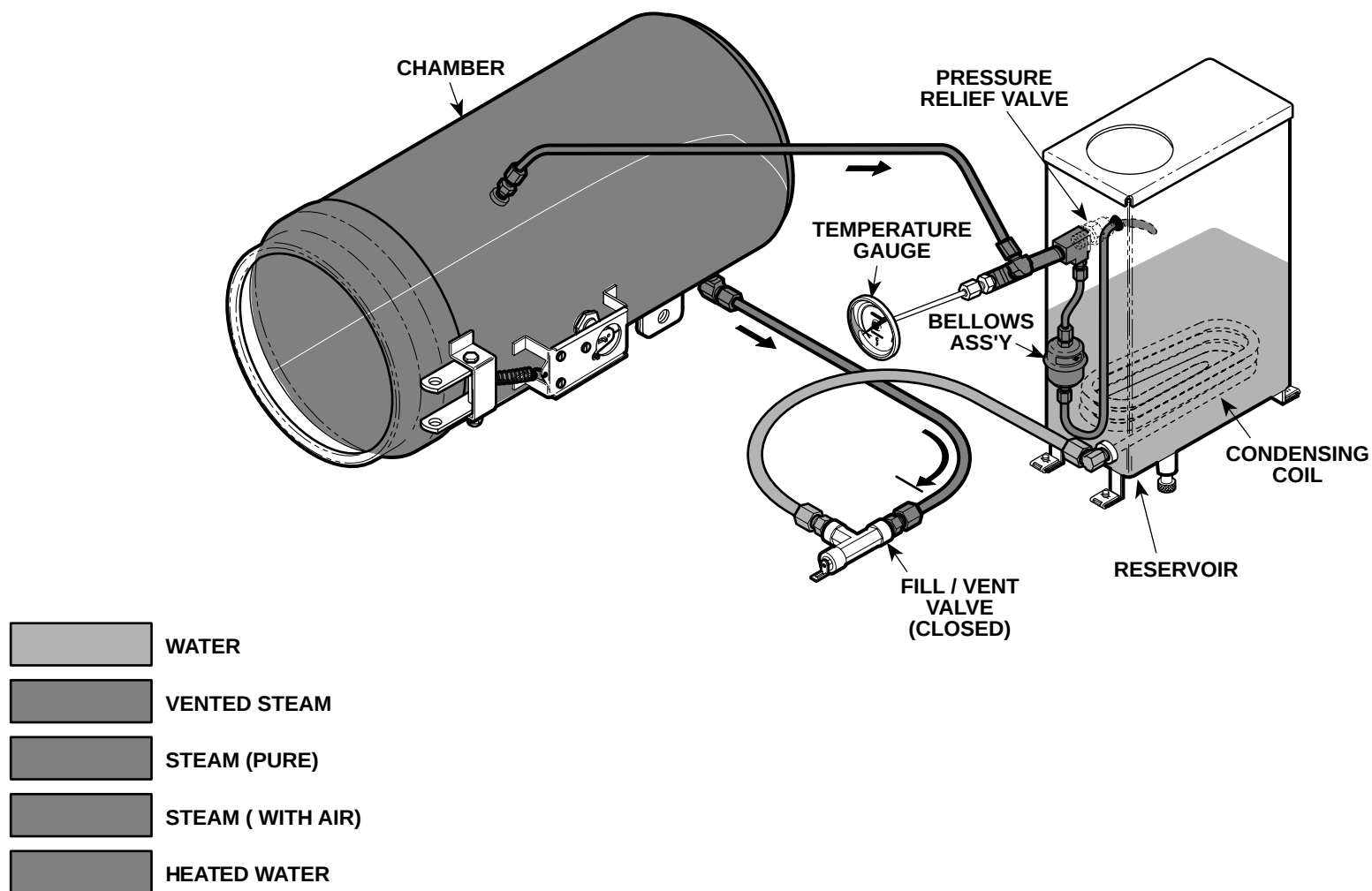

MA2542-00

Figure 5-3 (Sheet 2 of 4). Flow Diagram


# PHASE 3 (STERILIZING)

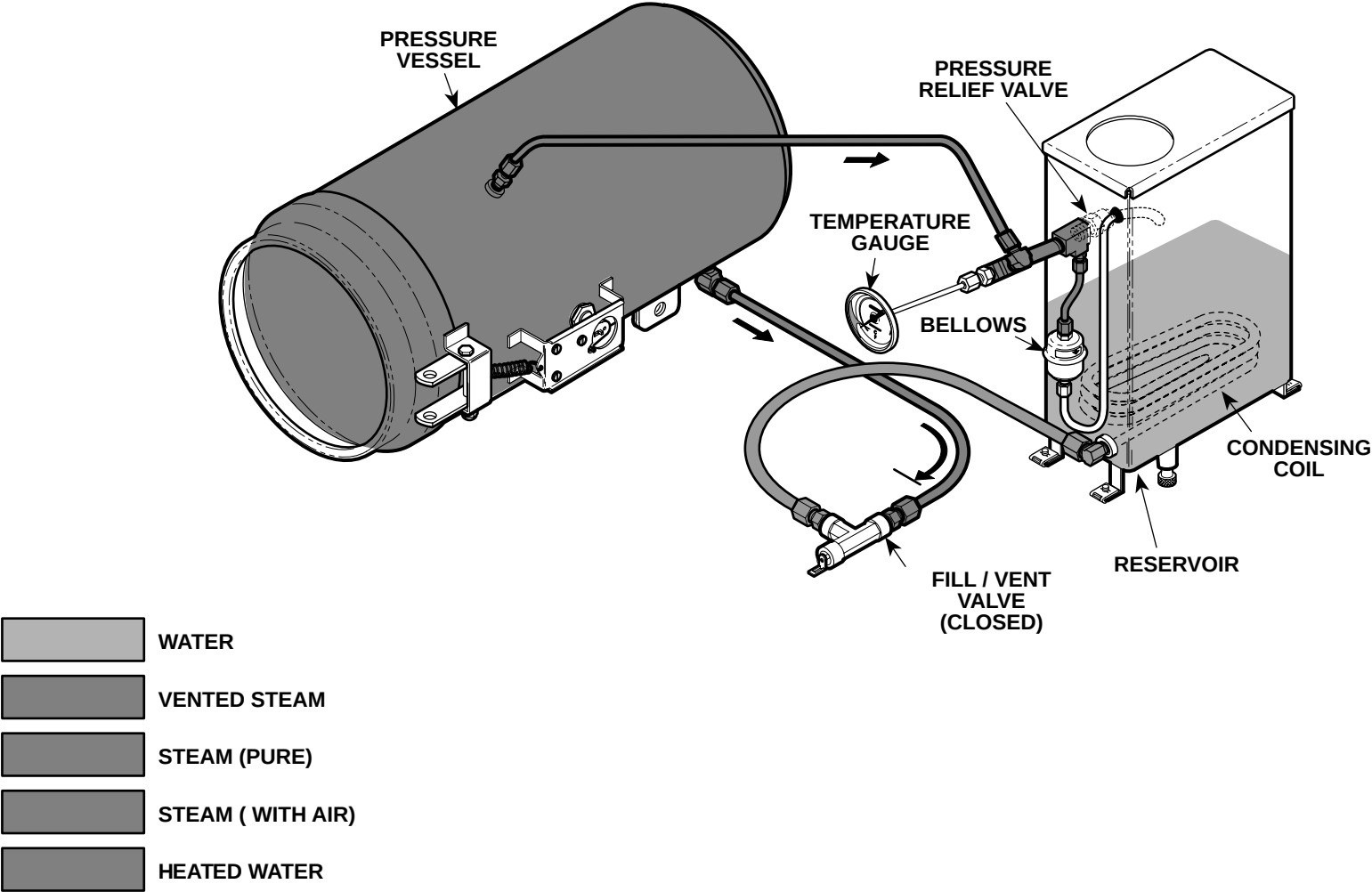

MA2543-00

Figure 5-3 (Sheet 3 of 4). Flow Diagram

## PHASE 4 (VENT)

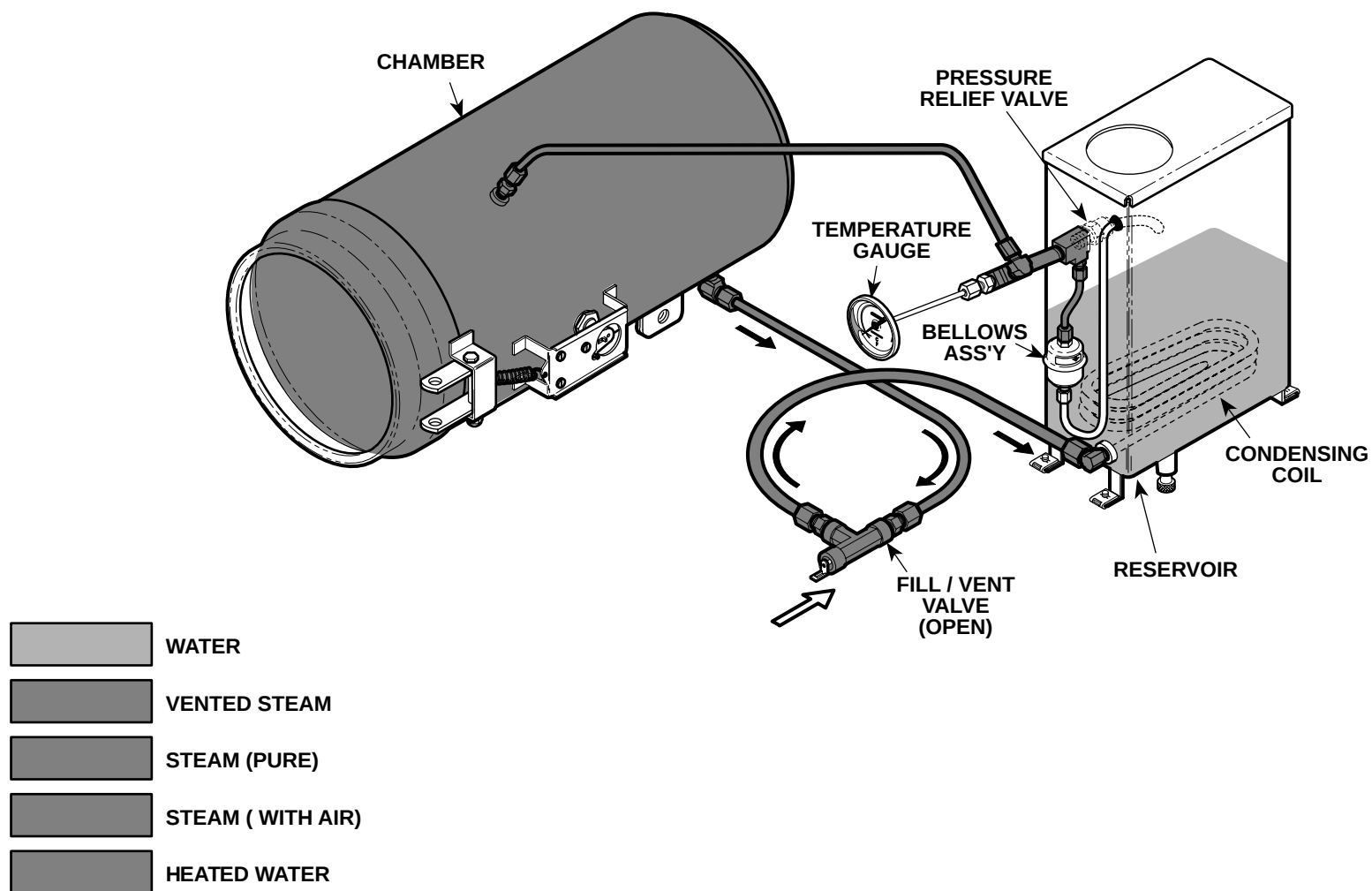

MA2544-00

Figure 5-3 (Sheet 4 of 4). Flow Diagram

### 5.3 Suggested Times And Temperatures Charts

Table 5-1 lists the suggested times and temperatures to use when sterilizing a certain type of load in the M7.

Table 5-2 lists the suggested extended time at reduced temperature for higher altitudes. Table 5-3 is a chart which lists what pressure should be present at particular temperature.

**Table 5-1. Suggested Times And Temperatures**

| Load                             | Temperature |      | Time       |
|----------------------------------|-------------|------|------------|
|                                  | °F          | °C   |            |
| Unwrapped Instruments*           | 270°        | 132° | 3 minutes  |
| Wrapped Instruments              | 270°        | 132° | 5 minutes  |
| Cotton Dressings & Bandages      | 250°        | 121° | 20 minutes |
| Syringes - (disassembled)        | 250°        | 121° | 20 minutes |
| Rubber Goods and Gloves          | 250°        | 121° | 15 minutes |
| * Instruments for immediate use. |             |      |            |

**Table 5-2. Suggested Extended Time At Reduced Temperature For Higher Altitudes**

| Altitude                               | Recommended Temperature Setting |      | Operating Pressure |     | Minimum Exposure Time |
|----------------------------------------|---------------------------------|------|--------------------|-----|-----------------------|
| Feet (Meters) Above Sea Level          | °F                              | °C   | PSI                | kPa | Full Load             |
| 0 ft. (0 m) to 984 ft. (300 m)         | 270°                            | 132° | 28                 | 190 | 10 minutes            |
| 984 ft. (300 m) to 3280 ft. (1000 m)   | 266°                            | 130° | 28                 | 190 | 13 minutes            |
| 3280 ft. (1000 m) to 6560 ft. (2000 m) | 250°                            | 121° | 28                 | 190 | 16 minutes            |
| 6560 ft. (2000 m) to 9840 ft. (3000 m) | 250°                            | 121° | 28                 | 190 | 19 minutes            |

## SECTION V SCHEMATICS AND DIAGRAMS

Table 5-3. Pressure / Temperature Chart

| PSI | °C    | °F    | PSI | °C    | °F    |
|-----|-------|-------|-----|-------|-------|
| 0   | 100.0 | 212.0 | 16  | 122.0 | 251.6 |
| 1   | 101.9 | 215.4 | 17  | 123.0 | 253.4 |
| 2   | 103.6 | 218.5 | 18  | 124.1 | 255.4 |
| 3   | 105.3 | 221.5 | 19  | 125.0 | 257.0 |
| 4   | 106.9 | 224.4 | 20  | 126.0 | 258.8 |
| 5   | 108.4 | 227.1 | 21  | 126.9 | 260.0 |
| 6   | 109.8 | 229.6 | 22  | 127.8 | 262.0 |
| 7   | 111.3 | 232.3 | 23  | 128.7 | 263.7 |
| 8   | 112.6 | 234.7 | 24  | 129.6 | 265.3 |
| 9   | 113.9 | 237.0 | 25  | 130.4 | 266.7 |
| 10  | 115.2 | 239.4 | 26  | 131.3 | 268.3 |
| 11  | 116.4 | 241.5 | 27  | 132.1 | 269.8 |
| 12  | 117.6 | 243.7 | 28  | 132.9 | 271.2 |
| 13  | 118.8 | 245.8 | 29  | 133.7 | 272.7 |
| 14  | 119.9 | 247.8 | 30  | 134.5 | 274.1 |
| 15  | 121.0 | 249.8 |     |       |       |

Normal steam sterilizing range is 250 °F to 270 °F.

Figures in chart are for steam pressure only. The presence of any air in the autoclave invalidates temperature readings.

## **SECTION VI**

### **PARTS LIST**

#### **6.1 Introduction**

The illustrated parts list provides information for identifying and ordering the parts necessary to maintain the unit in peak operating condition. Refer to paragraph 1.5 for parts ordering information.

The parts list also illustrates disassembly and assembly relationships of parts.

#### **6.2 Description of Columns**

The *Item* column of the parts list gives a component its own unique number. The same number is given to the component in the parts illustration. This allows a part number of a component to be found if the technician can visually spot the part on the illustration. The technician simply finds the component in question on the illustration and notes the item number of that component. Then, he finds that item number in the parts list. The row corresponding to the item number gives the technician the part number, a description of the component, and quantity of parts per subassembly. Also, if a part number is known, the location of that component can be determined by looking for the item number of the component on the illustration.

The *Part No.* column lists the MIDMARK part number for that component.

The *Description* column provides a physical description of the component.

The *Qty.* column lists the number of units of a particular component that is required for the subassembly. The letters "AR" denote "as required" when quantities of a particular component cannot be determined, such as: adhesive.


Bullets { • } in the *Part No.* column and the *Description* column show the indenture level of a component. If a component does not have a bullet, it is a main component of that illustration. If a component has a bullet, it is a subcomponent of the next component listed higher in the parts list than itself that does not have a bullet. Likewise, if a component has two bullets it is a subcomponent of the next component listed higher in the parts list than itself that has only one bullet.

#### **6.3 Torque Specifications and Important Assembly Notes**

When specific assembly torque specifications, measurements, or procedures have been identified, by our engineering department, as required to assure proper function of the unit, those torque specifications measurements, and procedures will be noted on the parts illustrations. Adherence to these requirements is essential.

# M7 Sterilizer

## SECTION VI PARTS LIST

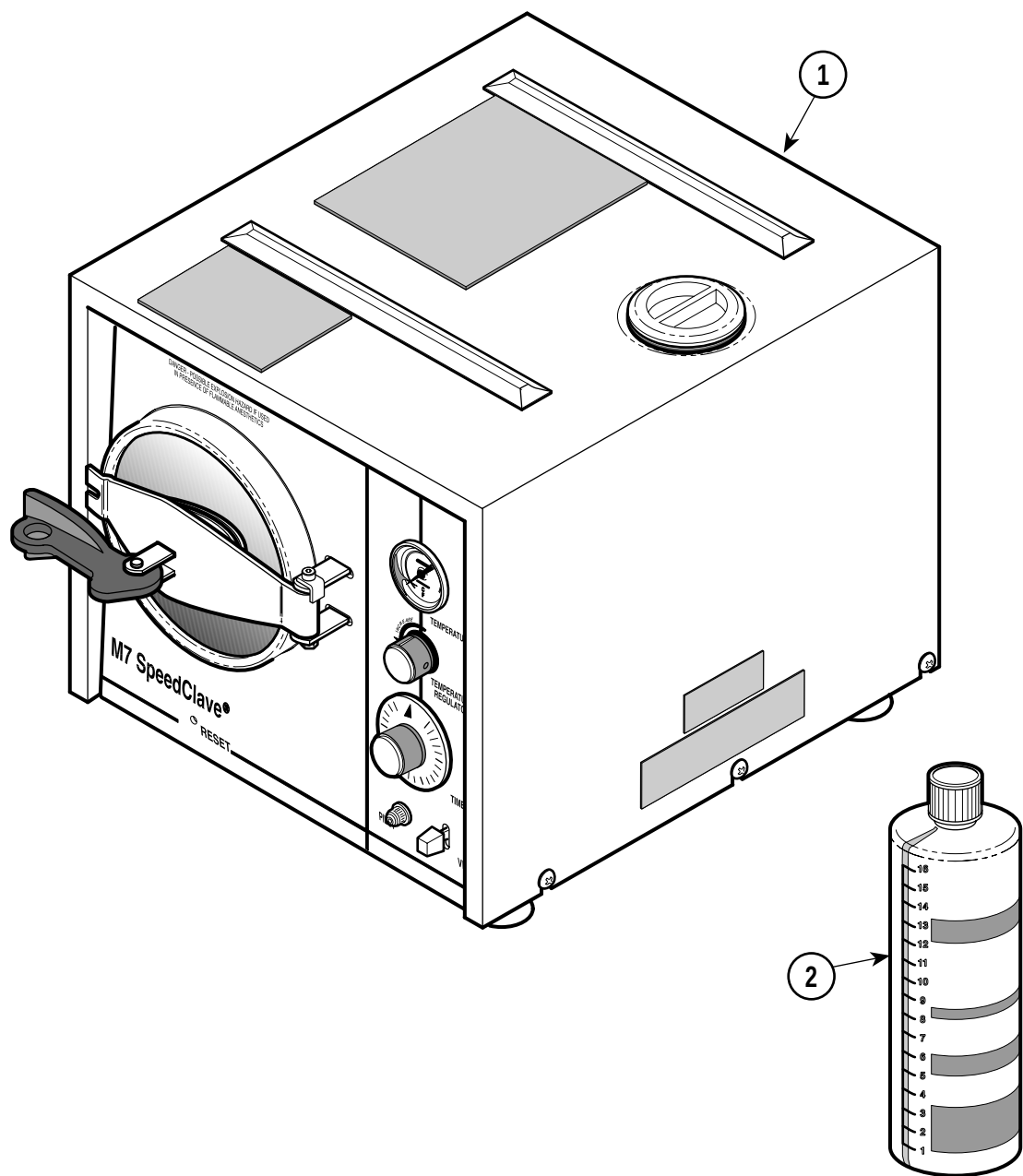


MA242600

| Item | Part No. | Description                                          | Qty. | Item | Part No.    | Description                                  | Qty. |
|------|----------|------------------------------------------------------|------|------|-------------|----------------------------------------------|------|
| 1    | M7-001   | Ritter 115/60/1<br>(S/N Prefix "RB*****") .....      | Ref  |      | M7-005      | M7 (Special)<br>(S/N Prefix "FM*****") ..... | Ref  |
|      | M7-002   | 220-240/50/1<br>(S/N Prefix "CR*****" & "DR*****") . | Ref  |      | M7-006      | M7 (Special)<br>(S/N Prefix "FN*****") ..... | Ref  |
|      | M7-003   | 100/60/1<br>(S/N Prefix "CP*****") .....             | Ref  | 2    | 002-0396-00 | Sterilizer Cleaner .....                     | 1    |
|      | M7-004   | Midmark<br>(S/N Prefix "CS*****") .....              | Ref  |      |             |                                              |      |

Always Specify Model & Serial Number

# Sterilizer Cover

## SECTION VI PARTS LIST

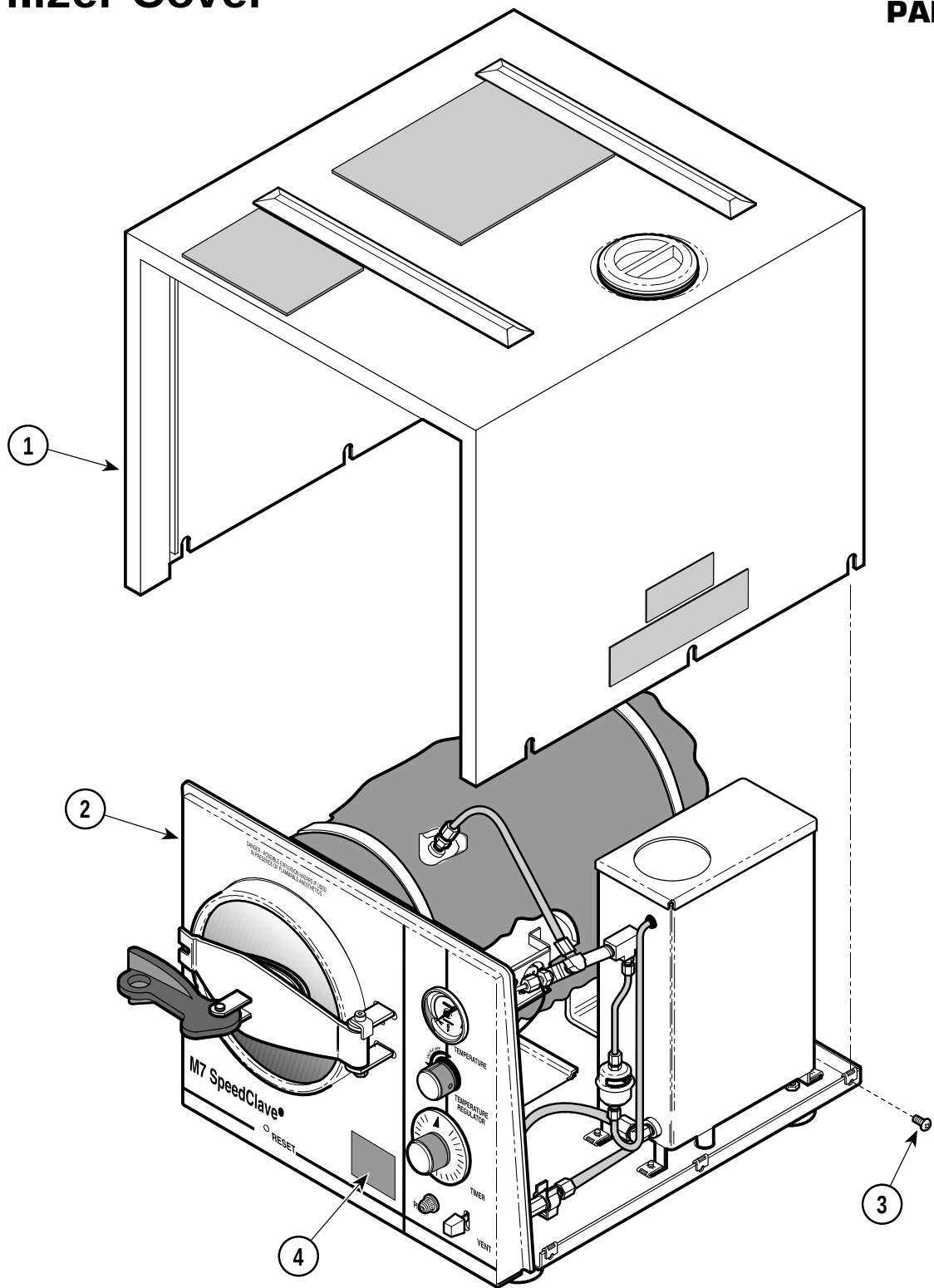


MA242800

| Item | Part No. | Description                                                                   | Qty. | Item | Part No.     | Description                             | Qty. |
|------|----------|-------------------------------------------------------------------------------|------|------|--------------|-----------------------------------------|------|
| 1    |          | Cover Assembly (Refer to "Sterilizer Cover Components" Elsewhere) .....       | 1    | 3    | 040-0010-125 | Screw .....                             | 6    |
| 2    |          | Sterilizer Components (Refer to "Sterilizer Main Components" Elsewhere) ..... | 1    | 4    | H353064      | Decal (M7-005 [Early Units Only]) ..... | 1    |

Always Specify Model & Serial Number

# Sterilizer Cover Components

## SECTION VI PARTS LIST

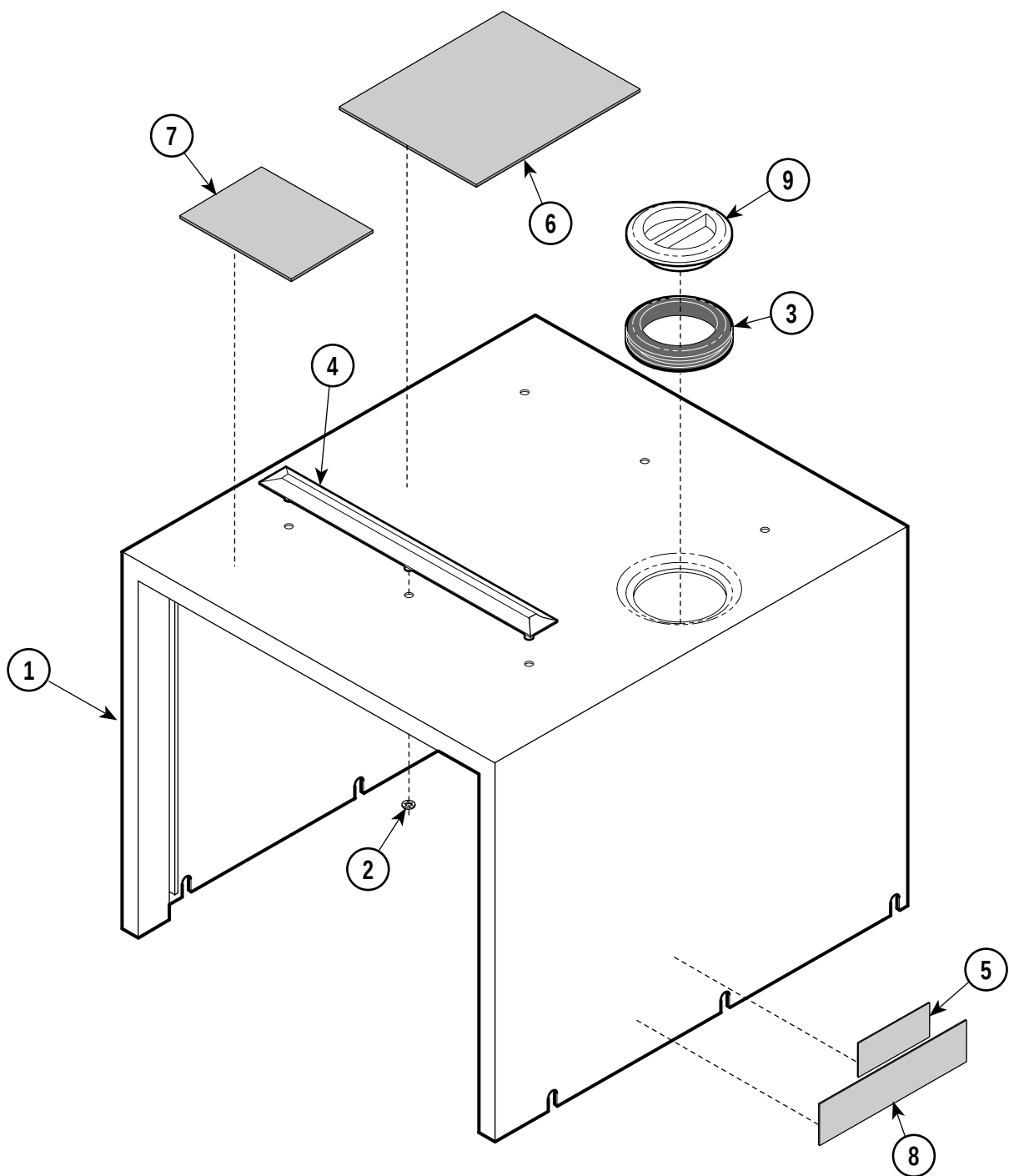


MA2413-00

| Item | Part No.    | Description                             | Qty. | Item | Part No.      | Description                    | Qty. |
|------|-------------|-----------------------------------------|------|------|---------------|--------------------------------|------|
|      | 002-0489-00 | M7 Cover Kit (Includes Items 1 thru 8)  | 1    | 5    | • H335669     | • Warning Label                | 1    |
| 1    | • H335994   | • Cover Assembly (Incl. Items 2 thru 4) | 1    | 6    | • 061-0443-00 | • Operation Instructions Plate | 1    |
| 2    | •• H98011   | •• Push Nut                             | 6    | 7    | • H284693     | • Danger Plate                 | 1    |
| 3    | •• H98135   | •• Rubber Seal                          | 1    | 8    | • 061-0203-00 | • Caution Label                | 1    |
| 4    | •• H284657  | •• Top Rail                             | 2    | 9    | H98009        | Filler Cap                     | 1    |

Always Specify Model & Serial Number

# Sterilizer Main Components

## SECTION VI PARTS LIST

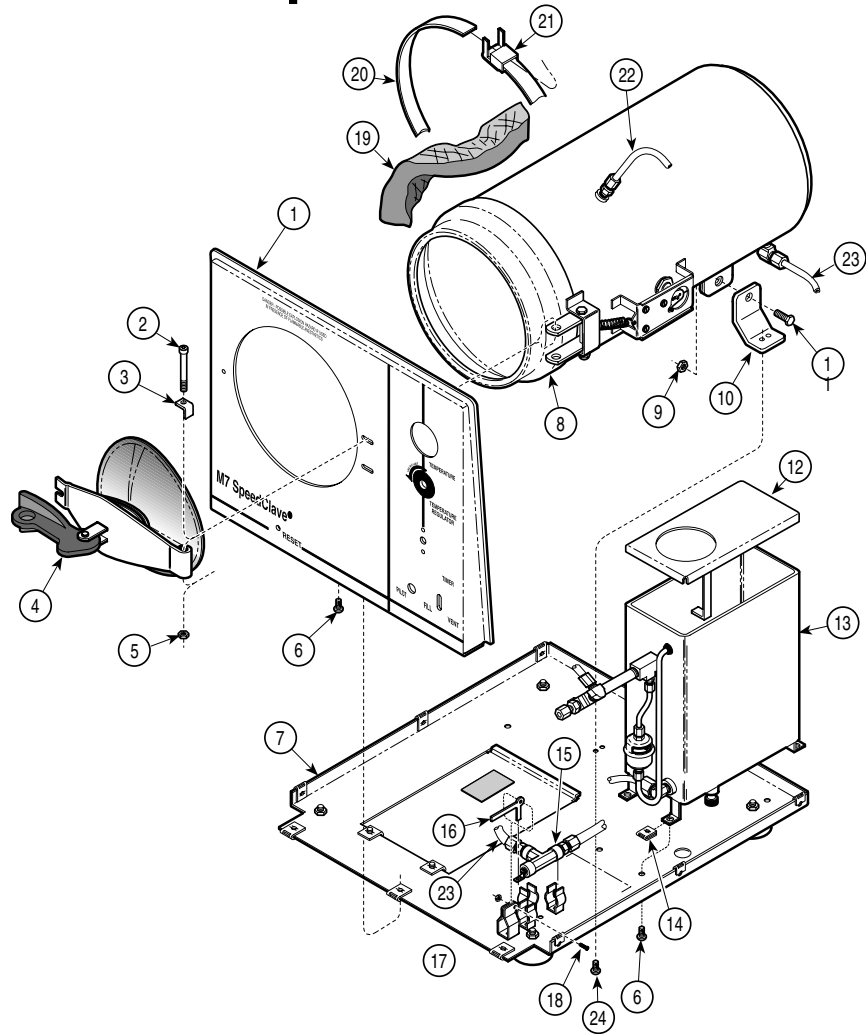

MA242701


**Used on Units with Serial Numbers:**  
**CP1000 thru CP1061    CR1000 thru CR1154    CS1000 thru CS1975**  
**RB1000 thru RB7986    DR1000 thru DR1014**

| Item | Part No.     | Description                                                                                       | Qty. | Item | Part No.    | Description                                                       | Qty. |
|------|--------------|---------------------------------------------------------------------------------------------------|------|------|-------------|-------------------------------------------------------------------|------|
| 1    |              | Panel (Refer to "Front Panel Components" Elsewhere) .....                                         | Ref  | 9    | 041-0250-15 | Nut .....                                                         | 1    |
| 2    | 040-0010-103 | Screw .....                                                                                       | 1    | 10   | H98007      | Chamber Leg Bracket .....                                         | 2    |
| 3    | H97991       | Door Stop .....                                                                                   | 1    | 11   | 040-0250-47 | Screw .....                                                       | 1    |
| 4    |              | Door Assembly (Refer to "Door Assembly" Elsewhere) .....                                          | Ref  | 12   | H98091      | Tank Cover .....                                                  | 1    |
| 5    | 041-0010-10  | Torque Nut .....                                                                                  | 1    | 13   |             | Tank Assembly (Refer to "Tank Assembly" Elsewhere) .....          | Ref  |
| 6    | 040-0010-18  | Screw .....                                                                                       | 7    | 14   | H352623     | Clip .....                                                        | 4    |
| 7    |              | Base Assembly (Refer to "Base Components" Elsewhere) .....                                        | Ref  | 15   |             | Vent Valve Components (Refer to "Valve Assembly" Elsewhere) ..... | Ref  |
| 8    | 029-1865-00  | Shell Assembly [M7-001 & M7-004] (Refer to "Chamber Components {Side or Bottom}" Elsewhere) ..... | 1    | 16   | H98074      | Lever .....                                                       | 1    |
|      | H284702-1    | Shell Assembly [M7-005] (Refer to "Chamber Components {Side or Bottom}" Elsewhere) .....          | 1    | 17   | 041-0010-10 | Nut .....                                                         | 1    |
|      | H284702-2    | Shell Assembly [M7-002] (Refer to "Chamber Components {Side or Bottom}" Elsewhere) .....          | 1    | 18   | 040-0010-73 | Screw .....                                                       | 1    |
|      | H284702-3    | Shell Assembly [M7-003] (Refer to "Chamber Components {Side or Bottom}" Elsewhere) .....          | 1    | 19   | H97980      | Insulation .....                                                  | 1    |
|      |              |                                                                                                   |      | 20   | H99388-29   | Banding Strap (shown) .....                                       | 2    |
|      |              |                                                                                                   |      | 21   | H99388-27   | Banding Strap End .....                                           | 1    |
|      |              |                                                                                                   |      | 22   | H99387      | Banding Strap Clip .....                                          | 2    |
|      |              |                                                                                                   |      | 23   | H335943     | Tubing .....                                                      | 1    |
|      |              |                                                                                                   |      | 24   | H98128      | Tubing .....                                                      | 1    |
|      |              |                                                                                                   |      |      | 040-0010-74 | Screw .....                                                       | 4    |

**Always Specify Model & Serial Number**

# Sterilizer Main Components

## SECTION VI PARTS LIST

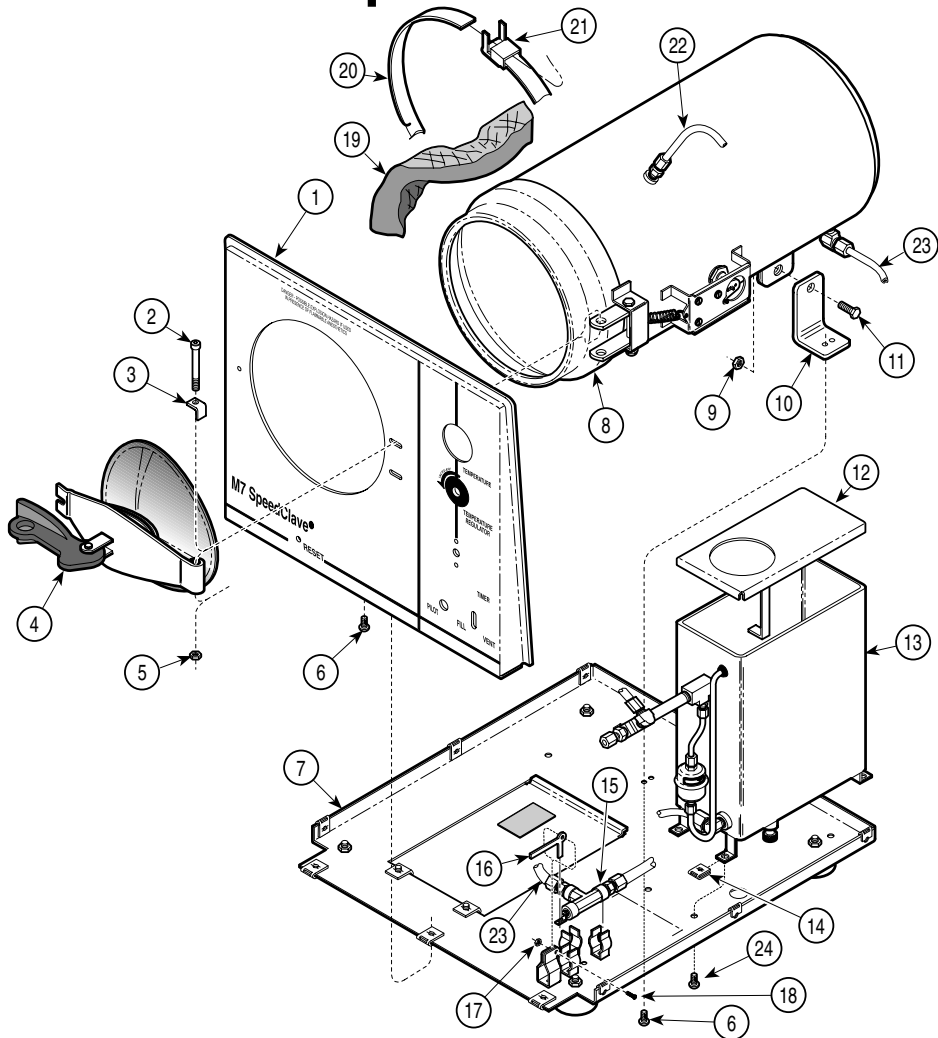

MA242702


**Used on Units with Serial Numbers:**  
**CP1062 thru CP1149   CR1155 thru CR1511   CS1976 thru CS7753**  
**RB7987 thru RB17992   DR1015 thru DR1267**

| Item | Part No.     | Description                                                                                      | Qty. | Item | Part No.    | Description                                                       | Qty. |
|------|--------------|--------------------------------------------------------------------------------------------------|------|------|-------------|-------------------------------------------------------------------|------|
| 1    |              | Panel (Refer to "Front Panel Components" Elsewhere) .....                                        | Ref  | 9    | 041-0250-15 | Nut .....                                                         | 1    |
| 2    | 040-0010-103 | Screw .....                                                                                      | 1    | 10   | 051-0721-00 | Chamber Leg Bracket .....                                         | 1    |
| 3    | H97991       | Door Stop .....                                                                                  | 1    | 11   | 040-0250-47 | Screw .....                                                       | 1    |
| 4    |              | Door Assembly (Refer to "Door Assembly" Elsewhere) .....                                         | Ref  | 12   | H98091      | Tank Cover .....                                                  | 1    |
| 5    | 041-0010-10  | Torque Nut .....                                                                                 | 1    | 13   |             | Tank Assembly (Refer to "Tank Assembly" Elsewhere) .....          | Ref  |
| 6    | 040-0010-125 | Screw .....                                                                                      | 5    | 14   | H352623     | Clip .....                                                        | 4    |
| 7    |              | Base Assembly (Refer to "Base Components" Elsewhere) .....                                       | Ref  | 15   |             | Vent Valve Components (Refer to "Valve Assembly" Elsewhere) ..... | Ref  |
| 8    | 029-1865-00  | Shell Assembly [M7-001 & M7-004] (Refer to "Chamber Components (Side or Lower)" Elsewhere) ..... | 1    | 16   | H98074      | Lever .....                                                       | 1    |
|      | H284702-1    | Shell Assembly [M7-005] (Refer to "Chamber Components (Side or Lower)" Elsewhere) .....          | 1    | 17   | 041-0010-10 | Nut .....                                                         | 1    |
|      | H284702-2    | Shell Assembly [M7-002] (Refer to "Chamber Components (Side or Lower)" Elsewhere) .....          | 1    | 18   | 040-0010-73 | Screw .....                                                       | 1    |
|      | H284702-3    | Shell Assembly [M7-003] (Refer to "Chamber Components (Side or Lower)" Elsewhere) .....          | 1    | 19   | H97980      | Insulation .....                                                  | 1    |
|      |              |                                                                                                  |      | 20   | H99388-29   | Banding Strap (shown) .....                                       | 2    |
|      |              |                                                                                                  |      |      | H99388-27   | Banding Strap End .....                                           | 1    |
|      |              |                                                                                                  |      | 21   | H99387      | Banding Strap Clip .....                                          | 2    |
|      |              |                                                                                                  |      | 22   | H335943     | Tubing .....                                                      | 1    |
|      |              |                                                                                                  |      | 23   | H98128      | Tubing .....                                                      | 1    |
|      |              |                                                                                                  |      | 24   | 040-0010-95 | Screw .....                                                       | 4    |

**Always Specify Model & Serial Number**

# Sterilizer Main Components

## SECTION VI PARTS LIST

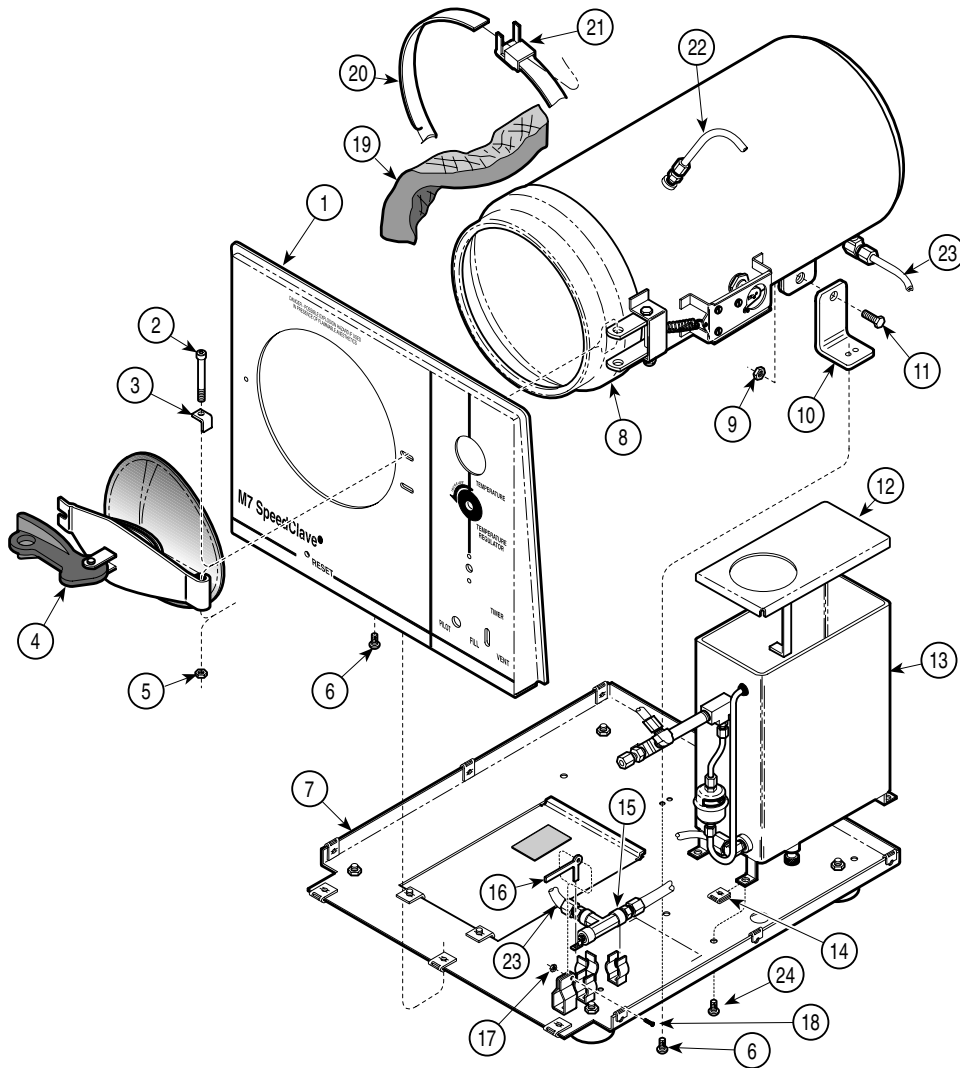

MA242702


### Used on Units with Serial Numbers CR1511 thru CR1685, FM1000 thru FM1425 and FN1000 thru FN1229

| Item | Part No.     | Description                                                                                      | Qty. | Item | Part No.    | Description                                                       | Qty. |
|------|--------------|--------------------------------------------------------------------------------------------------|------|------|-------------|-------------------------------------------------------------------|------|
| 1    |              | Panel (Refer to "Front Panel Components" Elsewhere) .....                                        | Ref  | 9    | 041-0250-15 | Nut .....                                                         | 1    |
| 2    | 040-0010-103 | Screw .....                                                                                      | 1    | 10   | 051-0721-00 | Chamber Leg Bracket .....                                         | 1    |
| 3    | H97991       | Door Stop .....                                                                                  | 1    | 11   | 040-0250-47 | Screw .....                                                       | 1    |
| 4    |              | Door Assembly (Refer to "Door Assembly" Elsewhere) .....                                         | Ref  | 12   | H98091      | Tank Cover .....                                                  | 1    |
| 5    | 041-0010-10  | Torque Nut .....                                                                                 | 1    | 13   |             | Tank Assembly (Refer to "Tank Assembly" Elsewhere) .....          | Ref  |
| 6    | 040-0010-125 | Screw .....                                                                                      | 5    | 14   | H352623     | Clip .....                                                        | 4    |
| 7    |              | Base Assembly (Refer to "Base Components" Elsewhere) .....                                       | Ref  | 15   |             | Vent Valve Components (Refer to "Valve Assembly" Elsewhere) ..... | Ref  |
| 8    | 029-1865-00  | Shell Assembly [M7-001 & M7-004] (Refer to "Chamber Components {Side or Lower}" Elsewhere) ..... | 1    | 16   | H98074      | Lever .....                                                       | 1    |
|      | H284702-1    | Shell Assembly [M7-005] (Refer to "Chamber Components {Side or Lower}" Elsewhere) .....          | 1    | 17   | 041-0010-10 | Nut .....                                                         | 1    |
|      | 029-1865-01  | Shell Assembly [M7-002] (Refer to "Chamber Components {Side or Lower}" Elsewhere) .....          | 1    | 18   | 040-0010-73 | Screw .....                                                       | 1    |
|      | 029-1865-02  | Shell Assembly [M7-003] (Refer to "Chamber Components {Side or Lower}" Elsewhere) .....          | 1    | 19   | H97980      | Insulation .....                                                  | 1    |
|      |              |                                                                                                  |      | 20   | H99388-29   | Banding Strap (shown) .....                                       | 2    |
|      |              |                                                                                                  |      |      | H99388-27   | Banding Strap End .....                                           | 1    |
|      |              |                                                                                                  |      | 21   | H99387      | Banding Strap Clip .....                                          | 2    |
|      |              |                                                                                                  |      | 22   | H335943     | Tubing .....                                                      | 1    |
|      |              |                                                                                                  |      | 23   | 052-0251-00 | Tubing .....                                                      | 1    |
|      |              |                                                                                                  |      | 24   | 040-0010-95 | Screw .....                                                       | 4    |

Always Specify Model & Serial Number

# Sterilizer Main Components

## SECTION VI PARTS LIST

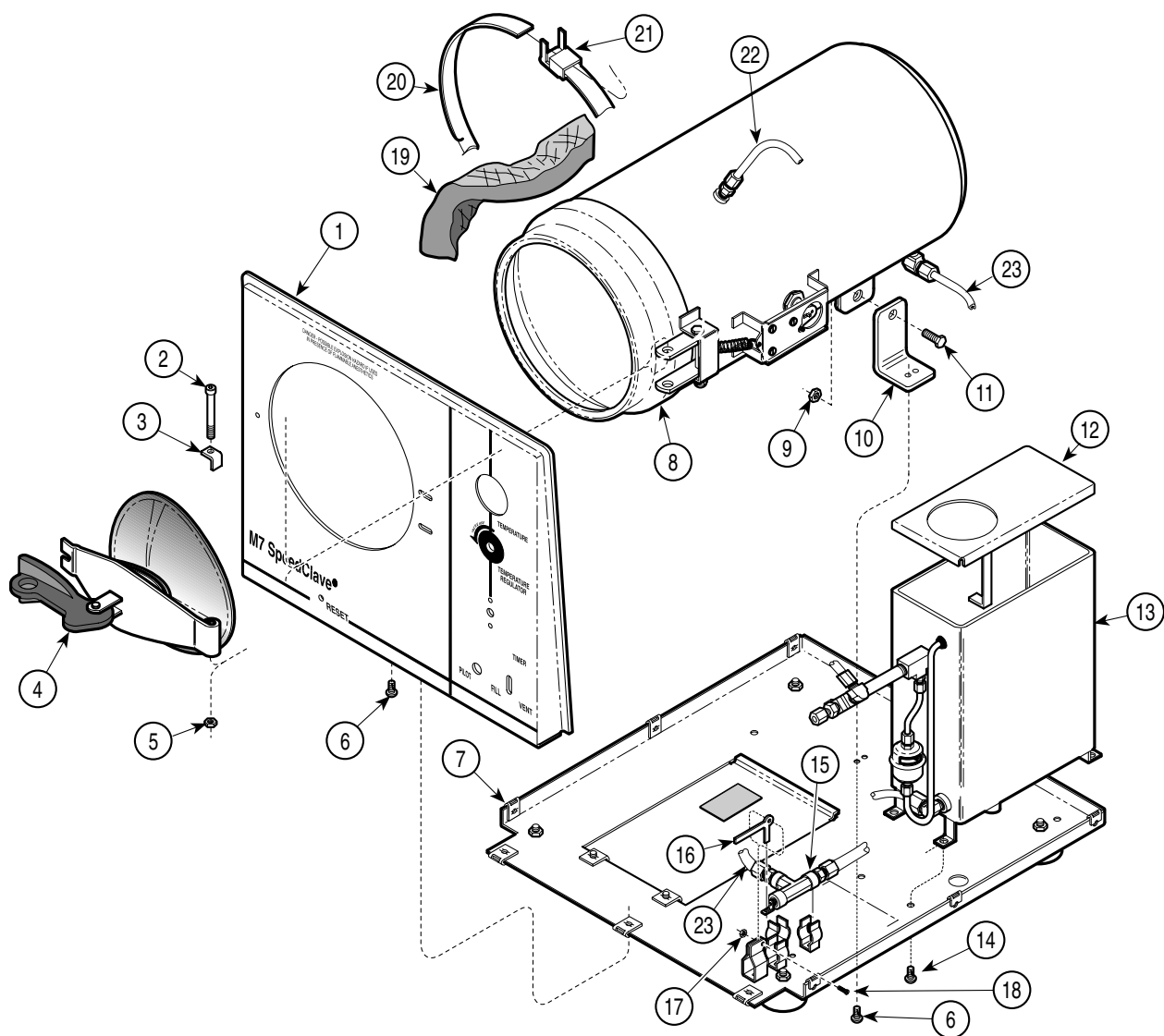


MA242702

**Used on Units with Serial Numbers:RB17993, CR1686, DR1268, CP1150, CS7754, FM1426 and FN11230 thru Present**

| Item | Part No.     | Description                                                                                          | Qty. | Item | Part No.    | Description                                                       | Qty. |
|------|--------------|------------------------------------------------------------------------------------------------------|------|------|-------------|-------------------------------------------------------------------|------|
| 1    |              | Panel (Refer to "Front Panel Components" Elsewhere) .....                                            | Ref  | 9    | 041-0250-15 | Nut .....                                                         | 1    |
| 2    | 040-0010-103 | Screw .....                                                                                          | 1    | 10   | 051-0721-00 | Chamber Leg Bracket .....                                         | 1    |
| 3    | H97991       | Door Stop .....                                                                                      | 1    | 11   | 040-0250-47 | Screw .....                                                       | 1    |
| 4    |              | Door Assembly (Refer to "Door Assembly" Elsewhere) .....                                             | Ref  | 12   | H98091      | Tank Cover .....                                                  | 1    |
| 5    | 041-0010-10  | Torque Nut .....                                                                                     | 1    | 13   |             | Tank Assembly (Refer to "Tank Assembly" Elsewhere) .....          | Ref  |
| 6    | 040-0010-125 | Screw .....                                                                                          | 5    | 14   | 040-0010-81 | Screw .....                                                       | 4    |
| 7    |              | Base Assembly (Refer to "Base Components" Elsewhere) .....                                           | Ref  | 15   |             | Vent Valve Components (Refer to "Valve Assembly" Elsewhere) ..... | Ref  |
| 8    | 029-1865-00  | Shell Assm. [M7-001,M7-004 & M7-005] (Refer to "Chamber Components (Side or Lower)" Elsewhere) ..... | 1    | 16   | H98074      | Lever .....                                                       | 1    |
|      | 029-1865-01  | Shell Assembly [M7-002 & M7-006] (Refer to "Chamber Components (Side or Lower)" Elsewhere) .....     | 1    | 17   | 041-0010-10 | Nut .....                                                         | 1    |
|      | 029-1865-02  | Shell Assembly [M7-003] (Refer to "Chamber Components (Side or Lower)" Elsewhere) .....              | 1    | 18   | 040-0010-73 | Screw .....                                                       | 1    |
|      |              |                                                                                                      |      | 19   | H97980      | Insulation .....                                                  | 1    |
|      |              |                                                                                                      |      | 20   | H99388-29   | Banding Strap (shown) .....                                       | 2    |
|      |              |                                                                                                      |      |      | H99388-27   | Banding Strap End .....                                           | 1    |
|      |              |                                                                                                      |      | 21   | H99387      | Banding Strap Clip .....                                          | 2    |
|      |              |                                                                                                      |      | 22   | H335943     | Tubing .....                                                      | 1    |
|      |              |                                                                                                      |      | 23   | 052-0251-00 | Tubing .....                                                      | 1    |

**Always Specify Model & Serial Number**

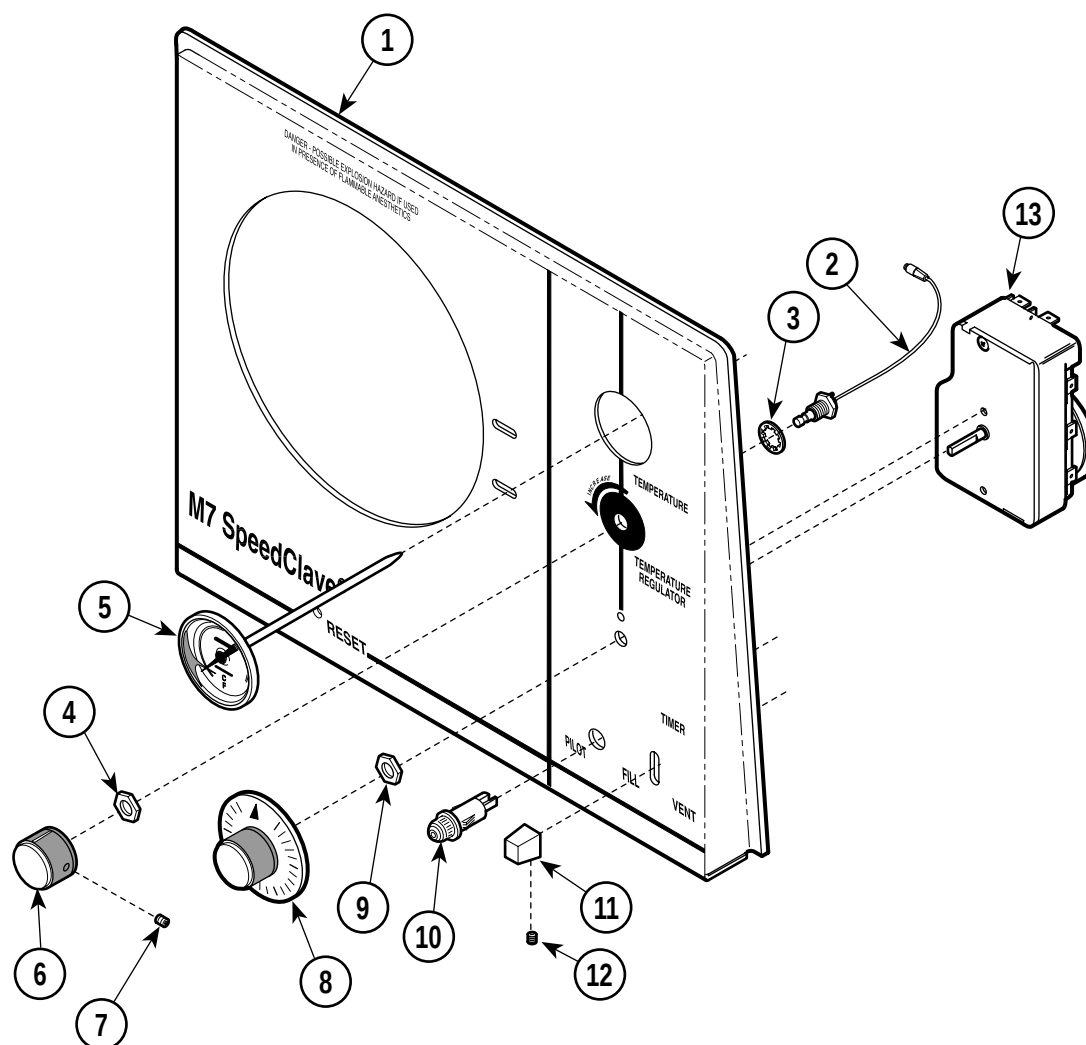

MA241102


**Used on Units with Serial Numbers CP1000 thru CP1069,  
CR1000 thru CR1125, CS1000 thru CS1694, RB1000 thru RB7499**

| Item | Part No.    | Description                                               | Qty. | Item | Part No.      | Description                                           | Qty. |
|------|-------------|-----------------------------------------------------------|------|------|---------------|-------------------------------------------------------|------|
| 1    | H335992-1   | Ritter Panel (M7-001) .....                               | 1    | 8    | H98062        | Timer Knob .....                                      | 1    |
|      | H335992-5   | Midmark Panel (M7-002, M7-003,<br>M7-004) .....           | 1    | 9    |               | Nut .....                                             | Ref  |
| 2    | H98064      | Flexible Shaft Assembly (Includes<br>Items 3 and 4) ..... | 1    | 10   | H98036        | Pilot Light (M7-001, M7-003, M7-004) .....            | 1    |
|      |             |                                                           |      |      | H97747        | Pilot Light (M7-002) .....                            | 1    |
| 3    | • HWL008-27 | • Lockwasher .....                                        | 1    | 11   | H98081        | Fill / Vent Knob Assembly (Includes<br>Item 12) ..... | 1    |
| 4    | • H98149    | • Nut .....                                               | 1    | 12   | • 040-0008-40 | • Setscrew .....                                      | 1    |
| 5    | 002-0242-00 | Thermometer .....                                         | 1    | 13   | 002-0373-00   | Timer (M7-001, M7-003, M7-004) .....                  | 1    |
| 6    | H98063      | Control Knob (Includes Item 7) .....                      | 1    |      | 002-0373-01   | Timer (M7-002) .....                                  | 1    |
| 7    | •           | • Setscrew .....                                          | 1    |      |               |                                                       |      |

### Always Specify Model & Serial Number

# Front Panel Components

## SECTION VI PARTS LIST

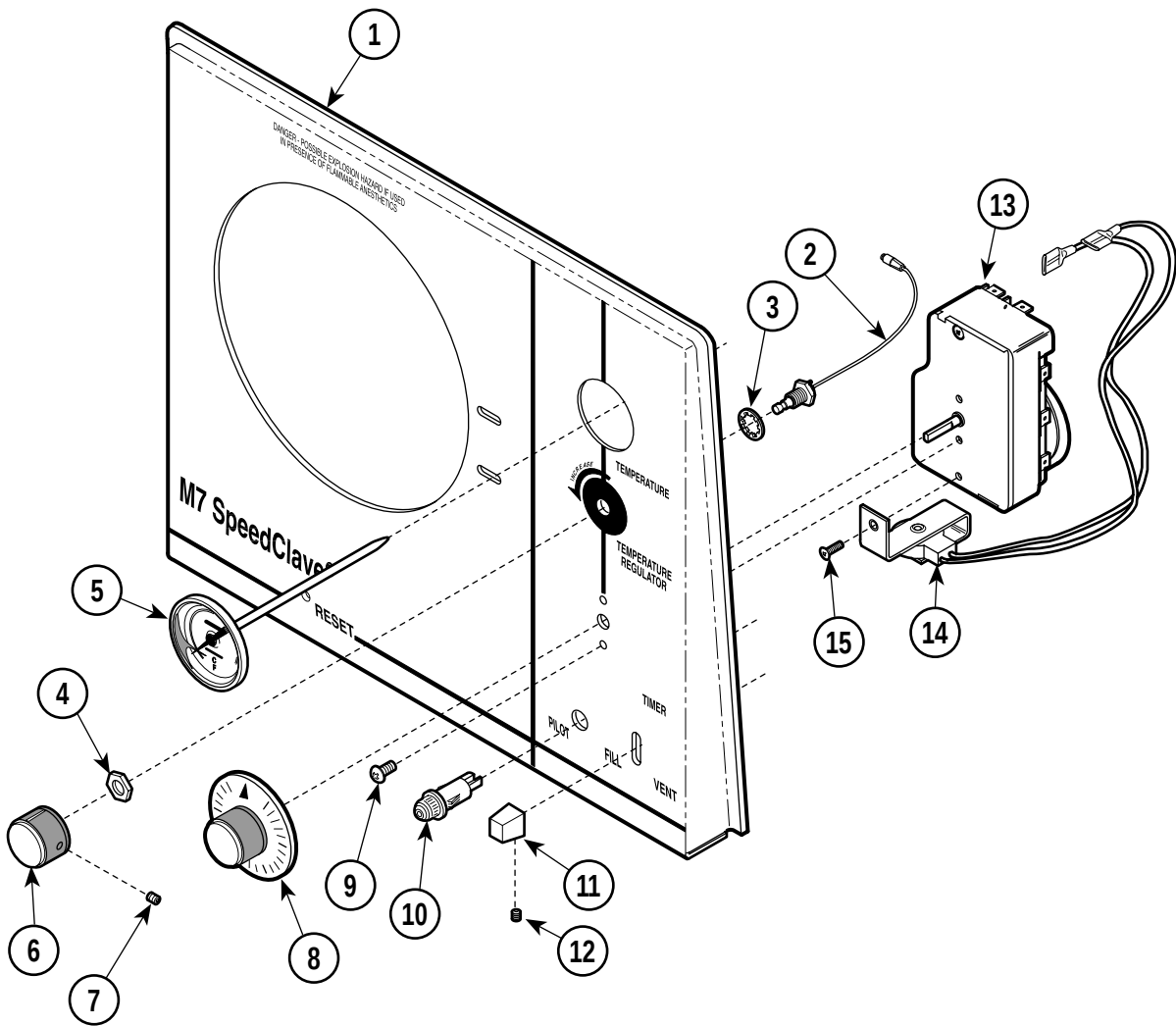

MA241101


### Used on units with Serial Number RB7500 thru RB7999

| Item | Part No.    | Description                                      | Qty. | Item | Part No.      | Description                                                    | Qty. |
|------|-------------|--------------------------------------------------|------|------|---------------|----------------------------------------------------------------|------|
| 1    | 050-2256-00 | Ritter Panel (M7-001, M7-005)                    | 1    | 9    | 040-0008-29   | Screw                                                          | 2    |
|      | 050-2256-01 | Midmark Panel (M7-002, M7-003, M7-004)           | 1    | 10   | H98036        | Pilot Light (M7-001, M7-003, M7-004, M7-005)                   | 1    |
| 2    | H98064      | Flexible Shaft Assembly (Includes Items 3 and 4) | 1    |      | H97747        | Pilot Light (M7-002)                                           | 1    |
| 3    | • HWL008-27 | • Lockwasher                                     | 1    | 11   | H98081        | Fill / Vent Knob Assembly (Incl. Item 12)                      | 1    |
| 4    | • H98149    | • Nut                                            | 1    | 12   | • 040-0008-40 | • Setscrew                                                     | 1    |
| 5    | 002-0242-00 | Thermometer                                      | 1    | 13   | 002-0373-00   | Timer (M7-001, M7-003, M7-004, M7-005) [Incl. Items 14 and 15] | 1    |
| 6    | H98063      | Control Knob (Includes Item 7)                   | 1    |      | 002-0373-01   | Timer (M7-002) [Incl. Items 14 and 15]                         | 1    |
| 7    | •           | • Setscrew                                       | 1    | 14   | • 015-0745-02 | • Timer Buzzer (120 volt)                                      | 1    |
| 8    | H98062      | Timer Knob                                       | 1    |      | • 015-0745-03 | • Timer Buzzer (230 volt)                                      | 1    |
|      |             |                                                  |      | 15   | • 040-0008-29 | • Screw                                                        | 1    |

Always Specify Model & Serial Number

# Front Panel Components

## SECTION VI PARTS LIST

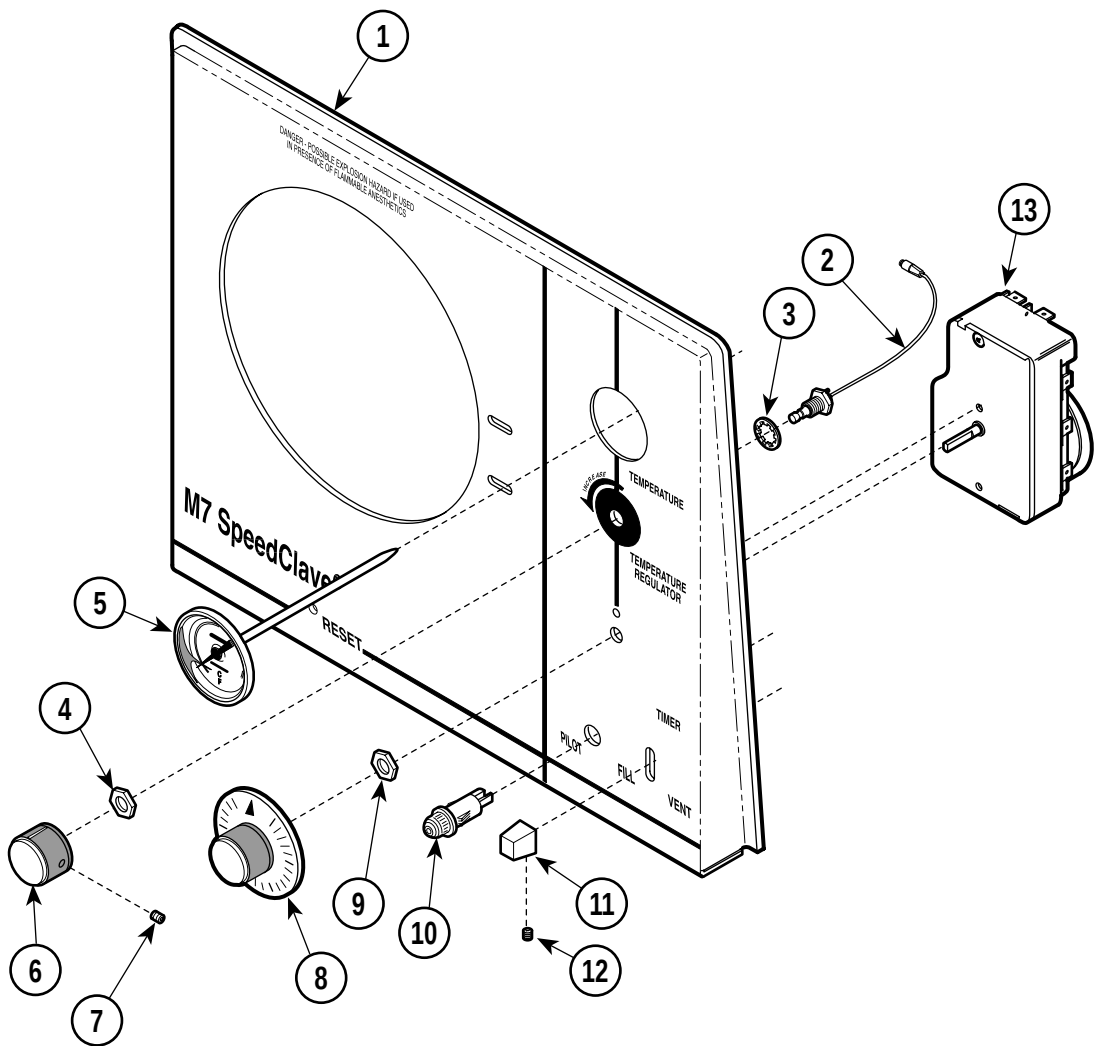

MA241102


### Used on units with Serial Number RB8000 thru RB8106

| Item | Part No.    | Description                                            | Qty. | Item | Part No.      | Description                                        | Qty. |
|------|-------------|--------------------------------------------------------|------|------|---------------|----------------------------------------------------|------|
| 1    | H335992-1   | Ritter Panel (M7-001) .....                            | 1    | 8    | H98062        | Timer Knob .....                                   | 1    |
|      | H335992-5   | Midmark Panel (M7-002, M7-003, M7-004) .....           | 1    | 9    |               | Nut .....                                          | Ref  |
| 2    | H98064      | Flexible Shaft Assembly (Includes Items 3 and 4) ..... | 1    | 10   | H98036        | Pilot Light (M7-001, M7-003, M7-004) ....          | 1    |
| 3    | • HWL008-27 | • Lockwasher .....                                     | 1    |      | H97747        | Pilot Light (M7-002) .....                         | 1    |
| 4    | • H98149    | • Nut .....                                            | 1    | 11   | H98081        | Fill / Vent Knob Assembly (Includes Item 12) ..... | 1    |
| 5    | 002-0242-00 | Thermometer .....                                      | 1    | 12   | • 040-0008-40 | • Setscrew .....                                   | 1    |
| 6    | H98063      | Control Knob (Includes Items 7) .....                  | 1    | 13   | 002-0373-00   | Timer (M7-001, M7-003, M7-004) .....               | 1    |
| 7    | •           | • Setscrew .....                                       | 1    |      | 002-0373-01   | Timer (M7-002) .....                               | 1    |

Always Specify Model & Serial Number

# Front Panel Components

## SECTION VI PARTS LIST

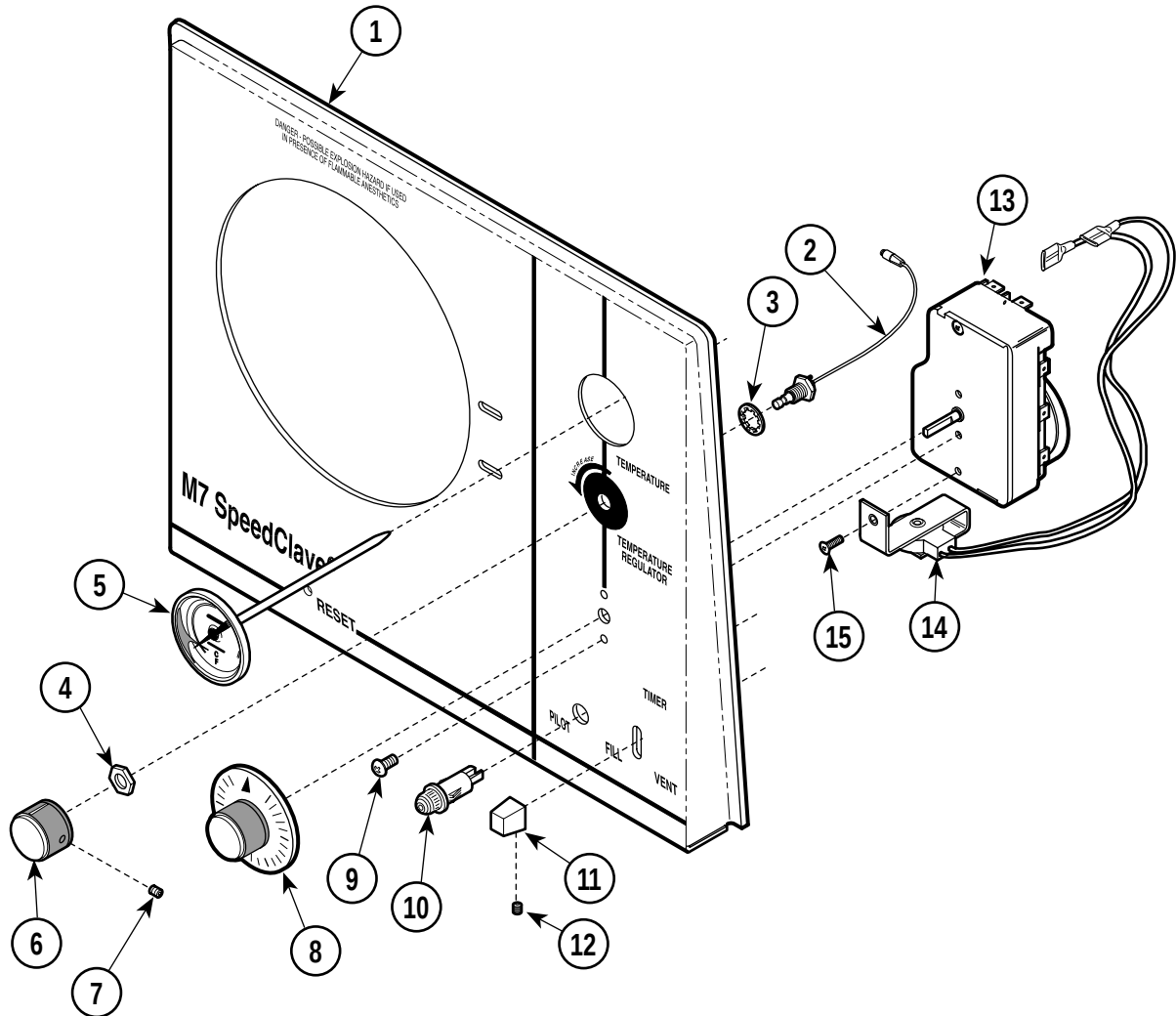

MA241101


**Used on Units with Serial Numbers:**  
**CP1070 thru CP1091    CR1126 thru CR1259    CS1695 thru CS3488**  
**RB8107 thru RB11004**

| Item | Part No.                   | Description                                                                            | Qty.   | Item | Part No.      | Description                                                             | Qty. |
|------|----------------------------|----------------------------------------------------------------------------------------|--------|------|---------------|-------------------------------------------------------------------------|------|
| 1    | 050-2256-00<br>050-2256-01 | Ritter Panel (M7-001, M7-005) .....<br>Midmark Panel (M7-002, M7-003,<br>M7-004) ..... | 1<br>1 | 9    | 040-0008-29   | Screw .....                                                             | 2    |
| 2    | H98064                     | Flexible Shaft Assembly (Includes<br>Item 3 and 4) .....                               | 1      | 10   | H98036        | Pilot Light (M7-001, M7-003, M7-004,<br>M7-005) .....                   | 1    |
| 3    | • HWL008-27                | • Lockwasher .....                                                                     | 1      |      | H97747        | Pilot Light (M7-002) .....                                              | 1    |
| 4    | • H98149                   | • Nut .....                                                                            | 1      | 11   | H98081        | Fill / Vent Knob Assy. (Includes Item 12) ..                            | 1    |
| 5    | 002-0242-00                | Thermometer .....                                                                      | 1      | 12   | • 040-0008-40 | • Setscrew .....                                                        | 1    |
| 6    | H98063                     | Control Knob (Includes Item 7) .....                                                   | 1      | 13   | 002-0373-00   | Timer (M7-001, M7-003, M7-004,<br>M7-005) [Incl. Items 14 and 15] ..... | 1    |
| 7    | •                          | • Setscrew .....                                                                       | 1      |      | 002-0373-01   | Timer (M7-002) [Incl. Items 14 and 15] ..                               | 1    |
| 8    | H98062                     | Timer Knob .....                                                                       | 1      | 14   | • 015-0745-02 | • Timer Buzzer (120 volt) .....                                         | 1    |
|      |                            |                                                                                        |        |      | • 015-0745-03 | • Timer Buzzer (230 volt) .....                                         | 1    |
|      |                            |                                                                                        |        | 15   | • 040-0008-29 | • Screw .....                                                           | 1    |

**Always Specify Model & Serial Number**

# Front Panel Components

## SECTION VI PARTS LIST

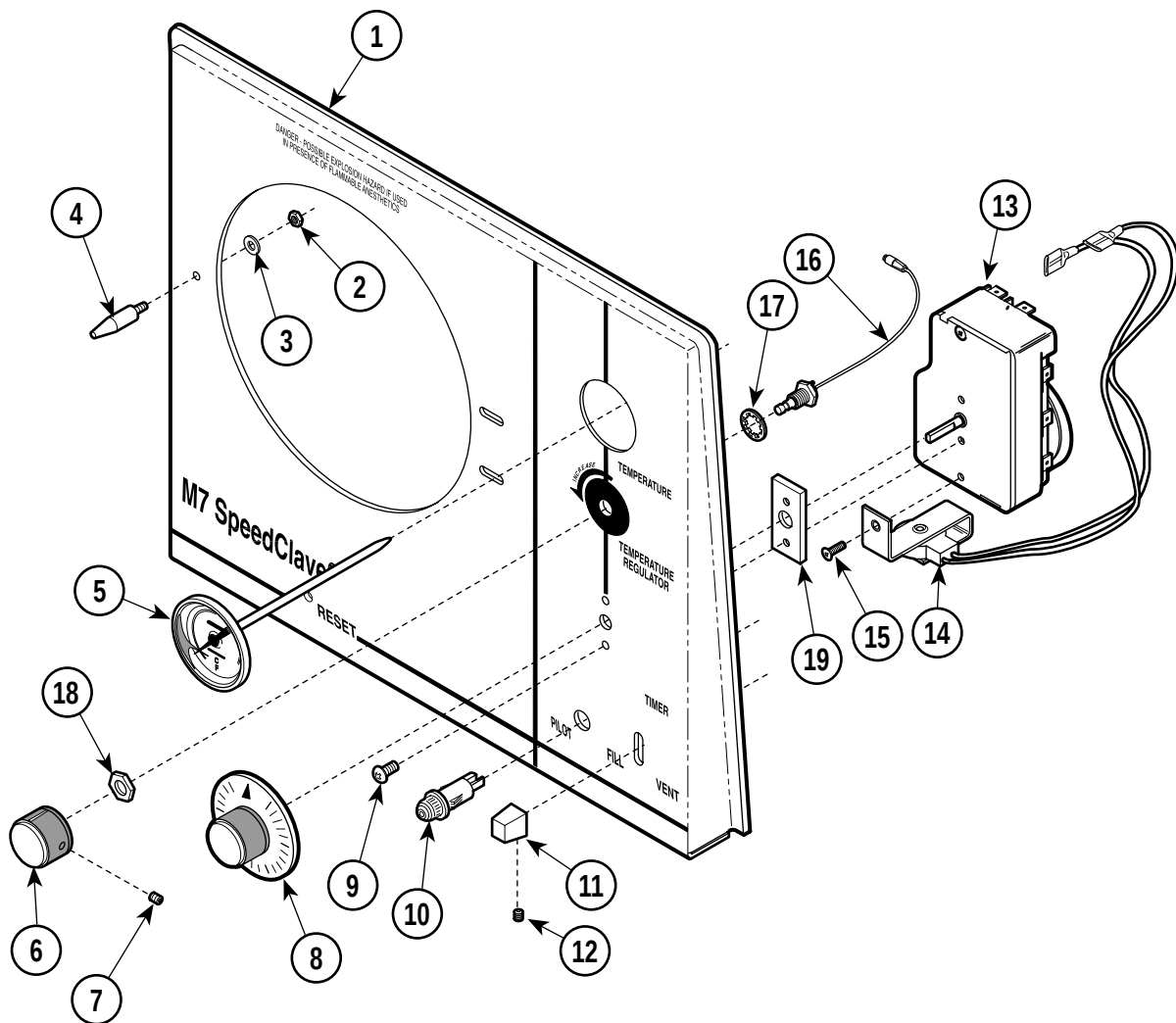

MA241103


**Used on Units with Serial Numbers RB11005 thru RB18474, CP1092 thru CP1149, DR1015 thru DR1267, CR-1260 thru CR1724, CS3489 thru CS8328, FM1000 thru FM1545 and FN1000 thru FN1309**

| Item | Part No.    | Description                                        | Qty. | Item | Part No.      | Description                                                          | Qty. |
|------|-------------|----------------------------------------------------|------|------|---------------|----------------------------------------------------------------------|------|
| 1    | 050-2360-00 | Ritter Panel (M7-001) .....                        | 1    |      | H97747        | Pilot Light (M7-002) .....                                           | 1    |
|      | 050-2360-01 | Midmark Panel (M7-002, M7-003, M7-004) .....       | 1    | 11   | H98076        | Fill / Vent Knob Assembly (Includes Item 12) .....                   | 1    |
|      | 050-2360-12 | Special Panel (M7-005, M7-006) .....               | 1    | 12   | • 040-0008-40 | • Setscrew .....                                                     | 1    |
| 2    | 041-0010-01 | Nut .....                                          | 1    | 13   | 002-0373-00   | Timer (M7-001, M7-003, M7-004, M7-005) [Incl. Items 14 and 15] ..... | 1    |
| 3    | 045-0001-08 | Lockwasher .....                                   | 1    |      | 002-0373-01   | Timer (M7-002) [Incl. Items 14 and 15] ..                            | 1    |
| 4    | 057-0370-00 | Alignment Pin .....                                | 1    | 14   | • 015-0745-02 | • Timer Buzzer (120 volt) .....                                      | 1    |
| 5    | 002-0242-00 | Thermometer .....                                  | 1    |      | • 015-0745-03 | • Timer Buzzer (230 volt) .....                                      | 1    |
| 6    | H98063      | Control Knob (Includes Item 7) .....               | 1    | 15   | • 040-0008-29 | • Screw .....                                                        | 1    |
| 7    | •           | • Setscrew .....                                   | 1    | 16   | H98064        | Flexible Shaft Assembly (Includes Item 17 and 18) .....              | 1    |
| 8    | H98062      | Timer Knob .....                                   | 1    | 17   | • HWL008-27   | • Lockwasher .....                                                   | 1    |
| 9    | 040-0008-29 | Screw .....                                        | 2    | 18   | • H98149      | • Nut .....                                                          | 1    |
| 10   | H98036      | Pilot Light (M7-001, M7-003, M7-004, M7-005) ..... | 1    | 19   | 050-3051-00   | Spacer .....                                                         | 1    |

Always Specify Model & Serial Number

# Front Panel Components

## SECTION VI PARTS LIST

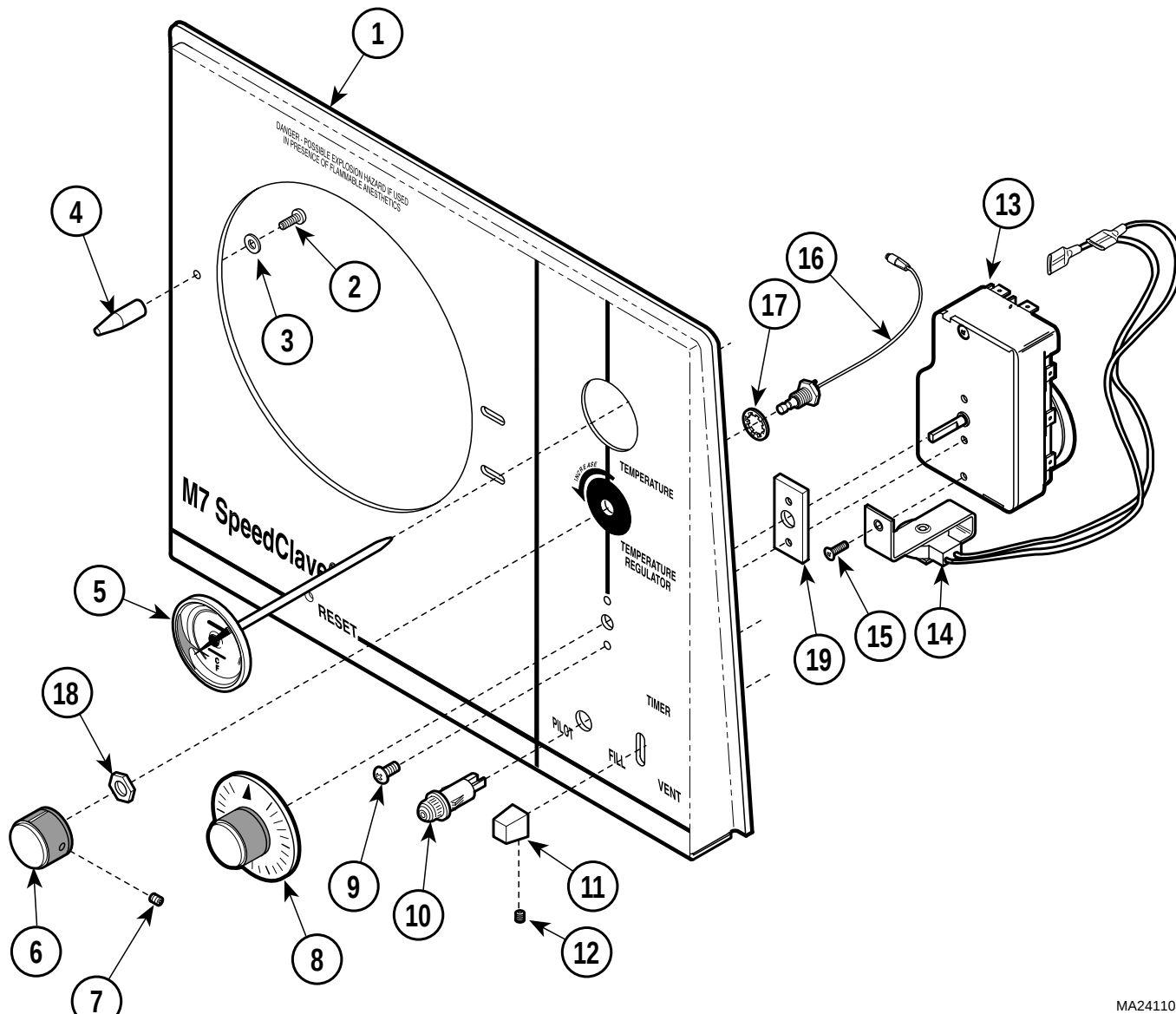

MA241104


**Used on Units with Serial Numbers RB18475 thru RB21249,  
CR1725 thru CR1904, CS8319 and CS11602, FM1546 thru FM1875, and FN1310  
thru FN1499**

| Item | Part No.    | Description                          | Qty. | Item | Part No.      | Description                              | Qty. |
|------|-------------|--------------------------------------|------|------|---------------|------------------------------------------|------|
| 1    | 050-2360-00 | Ritter Panel (M7-001) .....          | 1    | 11   | H98076        | Fill / Vent Knob Assembly (Includes      |      |
|      | 050-2360-01 | Midmark Panel (M7-002, M7-003,       |      |      |               | Item 12) .....                           | 1    |
|      |             | M7-004) .....                        | 1    | 12   | • 040-0008-40 | • Setscrew .....                         | 1    |
|      | 050-2360-12 | Special Panel (M7-005, M7-006) ..... | 1    | 13   | 002-0373-00   | Timer (M7-001, M7-003, M7-004,           |      |
| 2    | 040-0010-75 | Screw .....                          | 1    |      |               | M7-005) [Incl. Items 14 and 15] .....    | 1    |
| 3    | 045-0001-08 | Lockwasher .....                     | 1    |      | 002-0373-01   | Timer (M7-002, M7-006)                   |      |
| 4    | 057-0461-00 | Alignment Pin .....                  | 1    |      |               | [Incl. Items 14 and 15] .....            | 1    |
| 5    | 002-0242-00 | Thermometer .....                    | 1    |      | 015-0744-02   | Timer (Units with "DR" S/N Prefix) ..... | 1    |
| 6    | H98063      | Control Knob (Includes Item 7) ..... | 1    | 14   | • 015-0745-02 | • Timer Buzzer (120 volt) .....          | 1    |
| 7    | •           | • Setscrew .....                     | 1    |      | • 015-0745-03 | • Timer Buzzer (230 volt) .....          | 1    |
| 8    | H98062      | Timer Knob .....                     | 1    | 15   | • 040-0008-29 | • Screw .....                            | 1    |
| 9    | 040-0008-29 | Screw .....                          | 2    | 16   | H98064        | Flexible Shaft Assembly (Includes        |      |
| 10   | H98036      | Pilot Light (M7-001, M7-003, M7-004, |      |      |               | Item 17 and 18) .....                    | 1    |
|      |             | M7-005) .....                        | 1    | 17   | • HWL008-27   | • Lockwasher .....                       | 1    |
|      | H97747      | Pilot Light (M7-002, M7-006) .....   | 1    | 18   | • H98149      | • Nut .....                              | 1    |
|      |             |                                      |      | 19   | 050-3051-00   | Spacer .....                             | 1    |

**Always Specify Model & Serial Number**

# Front Panel Components

## SECTION VI PARTS LIST

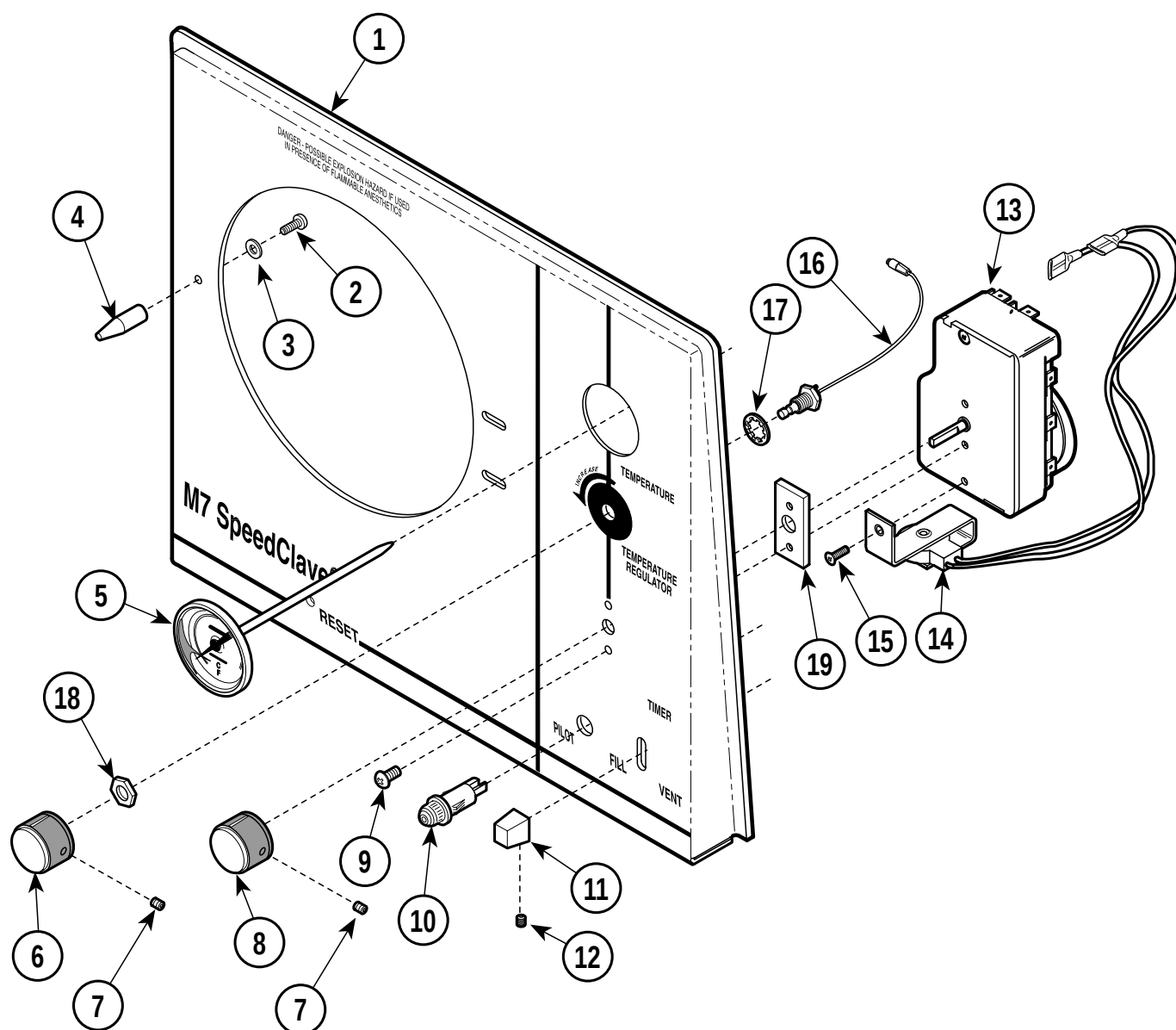


MA241105

**Used on Units with Serial Numbers RB21250, CP1150, DR1268, CR1905, CS11603 FM1876 and FN1500 thru Present**

| Item | Part No.    | Description                                        | Qty. | Item | Part No.      | Description                                                          | Qty. |
|------|-------------|----------------------------------------------------|------|------|---------------|----------------------------------------------------------------------|------|
| 1    | 050-2360-03 | Ritter Panel (M7-001) .....                        | 1    | 11   | H98076        | Fill / Vent Knob Assembly (Includes Item 12) .....                   | 1    |
|      | 050-2360-04 | Midmark Panel (M7-002, M7-003, M7-004) .....       | 1    | 12   | • 040-0008-40 | • Setscrew .....                                                     | 1    |
|      | 050-2360-15 | Special Panel (M7-005, M7-006) .....               | 1    | 13   | 002-0373-00   | Timer (M7-001, M7-003, M7-004, M7-005) [Incl. Items 14 and 15] ..... | 1    |
| 2    | 040-0010-75 | Screw .....                                        | 1    |      | 002-0373-01   | Timer (M7-002, M7-006) [Incl. Items 14 and 15] .....                 | 1    |
| 3    | 045-0001-08 | Lockwasher .....                                   | 1    |      | 015-0744-02   | Timer (Units with "DR" S/N Prefix) .....                             | 1    |
| 4    | 057-0461-00 | Alignment Pin .....                                | 1    | 14   | • 015-0745-02 | • Timer Buzzer (120 volt) .....                                      | 1    |
| 5    | 002-0242-00 | Thermometer .....                                  | 1    |      | • 015-0745-03 | • Timer Buzzer (230 volt) .....                                      | 1    |
| 6    | 053-1008-01 | Control Knob (Includes Item 7) .....               | 1    | 15   | • 040-0008-29 | • Screw .....                                                        | 1    |
| 7    | •           | • Setscrew .....                                   | 1    | 16   | H98064        | Flexible Shaft Assembly (Includes Item 17 and 18) .....              | 1    |
| 8    | 053-1008-00 | Timer Knob .....                                   | 1    | 17   | • HWL008-27   | • Lockwasher .....                                                   | 1    |
| 9    | 040-0008-29 | Screw .....                                        | 2    | 18   | • H98149      | • Nut .....                                                          | 1    |
| 10   | H98036      | Pilot Light (M7-001, M7-003, M7-004, M7-005) ..... | 1    | 19   | 050-3051-00   | Spacer .....                                                         | 1    |
|      | H97747      | Pilot Light (M7-002, M7-006) .....                 | 1    |      |               |                                                                      |      |

Always Specify Model & Serial Number

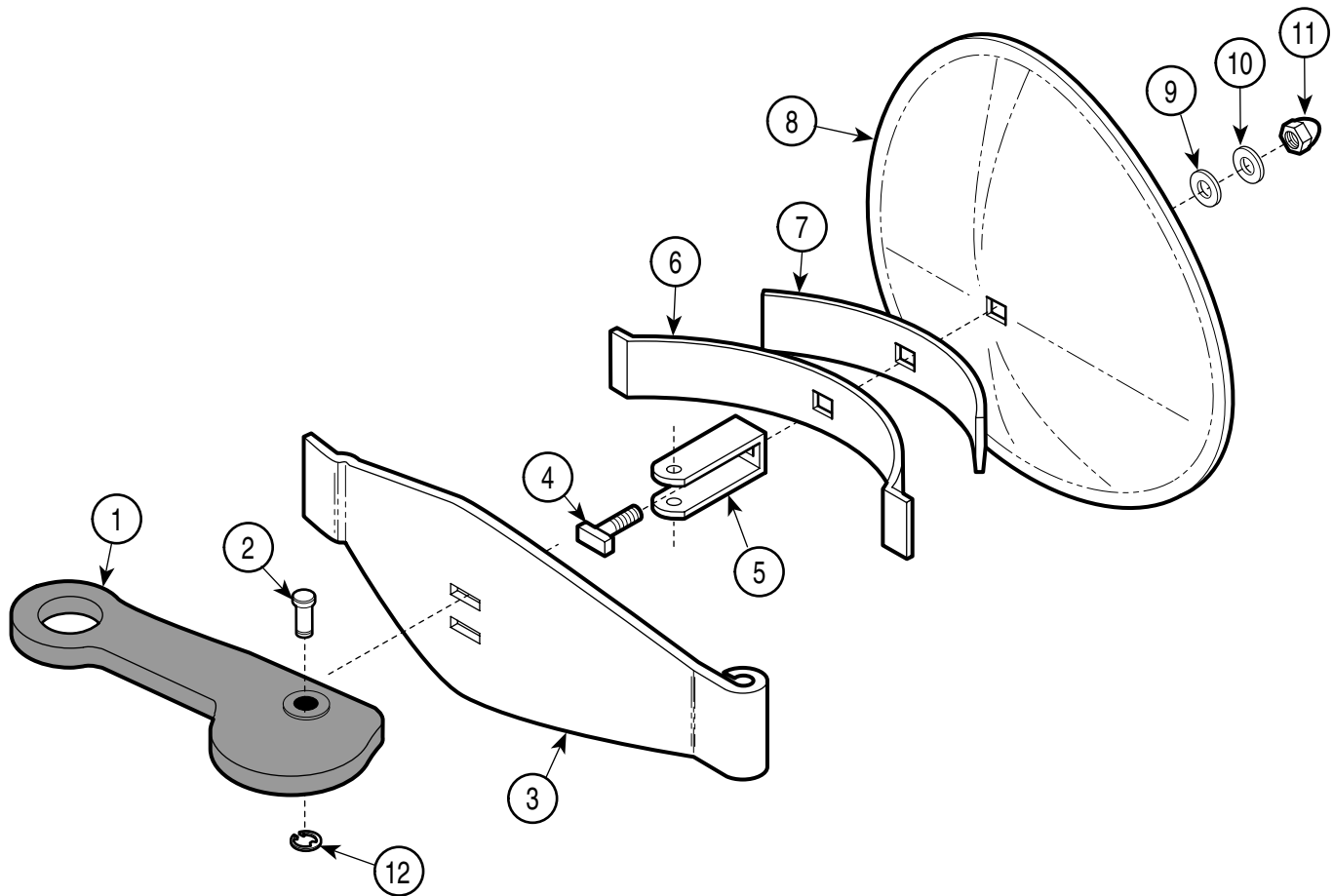


MA240702

**Used on Units with Serial Numbers:**  
**CP1000 thru CP1071 CR1000 thru CR1225 CS1000 thru CS3103**  
**RB1000 thru RB11004 DR1000 thru DR1014**

| Item | Part No.      | Description                                         | Qty. | Item | Part No.      | Description                              | Qty. |
|------|---------------|-----------------------------------------------------|------|------|---------------|------------------------------------------|------|
|      | 002-0249-00   | Door Assembly (Includes Items 1 thru 12) .....      | 1    | 6    | • H96775      | • Long Spring .....                      | 1    |
| 1    | • 002-0248-00 | • Handle (Apply Valve Lubricant #064-0002-01) ..... | 1    | 7    | • H96774      | • Short Spring .....                     | 1    |
| 2    | • H97989      | • Pin .....                                         | 1    | 8    | • H106313     | • Door .....                             | 1    |
| 3    | • H96751-1    | • Door Cross Arm .....                              | 1    | 9    | • 053-1579-00 | • Gasket .....                           | 1    |
| 4    | • H96772      | • "T" Bolt .....                                    | 1    | 10   | • H96771      | • Washer .....                           | 1    |
| 5    | • H96776      | • "U" Bracket .....                                 | 1    | 11   | • H96773      | • Cap Nut (Apply Loctite #042-0024-02) . | 1    |
|      |               |                                                     |      | 12   | • 042-0007-07 | • Retaining Ring .....                   | 1    |

**Always Specify Model & Serial Number**

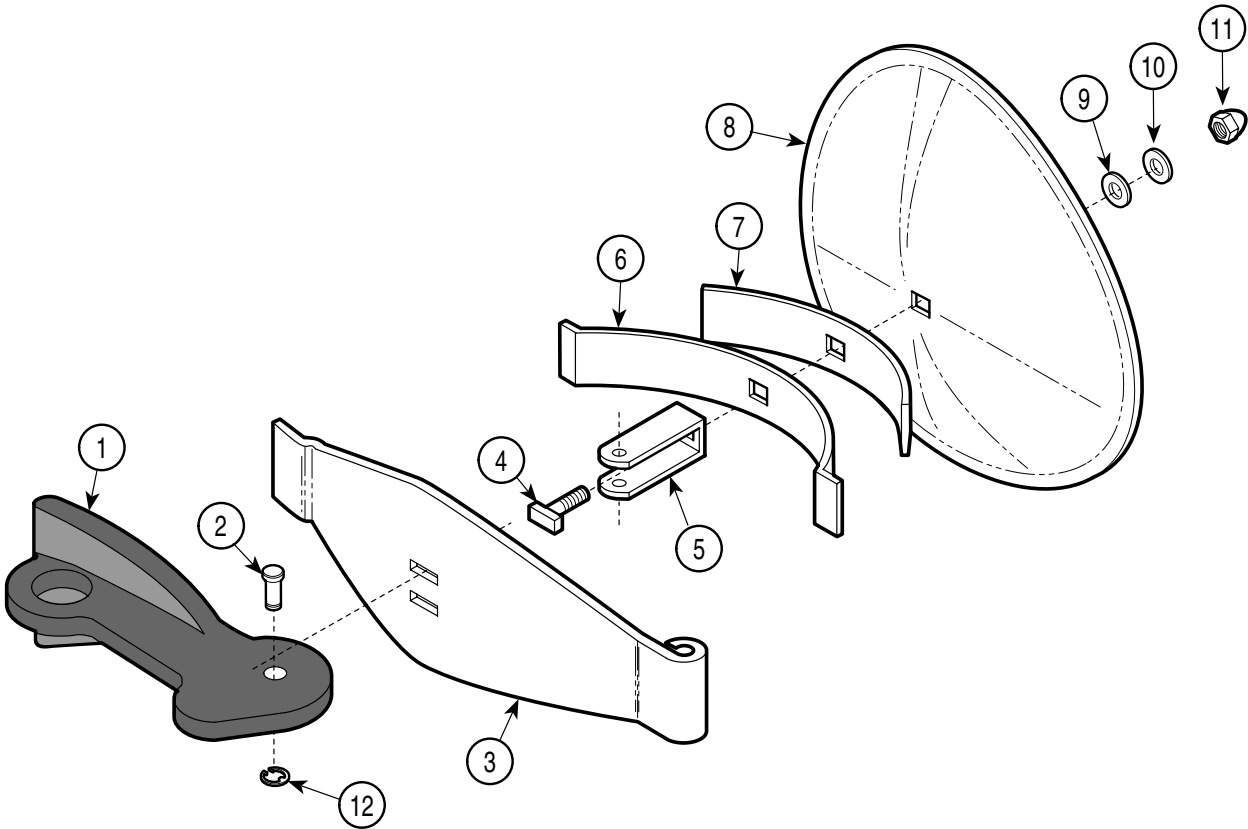


MA240700

**Used on Units with Serial Numbers:**  
**CP1072 thru CP1091    CR1226 thru CR1259    CS3104 thru CS3488**  
**RB10146 thru RB11004**

| Item | Part No.      | Description                                         | Qty. | Item | Part No.      | Description                              | Qty. |
|------|---------------|-----------------------------------------------------|------|------|---------------|------------------------------------------|------|
|      | 002-0249-00   | Door Assembly (Includes Items 1 thru 12) .....      | 1    | 6    | • H96775      | • Long Spring .....                      | 1    |
| 1    | • 053-0513-00 | • Handle (Apply Valve Lubricant #064-0002-01) ..... | 1    | 7    | • H96774      | • Short Spring .....                     | 1    |
| 2    | • H97989      | • Pin .....                                         | 1    | 8    | • H106313     | • Door .....                             | 1    |
| 3    | • H96751-1    | • Door Cross Arm .....                              | 1    | 9    | • 053-1579-00 | • Gasket .....                           | 1    |
| 4    | • H96772      | • "T" Bolt .....                                    | 1    | 10   | • H96771      | • Washer .....                           | 1    |
| 5    | • H96776      | • "U" Bracket .....                                 | 1    | 11   | • H96773      | • Cap Nut (Apply Loctite #042-0024-02) . | 1    |
|      |               |                                                     |      | 12   | • 042-0007-03 | • Retaining Ring .....                   | 1    |

**Always Specify Model & Serial Number**

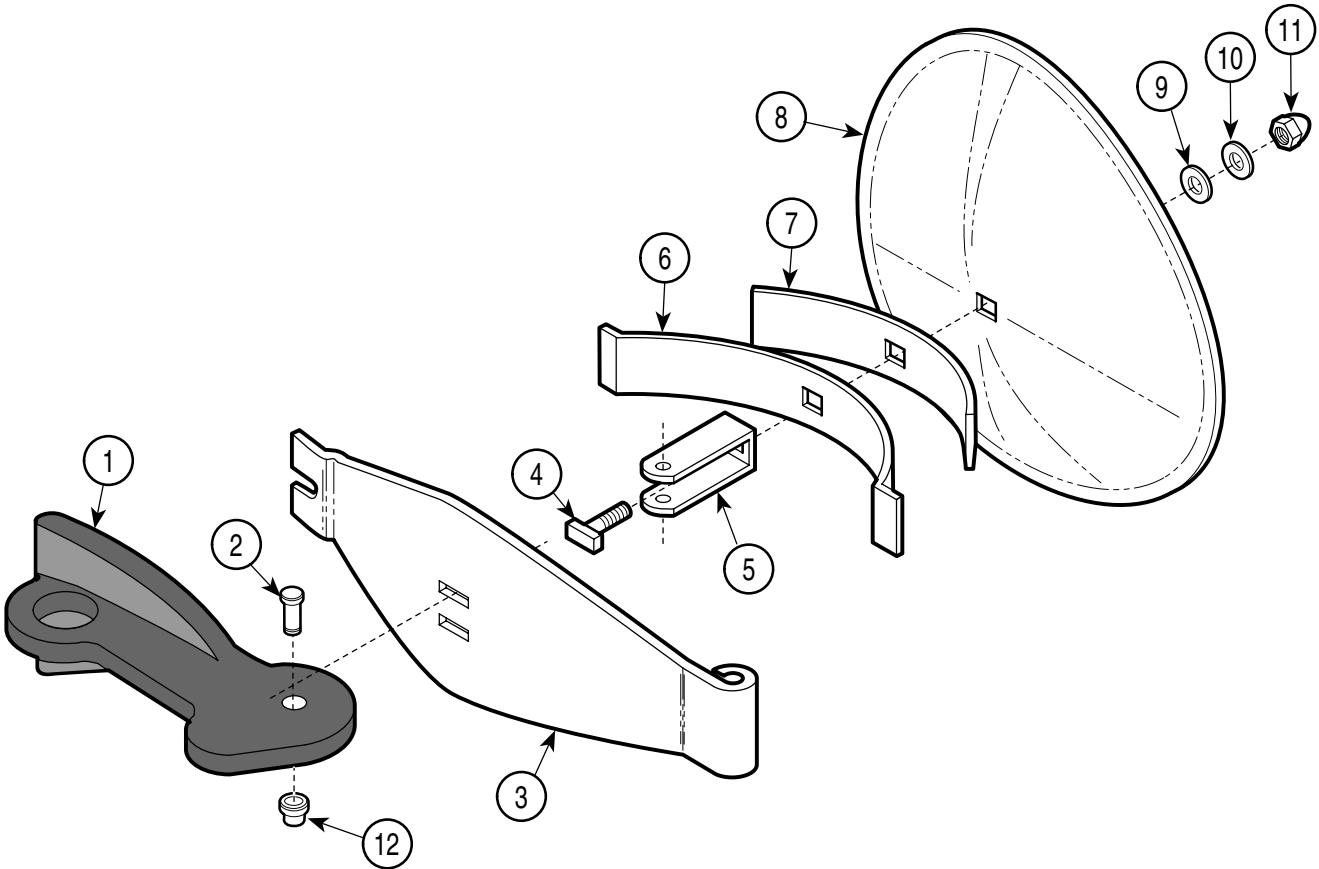


MA240703

**Used on Units with Serial Numbers RB11005, CP1092, DR1015, CR1260, CS3489, FM1000 and FN1000 thru Present**

| Item | Part No.      | Description                                   | Qty. | Item | Part No.      | Description                            | Qty. |
|------|---------------|-----------------------------------------------|------|------|---------------|----------------------------------------|------|
|      | 002-0249-00   | Door Assembly (Includes Items 1 thru 12)      | 1    | 6    | • H96775      | • Long Spring                          | 1    |
| 1    | • 053-0513-00 | • Handle (Apply Valve Lubricant #064-0002-01) | 1    | 7    | • H96774      | • Short Spring                         | 1    |
| 2    | • 042-0005-16 | • Clevis Pin                                  | 1    | 8    | • H106313     | • Door                                 | 1    |
| 3    | • 050-2359-00 | • Door Cross Arm                              | 1    | 9    | • 053-1579-00 | • Gasket                               | 1    |
| 4    | • H96772      | • "T" Bolt                                    | 1    | 10   | • H96771      | • Washer                               | 1    |
| 5    | • H96776      | • "U" Bracket                                 | 1    | 11   | • H96773      | • Cap Nut (Apply Loctite #042-0024-02) | 1    |
|      |               |                                               |      | 12   | • K906034     | • Push Nut                             | 1    |

Always Specify Model & Serial Number

**This Page Intentionally Left Blank**

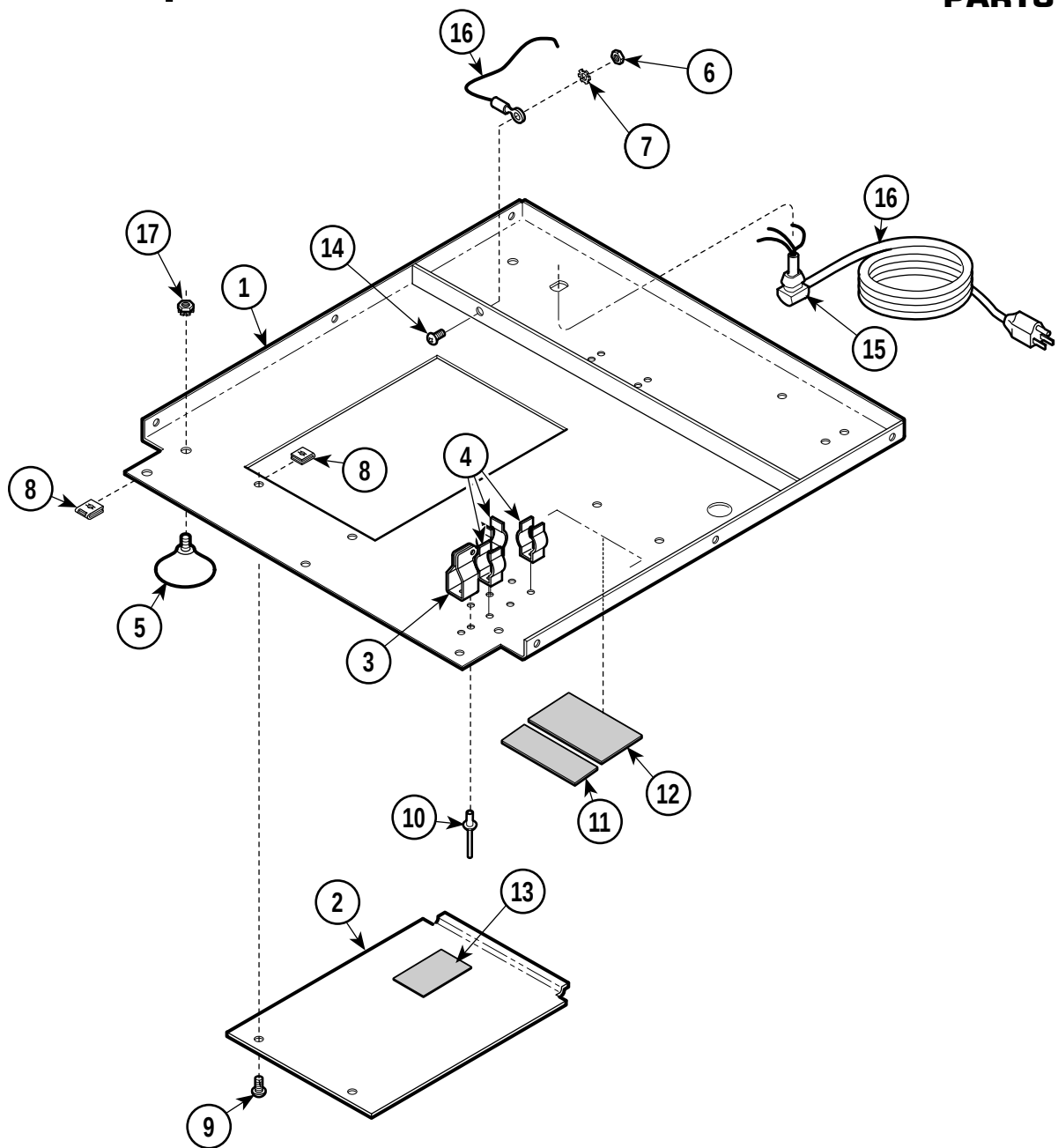

MA241201


**Used on Units with Serial Numbers:**  
**CP1000 thru CP1061    CR1000 thru CR1154    CS1000 thru CS1975**  
**RB1000 thru RB7986    DR1000 thru DR1014**

| Item | Part No.    | Description              | Qty. | Item | Part No.    | Description                                 | Qty. |
|------|-------------|--------------------------|------|------|-------------|---------------------------------------------|------|
| 1    | 050-2255-10 | Base .....               | 1    | 12   |             | Serial Number Tag .....                     | Ref  |
| 2    | H98006      | Inspection Cover .....   | 1    | 13   | 061-0394-00 | Warning Label .....                         | 1    |
| 3    | 050-1926-00 | Vent Lever Bracket ..... | 1    | 14   | 040-0010-74 | Screw .....                                 | 1    |
| 4    | H98072      | Clip .....               | 3    | 15   | 045-0001-08 | Lockwasher .....                            | 1    |
| 5    | H98021      | Foot .....               | 4    | 15   | H98160      | Strain Relief .....                         | 1    |
| 6    | 041-0010-01 | Nut .....                | 4    | 16   | H98159      | Cord (M7-001, M7-003, M7-004, M7-005) ..... | 1    |
| 7    | 045-0001-31 | Lockwasher .....         | 1    |      | H99025      | Cord (M7-002) .....                         | 1    |
| 8    | H352623     | Clip .....               | 11   | 17   | 122357      | Keps Nut .....                              | 4    |
| 9    | 040-0010-18 | Screw .....              | 2    |      |             |                                             |      |
| 10   | 042-0010-07 | Poprivet .....           | 5    |      |             |                                             |      |
| 11   | 061-0052-00 | U.L. Label .....         | Ref  |      |             |                                             |      |

Always Specify Model & Serial Number

# Base Components

## SECTION VI PARTS LIST

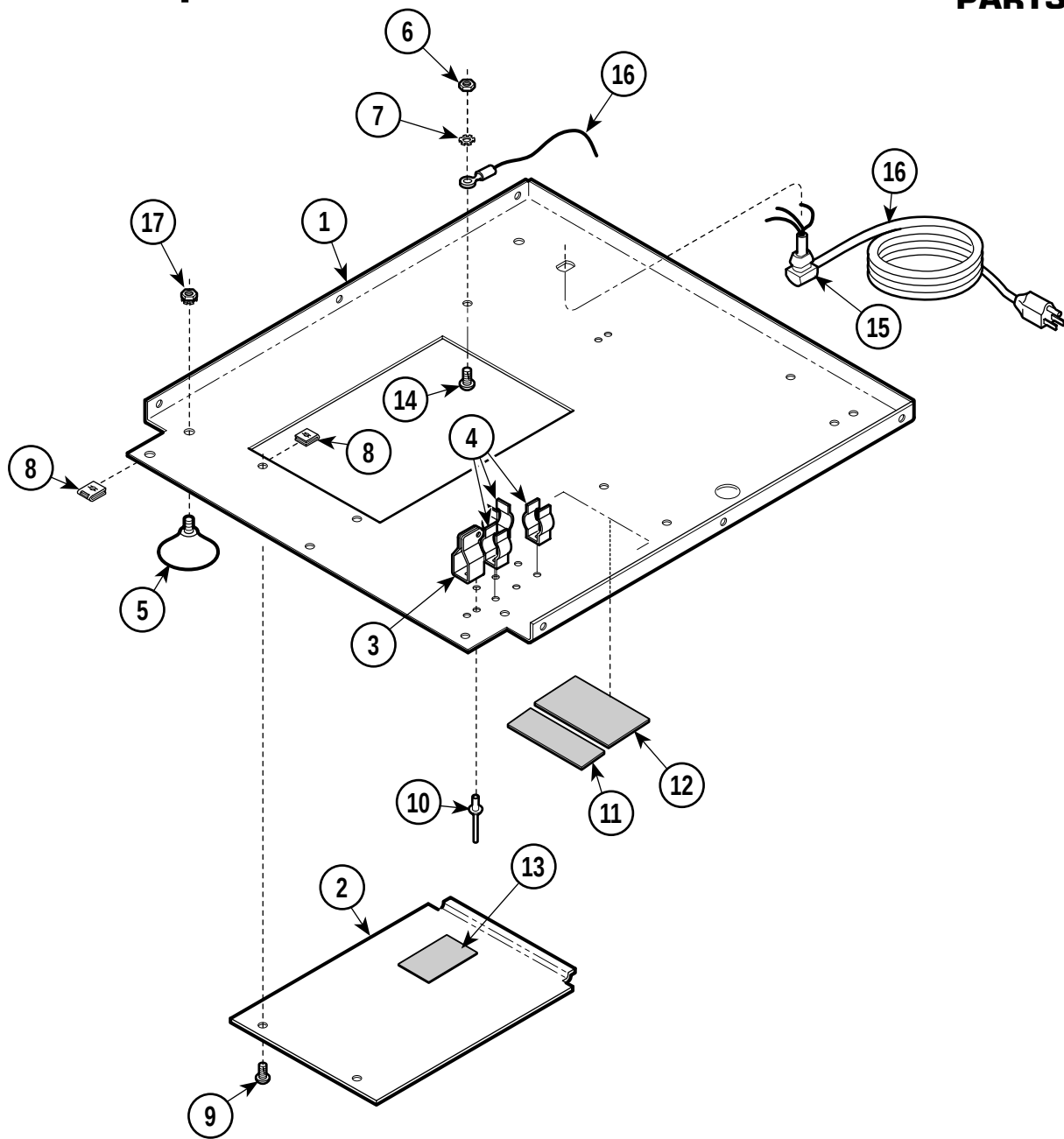

MA241206


**Used on Units with Serial Numbers:**  
**CP1062 thru CP1149   CR1155 thru CR1511   CS1976 thru CS7753**  
**RB7987 thru RB17992   DR1015 thru DR1267**

| Item | Part No.     | Description              | Qty. | Item | Part No.    | Description                                   | Qty. |
|------|--------------|--------------------------|------|------|-------------|-----------------------------------------------|------|
| 1    | 050-2383-10  | Base .....               | 1    | 11   | 061-0052-00 | U.L. Label .....                              | Ref  |
| 2    | H98006       | Inspection Cover .....   | 1    | 12   |             | Serial Number Tag .....                       | Ref  |
| 3    | 050-1926-00  | Vent Lever Bracket ..... | 1    | 13   | 061-0519-00 | Warning Label .....                           | 1    |
| 4    | H98072       | Clip .....               | 3    | 14   | 040-0010-74 | Screw .....                                   | 1    |
| 5    | H98021       | Foot .....               | 4    | 15   | H98160      | Strain Relief .....                           | 1    |
| 6    | 041-0010-01  | Nut .....                | 4    | 16   | H98159      | Cord (M7-00, M7-003, M7-004,<br>M7-005) ..... | 1    |
| 7    | 045-0001-08  | Lockwasher .....         | 15   |      | H99025      | Cord (M7-002) .....                           | 1    |
| 8    | H352623      | Clip .....               | 11   | 17   | 122357      | Keps Nut .....                                | 4    |
| 9    | 040-0010-125 | Screw .....              | 2    |      |             |                                               |      |
| 10   | 042-0010-07  | Poprivet .....           | 5    |      |             |                                               |      |

Always Specify Model & Serial Number

# Base Components

## SECTION VI PARTS LIST

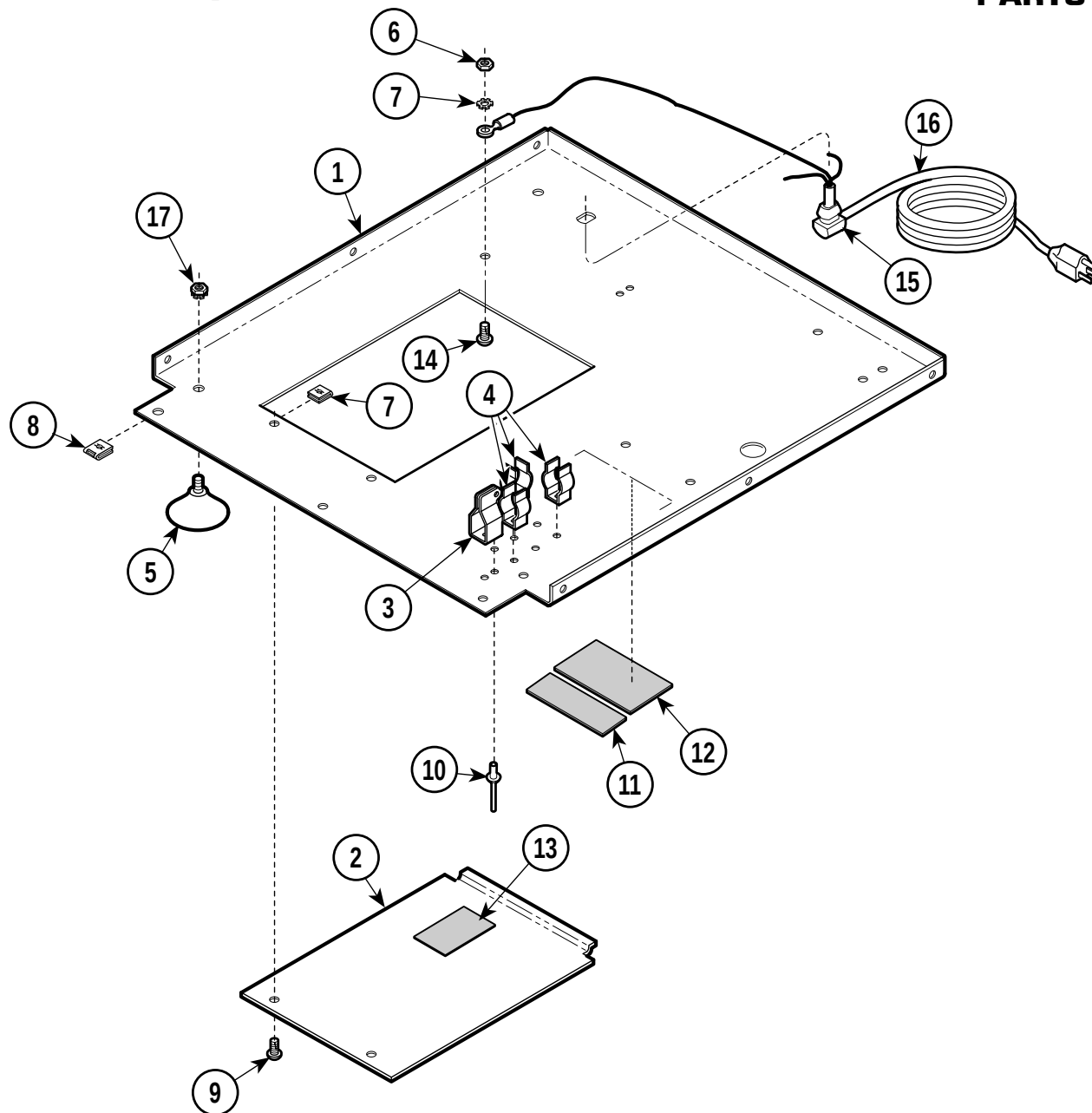

MA241205


### Used on Units with Serial Numbers CR1511 thru CR1685, FM1000 thru FM1425 and FN1000 thru FN1229

| Item | Part No.     | Description              | Qty. | Item | Part No.    | Description                                             | Qty. |
|------|--------------|--------------------------|------|------|-------------|---------------------------------------------------------|------|
| 1    | 050-2383-00  | Base .....               | 1    | 13   | 061-0519-00 | Wiring Diagram Label<br>(M7-001, M7-003 & M7-004) ..... | 1    |
| 2    | H98006       | Inspection Cover .....   | 1    |      | 061-0527-00 | Wiring Diagram Label (M7-002) .....                     | 1    |
| 3    | 050-1926-00  | Vent Lever Bracket ..... | 1    |      | 061-0601-00 | Wiring Diagram Label (M7-005) .....                     | 1    |
| 4    | H98072       | Clip .....               | 3    |      | 061-0607-00 | Wiring Diagram Label (M7-006) .....                     | 1    |
| 5    | H98021       | Foot .....               | 4    | 14   | 040-0010-74 | Screw .....                                             | 1    |
| 6    | 041-0010-01  | Nut .....                | 4    | 15   | H98160      | Strain Relief .....                                     | 1    |
| 7    | 045-0001-08  | Lockwasher .....         | 5    | 16   | H98159      | Cord (M7-001, M7-003, M7-004,<br>M7-005) .....          | 1    |
| 8    | H352623      | Clip .....               | 11   |      | 015-0688-13 | Cord (M7-002, M7-006) .....                             | 1    |
| 9    | 040-0010-125 | Screw .....              | 2    | 17   | 122357      | Keps Nut .....                                          | 4    |
| 10   | 042-0010-07  | Poprivet .....           | 5    |      |             |                                                         |      |
| 11   | 061-0052-00  | U.L. Label .....         | Ref  |      |             |                                                         |      |
| 12   |              | Serial Number Tag .....  | Ref  |      |             |                                                         |      |

Always Specify Model & Serial Number

# Base Components

## SECTION VI PARTS LIST

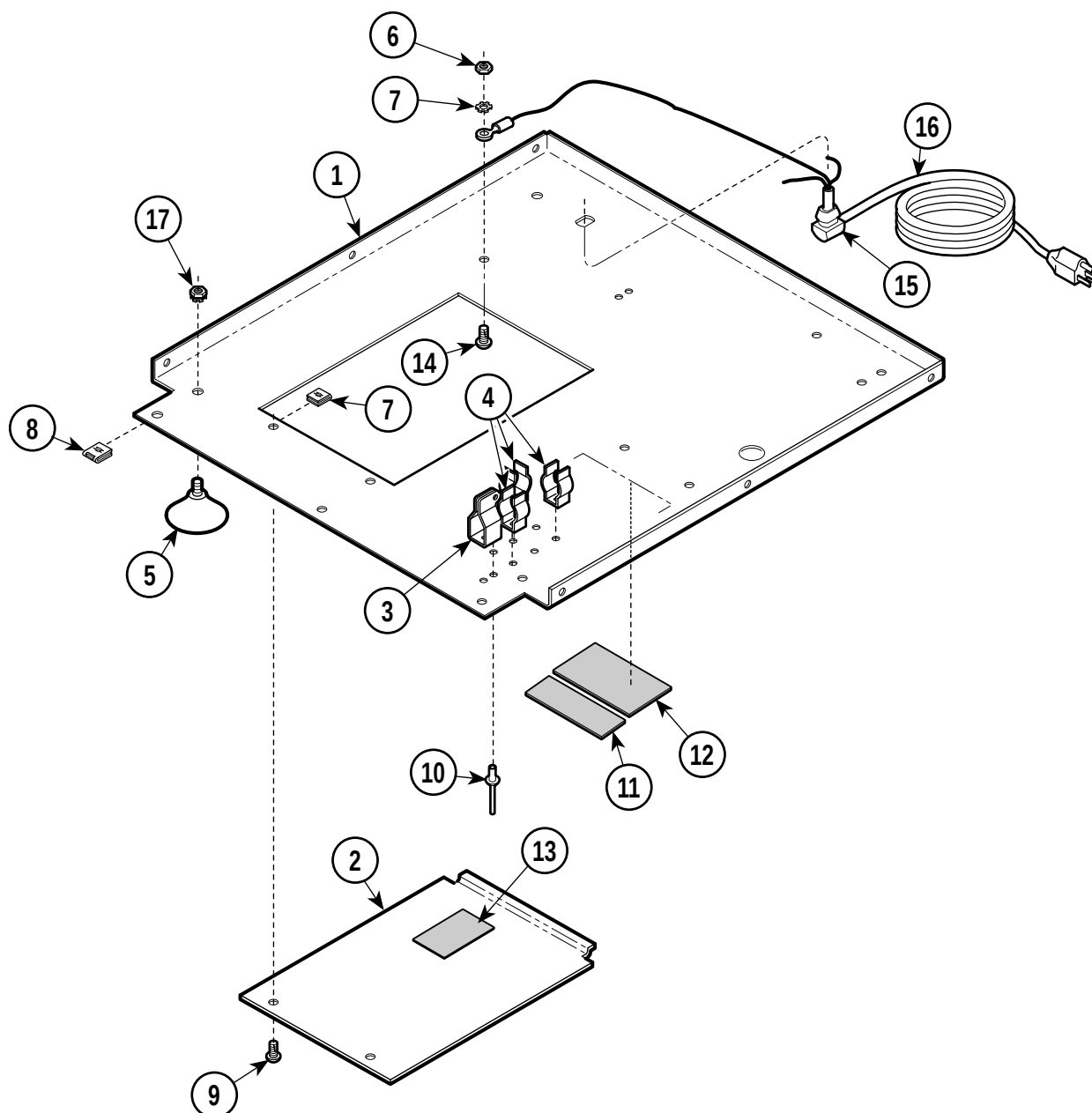

MA241205


**Used on Units with Serial Numbers:RB17993, CR1686, DR1268, CP1150, CS7754, FM1426 and FN11230 thru Present**

| Item | Part No.     | Description              | Qty. | Item | Part No.    | Description                                          | Qty. |
|------|--------------|--------------------------|------|------|-------------|------------------------------------------------------|------|
| 1    | 050-2383-00  | Base .....               | 1    | 13   | 061-0519-00 | Wiring Diagram Label (M7-001, M7-003 & M7-004) ..... | 1    |
| 2    | H98006       | Inspection Cover .....   | 1    |      | 061-0527-00 | Wiring Diagram Label (M7-002) .....                  | 1    |
| 3    | 050-1926-00  | Vent Lever Bracket ..... | 1    |      | 061-0601-00 | Wiring Diagram Label (M7-005) .....                  | 1    |
| 4    | H98072       | Clip .....               | 3    |      | 061-0607-00 | Wiring Diagram Label (M7-006) .....                  | 1    |
| 5    | H98021       | Foot .....               | 4    | 14   | 040-0010-74 | Screw .....                                          | 1    |
| 6    | 041-0010-01  | Nut .....                | 4    | 15   | H98160      | Strain Relief .....                                  | 1    |
| 7    | 045-0001-08  | Lockwasher .....         | 5    | 16   | H98159      | Cord (M7-001, M7-003, M7-004, M7-005) .....          | 1    |
| 8    | H352623      | Clip .....               | 11   |      | 015-0688-13 | Cord (M7-002, M7-006) .....                          | 1    |
| 9    | 040-0010-125 | Screw .....              | 2    | 17   | 122357      | Keps Nut .....                                       | 4    |
| 10   | 042-0010-07  | Poprivet.....            | 5    |      |             |                                                      |      |
| 11   | 061-0052-00  | U.L. Label .....         | Ref  |      |             |                                                      |      |
| 12   |              | Serial Number Tag .....  | Ref  |      |             |                                                      |      |

**Always Specify Model & Serial Number**

# Chamber Components - Side

## SECTION VI PARTS LIST

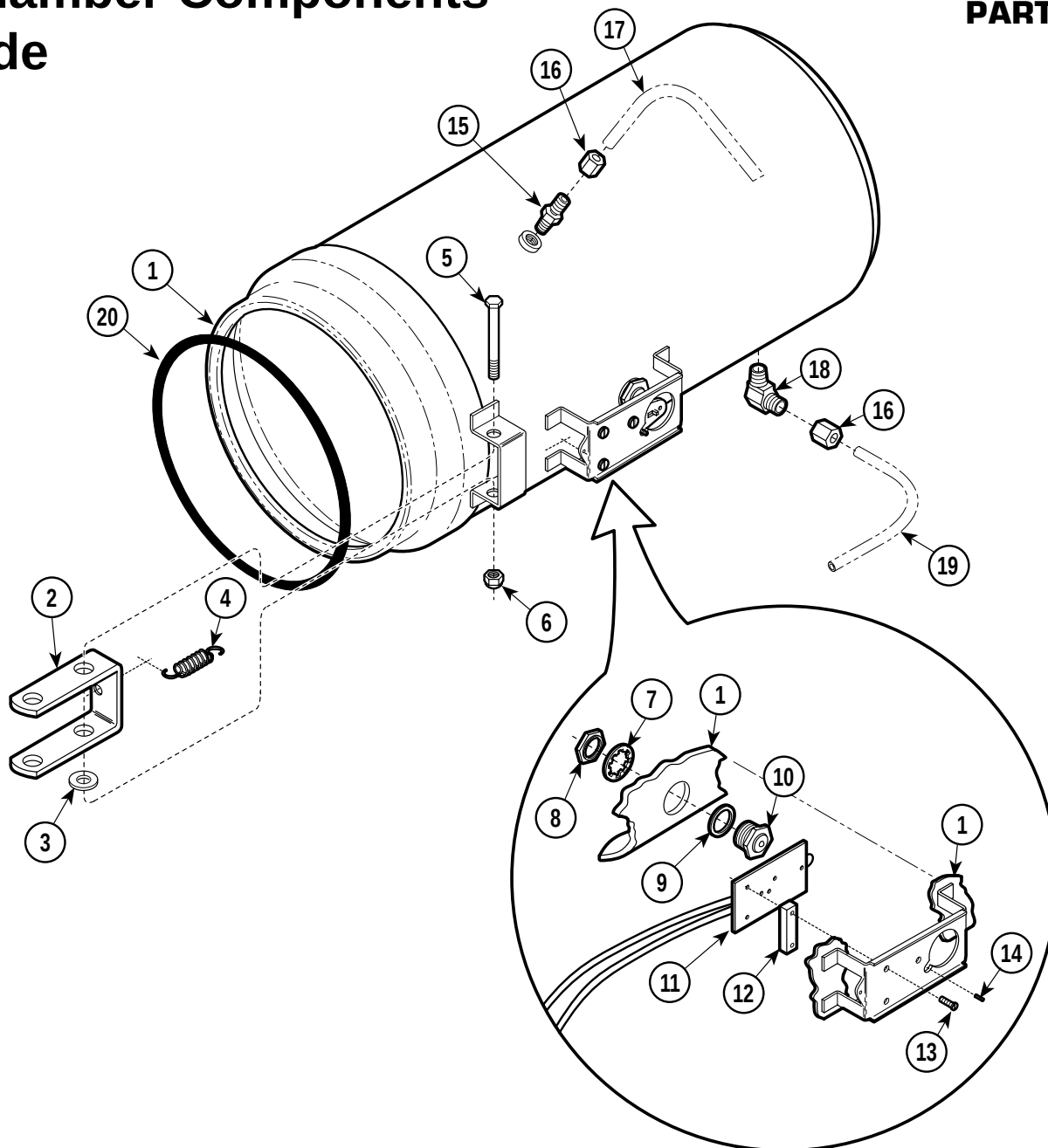

MA240903


### Used on Units with Serial Numbers RB1000, CP1000, CR1000 DR1000, CS1000, CS1000, FM1000 and FN1000 thru Present

| Item | Part No.    | Description                                          | Qty. | Item | Part No.    | Description                                         | Qty. |
|------|-------------|------------------------------------------------------|------|------|-------------|-----------------------------------------------------|------|
| 1    | 002-0516-00 | Shell (Includes Compression Fittings) ...            | 1    | 12   | H97951      | Relay Spacer (M7-001, M7-003, M7-004, M7-005) ..... | 1    |
| 2    | H96768      | Small Hinge .....                                    | 1    | 13   | 040-0006-44 | Screw .....                                         | 3    |
| 3    | 045-0001-57 | Washer .....                                         | 1    | 14   | 040-0008-47 | Set Screw .....                                     | 1    |
| 4    | H106330     | Spring .....                                         | 1    | 15   | 014-0183-01 | Fitting .....                                       | 1    |
| 5    | 040-0250-69 | Screw .....                                          | 1    | 16   | 014-0392-00 | Compression Nut .....                               | 1    |
| 6    | 041-0250-15 | Nut .....                                            | 1    | 17   |             | Tubing .....                                        | Ref  |
| 7    | H97076      | Lockwasher .....                                     | 1    | 18   | 014-0184-01 | Fitting .....                                       | 1    |
| 8    | H97075      | Nut .....                                            | 1    | 19   |             | Tubing .....                                        | Ref  |
| 9    | H97966      | Gasket .....                                         | 1    | 20   | 002-0243-00 | Door Gasket .....                                   | 1    |
| 10   | H97948      | Diaphragm Cup .....                                  | 1    |      |             |                                                     |      |
| 11   | 002-0263-00 | Relay (M7-001, M7-003, M7-004, M7-005) [Shown] ..... | 1    |      |             |                                                     |      |
|      | 002-0265-00 | Relay (M7-002 & M7-006) .....                        | 1    |      |             |                                                     |      |

Always Specify Model & Serial Number

# Chamber Components - Lower

## SECTION VI PARTS LIST

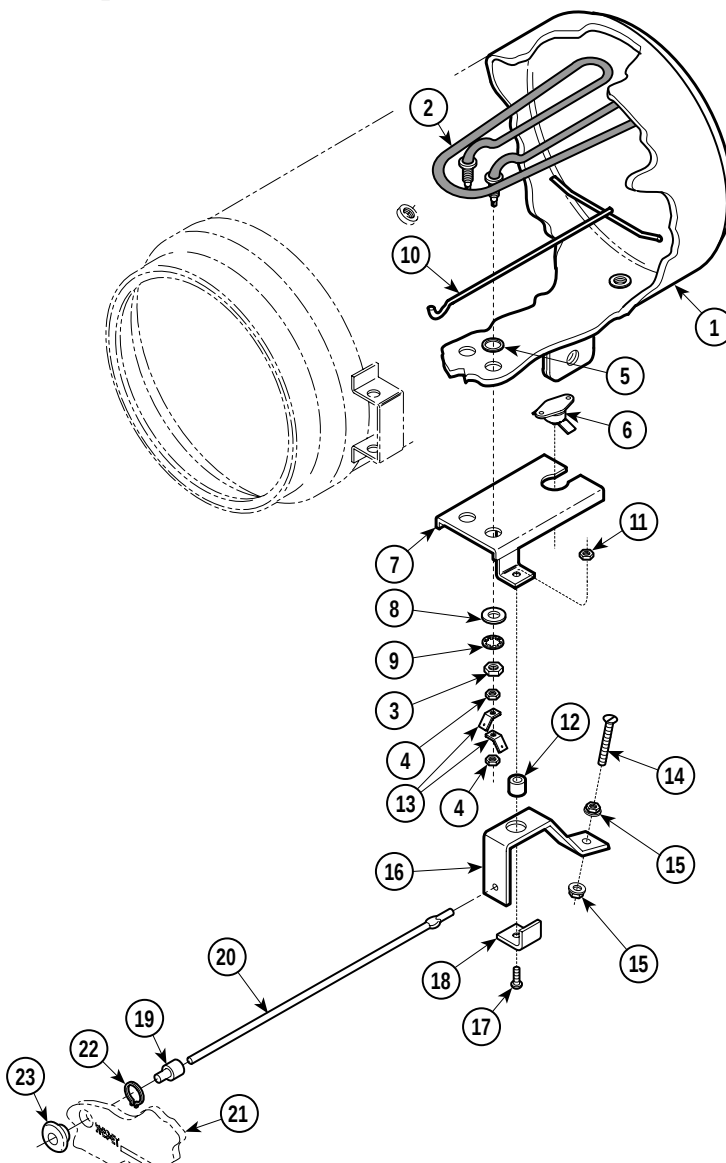

MA241001


**Used on Units with Serial Numbers:**  
**CP1000 thru CP1091    CR1000 thru CR1259    CS1000 thru CS3488**  
**RB1000 thru RB11004    DR1000 thru DR1014**

| Item | Part No.    | Description                                                    | Qty. | Item | Part No.    | Description                                                     | Qty. |
|------|-------------|----------------------------------------------------------------|------|------|-------------|-----------------------------------------------------------------|------|
| 1    |             | Chamber (Refer to "Chamber Components - Side" Elsewhere) ..... | Ref  | 13   | 015-0595-00 | Terminal .....                                                  | 4    |
| 2    | 002-0245-00 | Heater (M7-001, M7-004, M7-005) .....                          | 1    |      | H225404     | Rocker Assembly (Includes Item 12, Item 13 and Item 14) .....   | 1    |
|      | H284687-2   | Heater (M7-002) [Includes Items 3] ....                        | 1    | 14   | 040-0006-43 | Screw .....                                                     | 1    |
|      | H284687-3   | Heater (M7-003) [Includes Items 3] ....                        | 1    | 15   | 041-0006-02 | Locknut .....                                                   | 2    |
| 3    | H97949      | Nut .....                                                      | 2    | 16   | H98050      | Rocker Bracket .....                                            | 1    |
| 4    |             | Nut .....                                                      | 4    | 17   | 040-0010-74 | Screw .....                                                     | 1    |
| 5    | H98137      | Gasket .....                                                   | 2    | 18   | H225619     | Bracket .....                                                   | 1    |
| 6    | H225639     | Thermostat .....                                               | 1    |      | 002-0239-00 | Reset Button Kit .....                                          | 1    |
| 7    | H225616     | Bracket .....                                                  | 1    | 19   | H98040      | Reset Button .....                                              | 1    |
| 8    | H97971      | Washer .....                                                   | 1    | 20   | H98043      | Rod .....                                                       | 1    |
| 9    | 045-0001-56 | Lockwasher .....                                               | 2    | 21   |             | Front Panel (Refer to "Front Panel Components" Elsewhere) ..... | Ref  |
| 10   | H225913     | Spacer .....                                                   | 1    | 22   | 042-0065-01 | Retaining Ring .....                                            | 1    |
| 11   | 041-0010-10 | Nut .....                                                      | 1    | 23   | H98041      | Reset Button Guide .....                                        | 1    |
| 12   | H225620     | Spacer .....                                                   | 1    |      |             |                                                                 |      |

**Always Specify Model & Serial Number**

# Chamber Components - Lower

## SECTION VI PARTS LIST

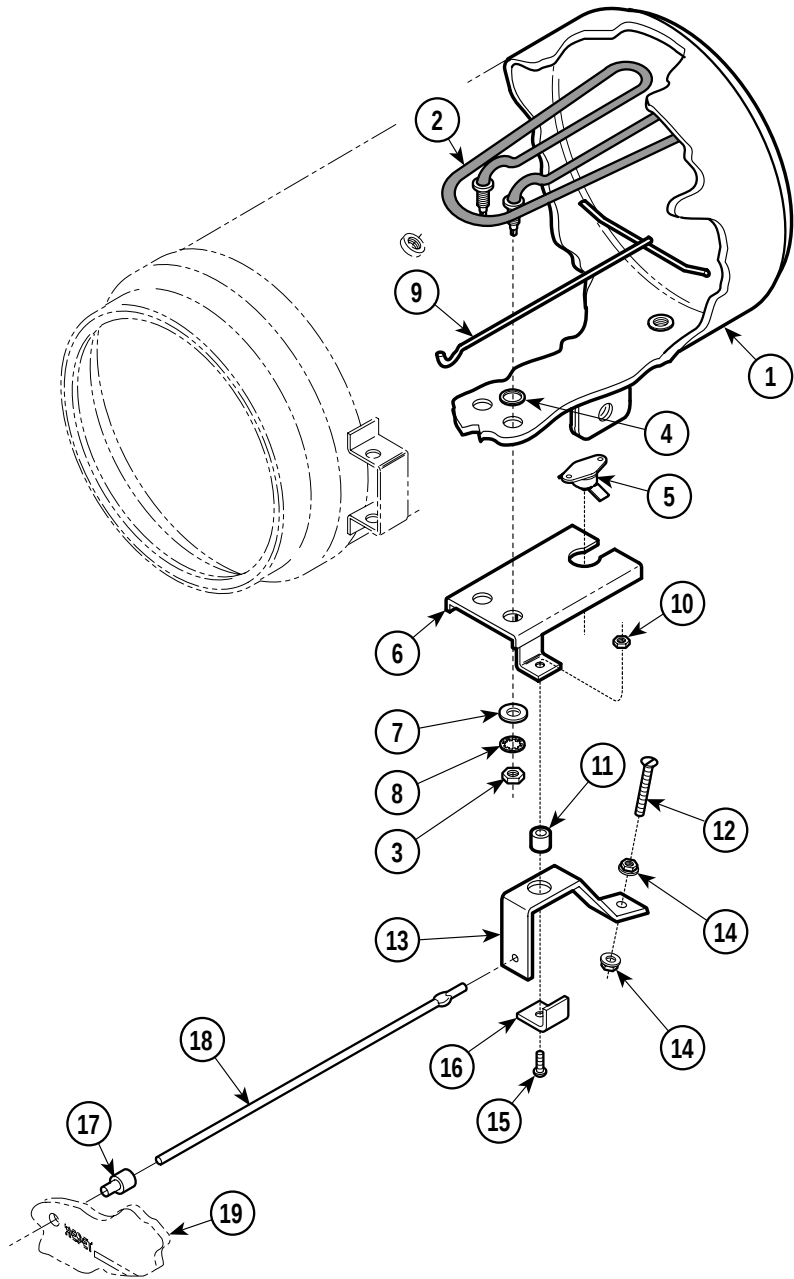

MA241000


### Used on Units with Serial Numbers: CS3489 thru CS5296 RB11005 thru RB14564

| Item | Part No.    | Description                                                    | Qty. | Item | Part No.    | Description                                                     | Qty. |
|------|-------------|----------------------------------------------------------------|------|------|-------------|-----------------------------------------------------------------|------|
| 1    |             | Chamber (Refer to "Chamber Components - Side" Elsewhere) ..... | Ref  | 10   | 041-0010-10 | Nut .....                                                       | 1    |
| 2    | 002-0245-00 | Heater (M7-001, M7-004, M7-005) [Includes Items 3 and 4] ..... | 1    | 11   | H225620     | Spacer .....                                                    | 1    |
|      | 015-0894-01 | Heater (M7-002) [Includes Items 3] .....                       | 1    |      | H225404     | Rocker Assembly (Includes Item 6, Item 15, Item 14) .....       | 1    |
|      | 015-0894-02 | Heater (M7-003) [Includes Items 3] .....                       | 1    | 12   | 040-0006-43 | Screw .....                                                     | 1    |
| 3    | H97949      | Nut .....                                                      | 2    | 13   | 050-2597-00 | Rocker Bracket .....                                            | 1    |
| 4    | H98137      | Gasket .....                                                   | 2    | 14   | 041-0006-02 | Locknut .....                                                   | 2    |
| 5    | H225639     | Thermostat .....                                               | 1    | 15   | 040-0010-74 | Screw .....                                                     | 1    |
| 6    | H225616     | Bracket .....                                                  | 1    | 16   | H225619     | Bracket .....                                                   | 1    |
| 7    | H97971      | Washer .....                                                   | 1    | 17   | 053-0529-00 | Reset Button .....                                              | 1    |
| 8    | 045-0001-56 | Lockwasher .....                                               | 2    | 18   | 057-0372-00 | Rod .....                                                       | 1    |
| 9    | H225913     | Spacer .....                                                   | 1    | 19   |             | Front Panel (Refer to "Front Panel Components" Elsewhere) ..... | Ref  |

Always Specify Model & Serial Number

# Chamber Components - Lower

## SECTION VI PARTS LIST

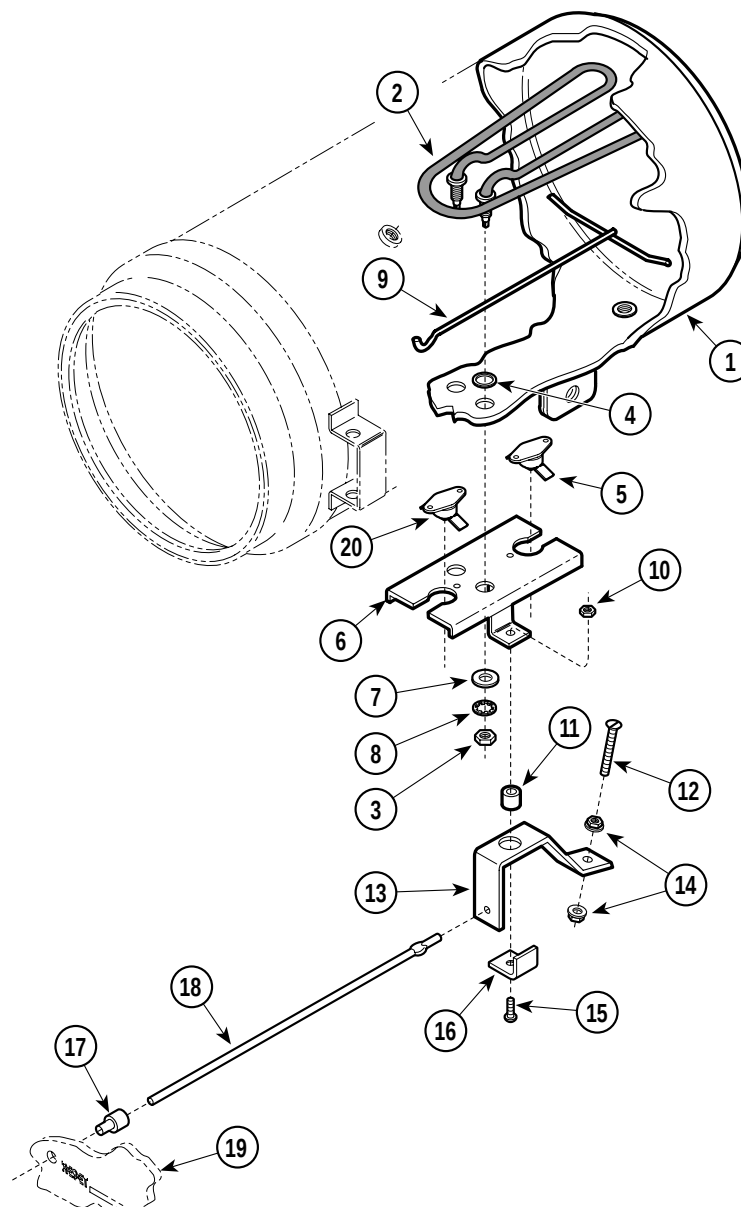

MA241002


**Used on Units with Serial Numbers RB14565, CP1092, DR1015  
CR1260, CS5297, FM1000 and FN1000 thru Present**

| Item | Part No.    | Description                                                    | Qty. | Item | Part No.    | Description                                                     | Qty. |
|------|-------------|----------------------------------------------------------------|------|------|-------------|-----------------------------------------------------------------|------|
| 1    |             | Chamber (Refer to "Chamber Components - Side" Elsewhere) ..... | Ref  | 10   | 041-0010-10 | Nut .....                                                       | 1    |
| 2    | 002-0245-00 | Heater (M7-001, M7-004, M7-005) .....                          | 1    | 11   | H225620     | Spacer .....                                                    | 1    |
|      | 015-0894-01 | Heater (M7-002, M7-006 [Includes Items 3]) .....               | 1    |      | H225404     | Rocker Assembly (Includes Item 6, Item 15, Item 14) .....       | 1    |
|      | 015-0894-02 | Heater (M7-003) [Includes Items 3] .....                       | 1    | 12   | 040-0006-43 | Screw .....                                                     | 1    |
| 3    | H97949      | Nut .....                                                      | 2    | 13   | 050-2597-00 | Rocker Bracket .....                                            | 1    |
| 4    | H98137      | Gasket .....                                                   | 2    | 14   | 041-0006-02 | Locknut .....                                                   | 2    |
| 5    | H225639     | Thermostat (Primary) .....                                     | 2    | 15   | 040-0010-74 | Screw .....                                                     | 1    |
| 6    | 030-0929-00 | Bracket .....                                                  | 1    | 16   | H225619     | Bracket .....                                                   | 1    |
| 7    | H97971      | Washer .....                                                   | 1    | 17   | 053-0529-00 | Reset Button .....                                              | 1    |
| 8    | 045-0001-56 | Lockwasher .....                                               | 2    | 18   | 057-0372-00 | Rod .....                                                       | 1    |
| 9    | H225913     | Spacer .....                                                   | 1    | 19   |             | Front Panel (Refer to "Front Panel Components" Elsewhere) ..... | Ref  |
|      |             |                                                                |      | 20   | 015-0647-00 | Thermostat (Secondary)(M7-001 & 004) .....                      | 1    |

**Always Specify Model & Serial Number**

# Tank Assembly

## SECTION VI PARTS LIST

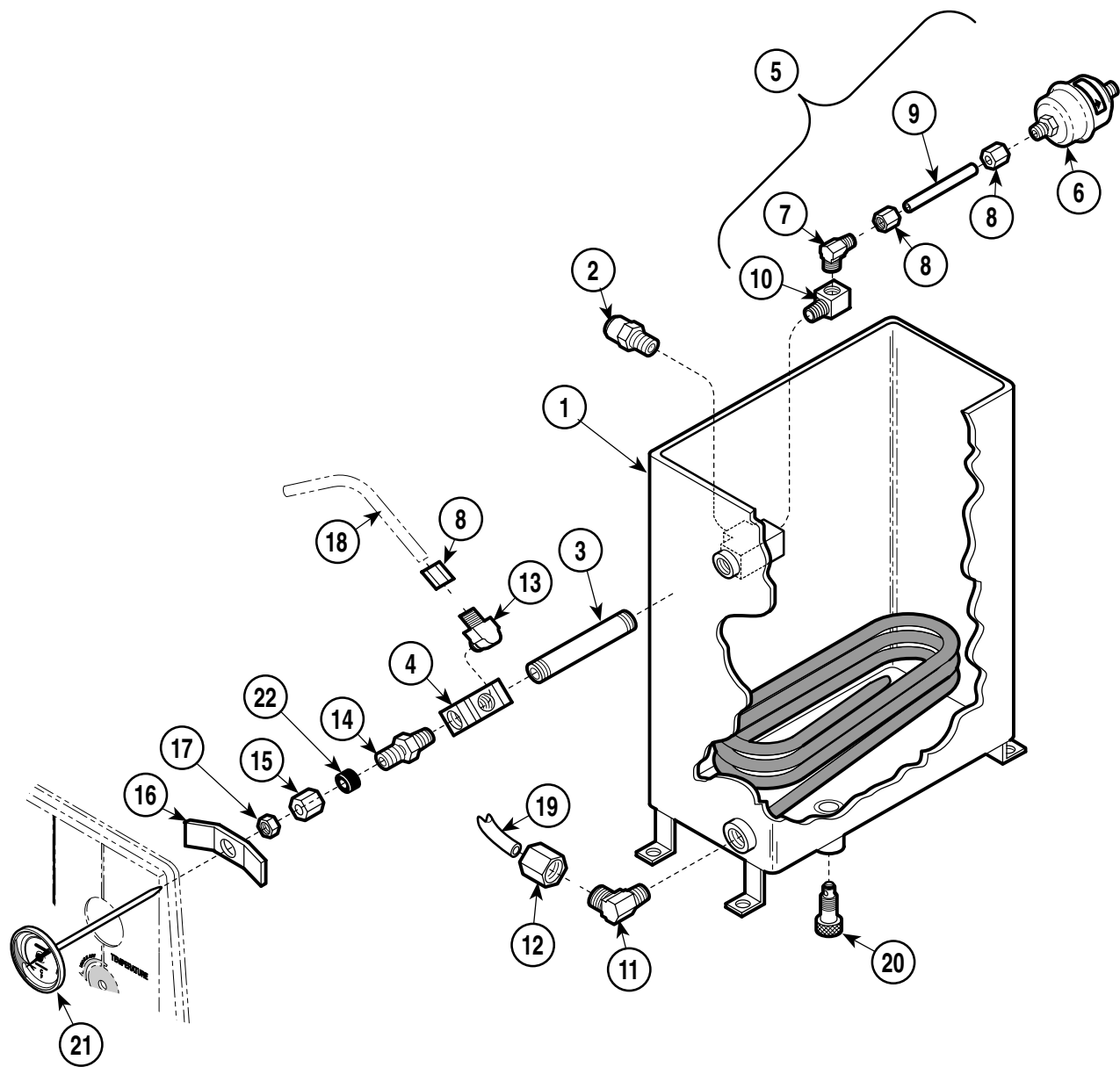

MA240804


**Used on Units with Serial Numbers:**  
**CP1000 thru CP1061    CR1000 thru CR1113    CS1000 thru CS1569**  
**RB1000 thru RB7193**

| Item | Part No.        | Description                            | Qty. | Item | Part No.      | Description                                          | Qty. |
|------|-----------------|----------------------------------------|------|------|---------------|------------------------------------------------------|------|
|      | H335950         | Tank Assembly (Incls. Items 1 thru 19) | 1    | 12   | • 014-0392-01 | • Compression Nut                                    | 1    |
| 1    | • H335946       | • Tank                                 | 1    | 13   | • 014-0184-00 | • Male Elbow                                         | 1    |
| 2    | • H98089        | • Safety Valve                         | 1    | 14   | • 014-0237-00 | • Connector Drilled                                  | 1    |
|      |                 | (Apply Teflon Tape #014-0026-00)       | 1    | 15   | • •           | • Compression Nut                                    | 1    |
| 3    | • 014-0182-01   | • Pipe Nipple                          | 1    | 16   | H224033       | Bracket                                              | 1    |
| 4    | • H335945-1     | • Pipe "T"                             | 1    | 17   | 041-0250-07   | Thermometer Nut                                      | 1    |
| 5    | • 002-0272-00   | • Bellows Assembly                     | 1    | 18   | H335943       | Tube                                                 | Ref  |
| 6    | • • 002-0375-00 | • • Bellows                            | 1    | 19   |               | Tube (Refer to "Sterilizer Main Components" Item 23) | 6-5  |
| 7    | • • 014-0184-00 | • • Elbow                              | 1    | 20   | 002-0262-00   | Drain (Includes O-Ring)                              | 1    |
| 8    | • • 014-0392-00 | • • Compression Nut                    | 1    |      | • H98138      | • O-ring                                             | 1    |
| 9    | • • 014-0190-00 | • • Tube                               | 1    | 21   | 002-0242-00   | Thermometer (Includes Item 22)                       | 1    |
| 10   | • • 014-0194-00 | • • Straight "T"                       | 1    | 22   | • H225730     | • Thermometer Sleeve                                 | 1    |
| 11   | • 014-0184-01   | • Male Elbow                           | 1    |      |               |                                                      |      |

Always Specify Model & Serial Number

# Tank Assembly

## SECTION VI PARTS LIST

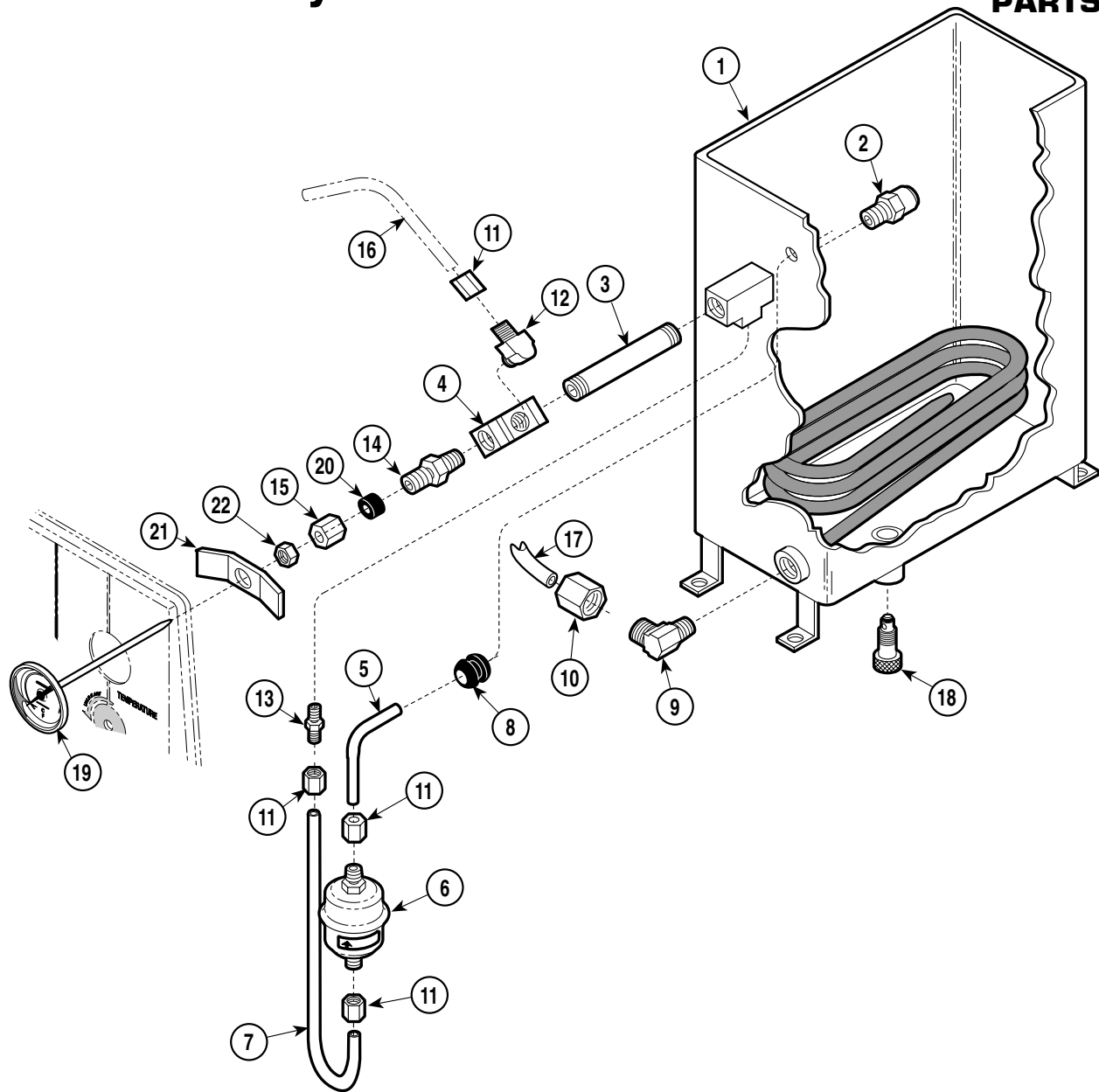


MA240805

**Used on Units with Serial Numbers:**  
**CR1114 thru CR1157 CS1570 thru CS1975**  
**RB7194 thru RB7986**

| Item | Part No.        | Description                              | Qty. | Item | Part No.      | Description                          | Qty.  |
|------|-----------------|------------------------------------------|------|------|---------------|--------------------------------------|-------|
| 1    | 029-1462-00     | Tank Assembly(Incls. Items 1 thru 21) .. | 1    | 12   | • 014-0184-00 | • Male Elbow .....                   | 1     |
| 2    | • 030-0799-00   | • Tank .....                             | 1    | 13   | • 014-0183-01 | • Male Connector .....               | 1     |
| 3    | • H98089        | • Safety Valve (Apply Teflon Tape        |      | 14   | • 014-0237-00 | • Connector Drilled .....            | 1     |
| 4    |                 | #014-0026-00) .....                      | 1    | 15   | • •           | • • Compression Nut .....            | 1     |
| 5    | • 014-0182-01   | • Pipe Nipple .....                      | 1    | 16   | H335943       | Tube .....                           | Ref   |
| 6    | • H335945-1     | • Pipe "T" .....                         | 1    | 17   |               | Tube (Refer to "Sterilizer Main      |       |
| 7    | • 014-0231-00   | • Tube .....                             | 1    |      |               | Components" Item 23) .....           | 6-5.1 |
| 8    | • 002-0375-00   | • Bellows .....                          | 1    | 18   | 002-0262-00   | Drain (Includes O-Ring) .....        | 1     |
| 9    | • 014-0232-00   | • Tube .....                             | 1    |      | • H98138      | • O-ring .....                       | 1     |
| 10   | • 053-0394-00   | • Grommet .....                          | 1    | 19   | 002-0242-00   | Thermometer (Includes Item 22) ..... | 1     |
| 11   | • 014-0184-01   | • Male Elbow .....                       | 1    | 20   | • H225730     | • Thermometer Sleeve .....           | 1     |
|      | • • 014-0392-01 | • • Compression Nut .....                | 1    | 21   | H224033       | Bracket .....                        | 1     |
|      | • 014-0392-00   | • Compression Nut .....                  | 4    | 22   | 041-0250-07   | Thermometer Nut .....                | 1     |

Always Specify Model & Serial Number

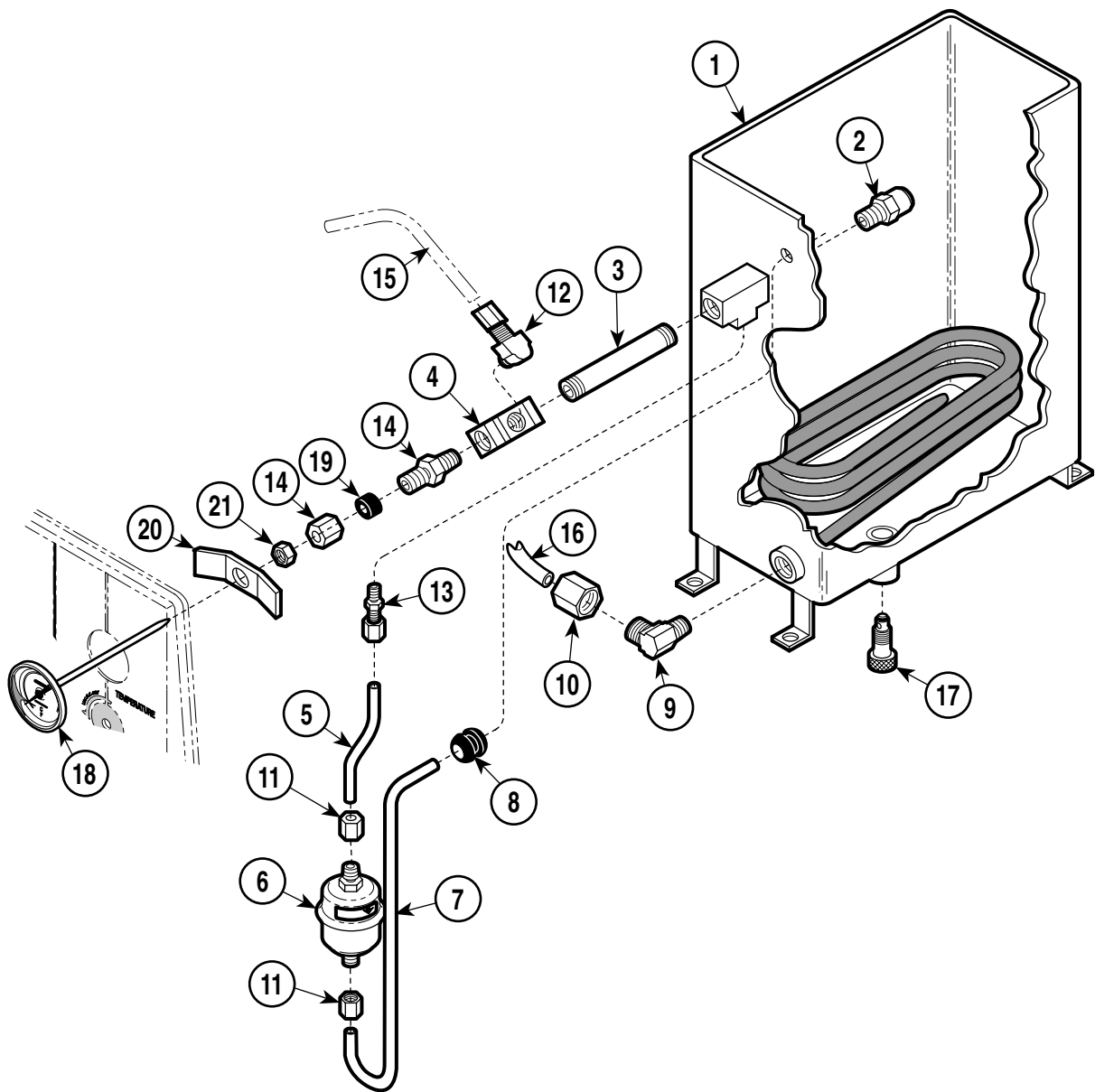

MA240808


**Used on Units with Serial Numbers RB7987 thru RB17992, CR1158 thru CR1685, DR1000 thru DR1267, CP1062 thru CP1149, CS1976 thru CS7753 FM1000 thru FM1425 and FN1000 thru FN1229**

| Item | Part No.      | Description                                     | Qty. | Item | Part No.      | Description                                            | Qty.  |
|------|---------------|-------------------------------------------------|------|------|---------------|--------------------------------------------------------|-------|
|      | 029-1462-00   | Tank Assembly (Incls. Items 1 thru 21)          | 1    | 12   | • 014-0184-00 | • Male Connector                                       | 1     |
| 1    | • 030-0799-00 | • Tank                                          | 1    | 13   | • 014-0183-01 | • Connector Drilled                                    | 1     |
| 2    | • H98089      | • Safety Valve (Apply Teflon Tape #014-0026-00) | 1    | 14   | • 014-0237-00 | • Compression Nut                                      | 1     |
| 3    | • 014-0182-01 | • Pipe Nipple                                   | 1    | 15   |               | Tube (Refer to "Sterilizer Main Components" Elsewhere) | Ref   |
| 4    | • H335945-1   | • Pipe "T"                                      | 1    | 16   |               | Tube (Refer to "Sterilizer Main Components" Item 23)   | 6-5.1 |
| 5    | • 014-0241-00 | • Tube                                          | 1    | 17   | 002-0262-00   | Drain (Includes O-ring)                                | 1     |
| 6    | • 002-0375-00 | • Bellows                                       | 1    |      | • H98138      | • O-ring                                               | 1     |
| 7    | • 014-0240-00 | • Tube                                          | 1    | 18   | 002-0242-00   | Thermometer (Includes Item 26)                         | 1     |
| 8    | • 053-0394-00 | • Grommet                                       | 1    | 19   | • H225730     | • Thermometer Sleeve                                   | 1     |
| 9    | • 014-0184-01 | • Male Elbow                                    | 1    | 20   | H224033       | Bracket                                                | 1     |
| 10   | • 014-0392-01 | • Compression Nut                               | 1    | 21   | 041-0250-07   | Thermometer Nut                                        | 1     |
| 11   | • 014-0392-00 | • Compression Nut                               | 4    |      |               |                                                        |       |

Always Specify Model & Serial Number

# Tank Assembly

## SECTION VI PARTS LIST

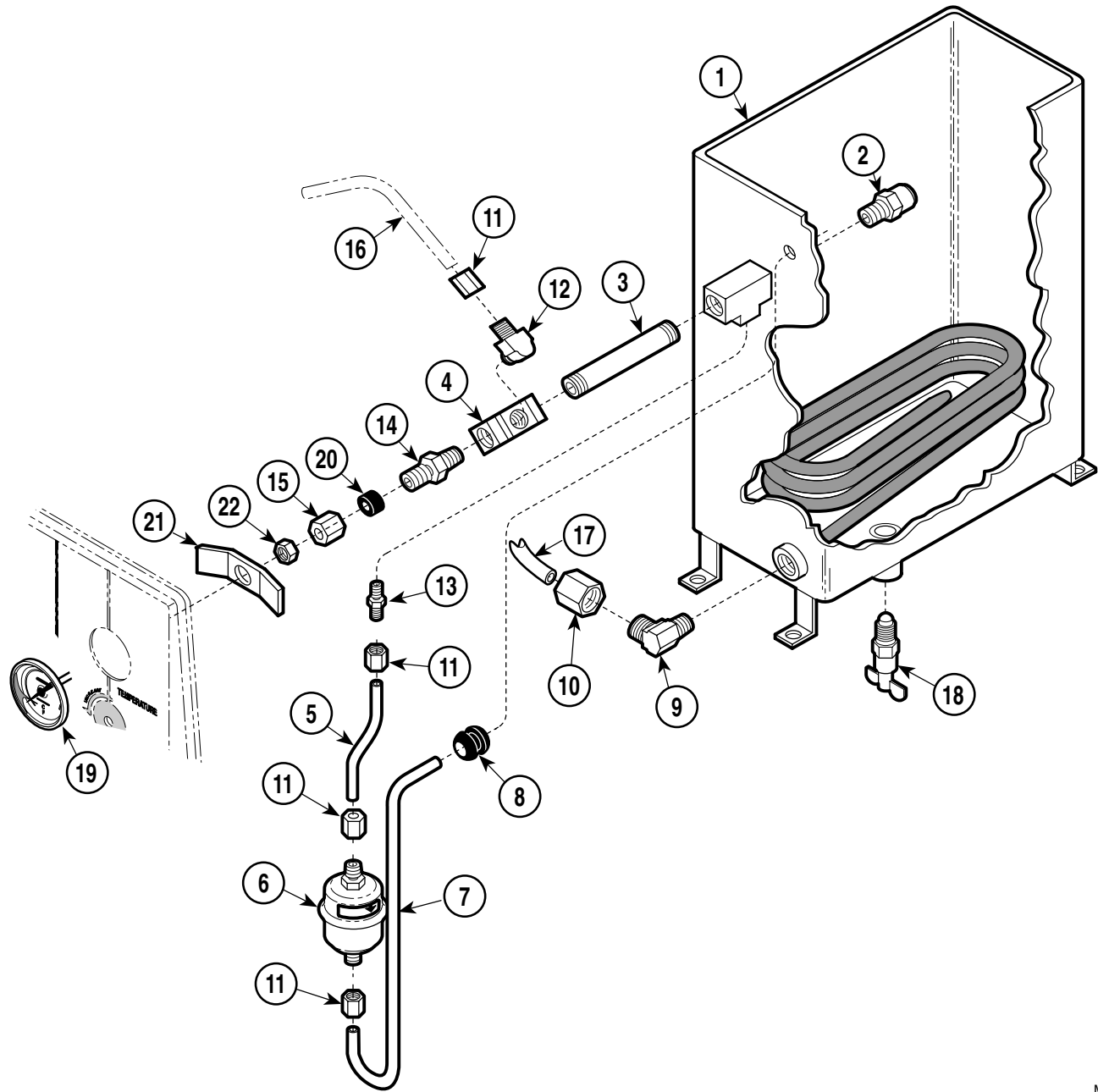


MA240807

### Used on Units with Serial Numbers RB17993, CR1686, DR1268, CP1150, CS7754, FM1426 and FN1230 thru Present

| Item | Part No.        | Description                                     | Qty. | Item | Part No.      | Description                                            | Qty.  |
|------|-----------------|-------------------------------------------------|------|------|---------------|--------------------------------------------------------|-------|
| 1    | 029-2085-00     | Tank Assembly (Incls. Items 1 thru 21)          | 1    | 12   | • 014-0184-00 | • Male Elbow                                           | 1     |
| 2    | • 030-0994-00   | • Tank                                          | 1    | 13   | • 014-0183-01 | • Male Connector                                       | 1     |
| 3    | • H98089        | • Safety Valve (Apply Teflon Tape #014-0026-00) | 1    | 14   | • 014-0237-00 | • Connector Drilled                                    | 1     |
| 4    | • 014-0182-01   | • Pipe Nipple                                   | 1    | 15   | • •           | • • Compression Nut                                    | 1     |
| 5    | • H335945-1     | • Pipe "T"                                      | 1    | 16   |               | Tube (Refer to "Sterilizer Main Components" Elsewhere) | Ref   |
| 6    | • 014-0241-00   | • Tube                                          | 1    | 17   |               | Tube (Refer to "Sterilizer Main Components" Item 23)   | 6-5.3 |
| 7    | • 002-0375-00   | • Bellows                                       | 1    | 18   | 002-0537-00   | Drain-Cock Replacement Kit                             | 1     |
| 8    | • 014-0240-00   | • Tube                                          | 1    | 19   | 002-0242-00   | Thermometer (Includes Item 26)                         | 1     |
| 9    | • 053-0394-01   | • Grommet                                       | 1    | 20   | • H225730     | • Thermometer Sleeve                                   | 1     |
| 10   | • 014-0184-01   | • Male Elbow                                    | 1    | 21   | H224033       | Bracket                                                | 1     |
| 11   | • • 014-0392-01 | • • Compression Nut                             | 1    | 22   | 041-0250-07   | Thermometer Nut                                        | 1     |
|      | • 014-0392-00   | • Compression Nut                               | 4    |      |               |                                                        |       |

Always Specify Model & Serial Number

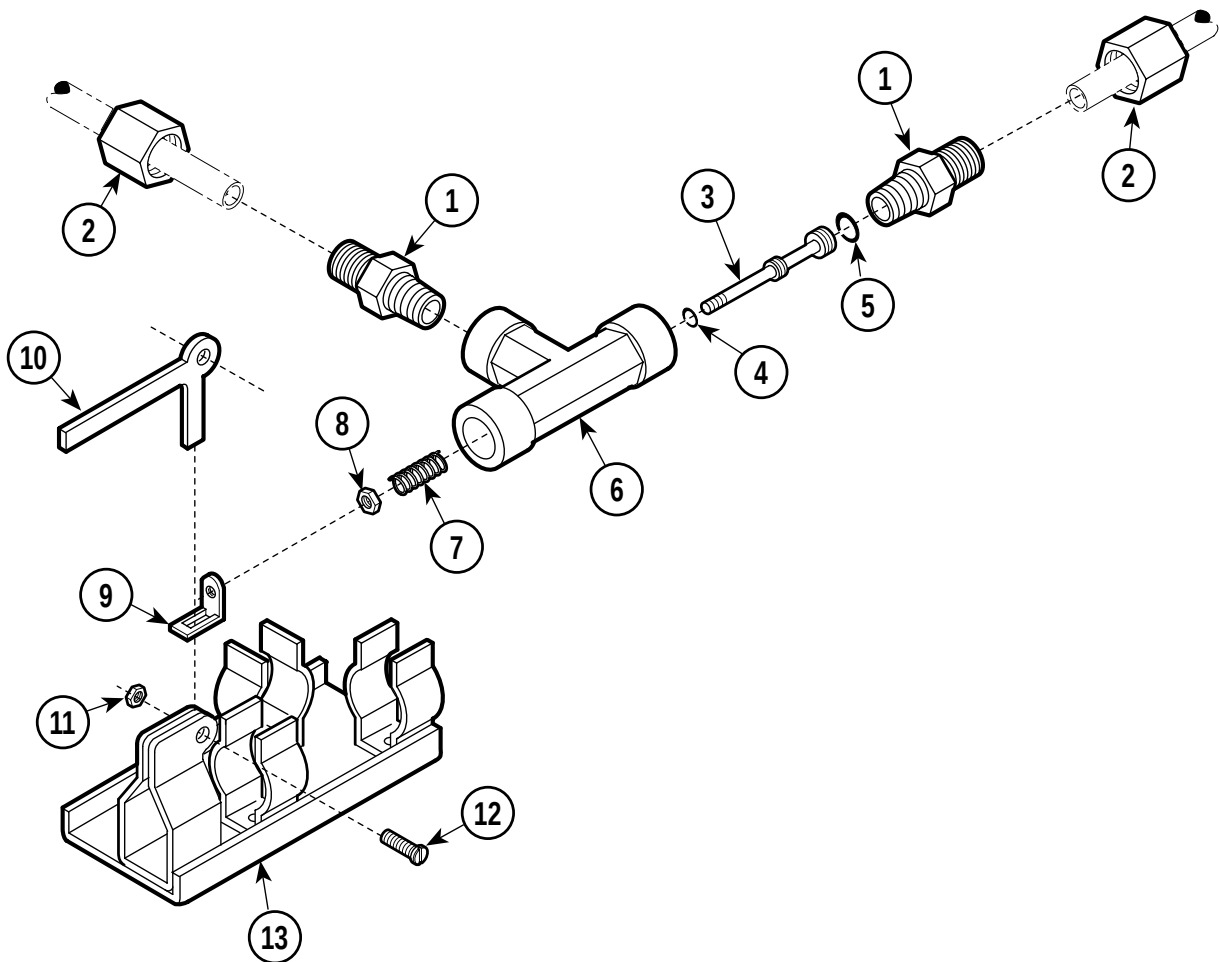


MA240602

## Used On Units Built Prior To 6-21-90

| Item | Part No.      | Description                                         | Qty. | Item | Part No.      | Description             | Qty. |
|------|---------------|-----------------------------------------------------|------|------|---------------|-------------------------|------|
|      | H98080        | Valve & Bracket Assembly (Includes Items 1 thru 14) | 1    | 6    | • H98068      | • Valve Body .....      | 1    |
| 1    | • 014-0183-02 | • Fitting .....                                     | 2    | 7    | • H98070      | • Valve Spring .....    | 1    |
| 2    | • 014-0392-01 | • Compression Nut .....                             | 1    | 8    | • 041-0010-09 | • Nut .....             | 1    |
| 3    | • H98069      | • Plunger .....                                     | 1    | 9    | • H98077      | • Closing Bracket ..... | 1    |
| 4    | • H98138      | • O-Ring (Apply High Temp Lube #064-0002-01) .....  | 1    | 10   | • H98074      | • Lever .....           | 1    |
| 5    | • H98136      | • O-Ring (Apply High Temp Lube #064-0002-01) .....  | 1    | 11   | • 041-0010-08 | • Nut .....             | 1    |
|      |               |                                                     |      | 12   | • 040-0010-73 | • Screw .....           | 1    |
|      |               |                                                     |      | 13   | • H98079      | • Valve Bracket .....   | 1    |


Always Specify Model & Serial Number

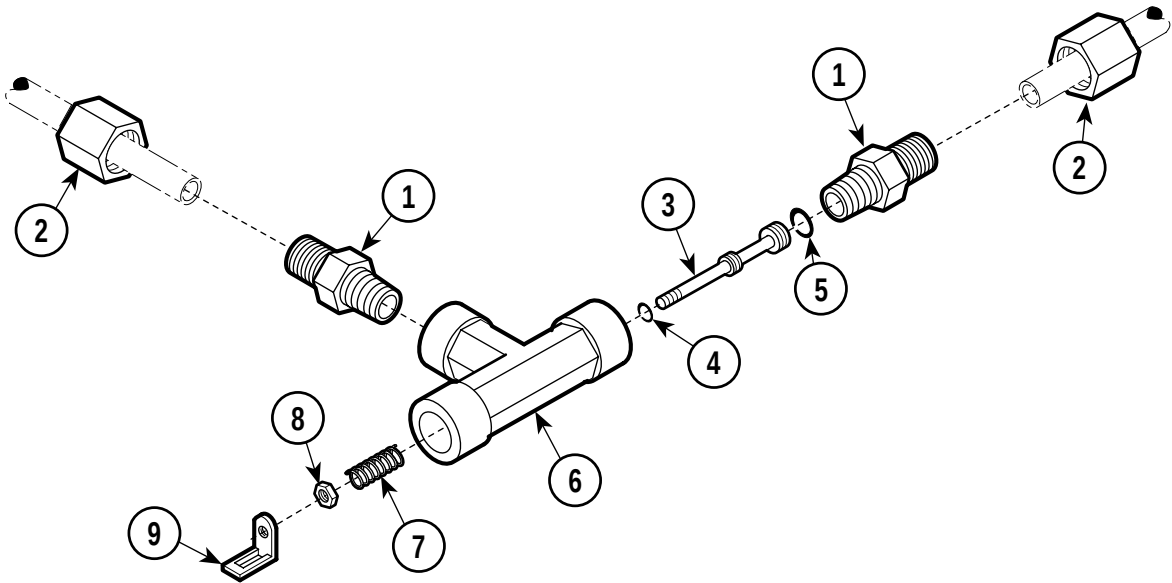

MA240603

| Used On Units Built After 6-21-90 |                 |                                                       |      |      |               |                                    |      |
|-----------------------------------|-----------------|-------------------------------------------------------|------|------|---------------|------------------------------------|------|
| Item                              | Part No.        | Description                                           | Qty. | Item | Part No.      | Description                        | Qty. |
|                                   | 002-0247-00     | Valve Assembly .....                                  | 1    |      |               | High Temp Lube #064-0002-01) ..... | 1    |
| 1                                 | • 014-0183-02   | • Fitting .....                                       | 2    | 6    | • H98068      | • Valve Body .....                 | 1    |
| 2                                 | • • 014-0392-01 | • • Compression Nut .....                             | 1    | 7    | • H98070      | • Valve Spring .....               | 1    |
| 3                                 | • H98069        | • Plunger .....                                       | 1    | 8    | • 041-0010-09 | • Nut .....                        | 1    |
| 4                                 | • H98138        | • O-Ring (Apply<br>High Temp Lube #064-0002-01) ..... | 1    | 9    | • H98077      | • Closing Bracket .....            | 1    |
| 5                                 | • H98136        | • O-Ring (Apply                                       |      |      |               |                                    |      |


Always Specify Model & Serial Number

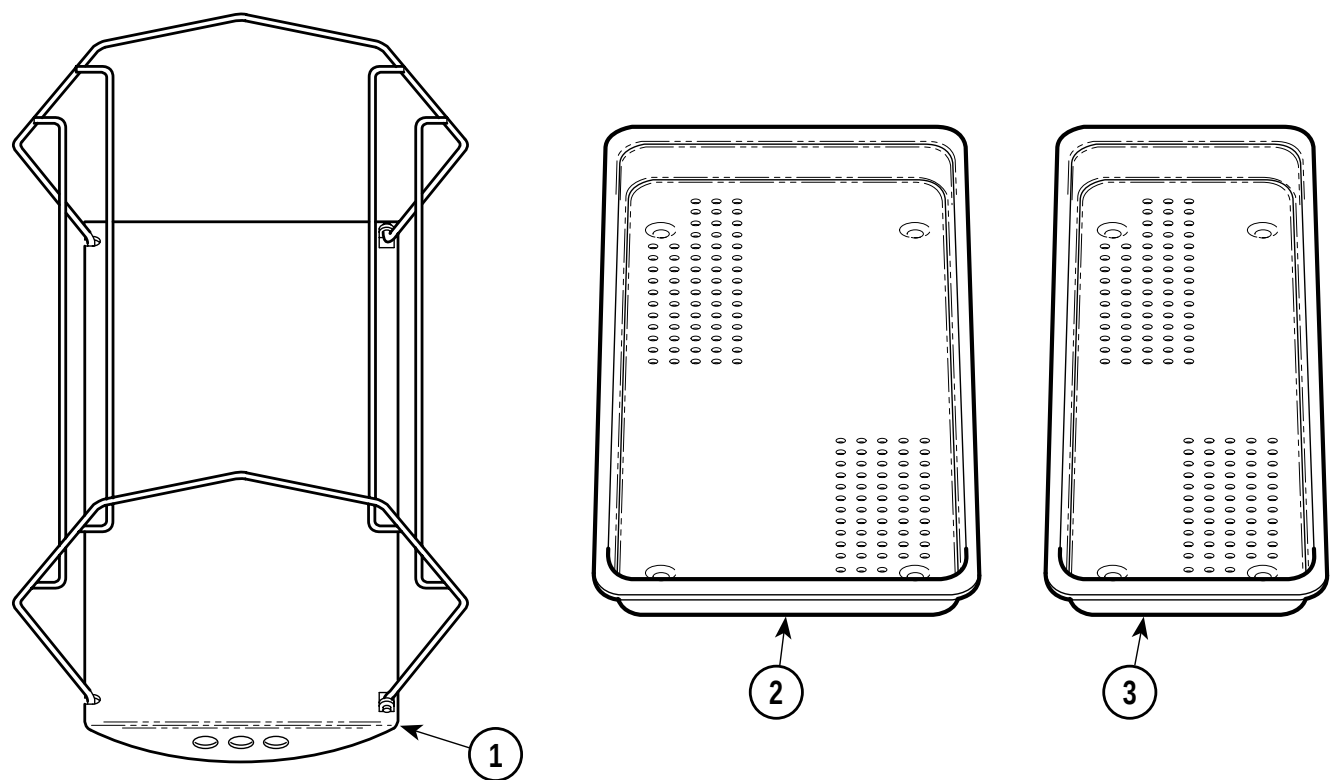

MA245700

| Item | Part No.    | Description              | Qty. | Item | Part No.    | Description   | Qty. |
|------|-------------|--------------------------|------|------|-------------|---------------|------|
| 1    | 002-0251-00 | Wire Rack Assembly ..... | 1    | 3    | 002-0252-00 | 4" Tray ..... | 1    |
| 2    | 002-0253-00 | 5" Tray .....            | 2    |      |             |               |      |

Always Specify Model & Serial Number

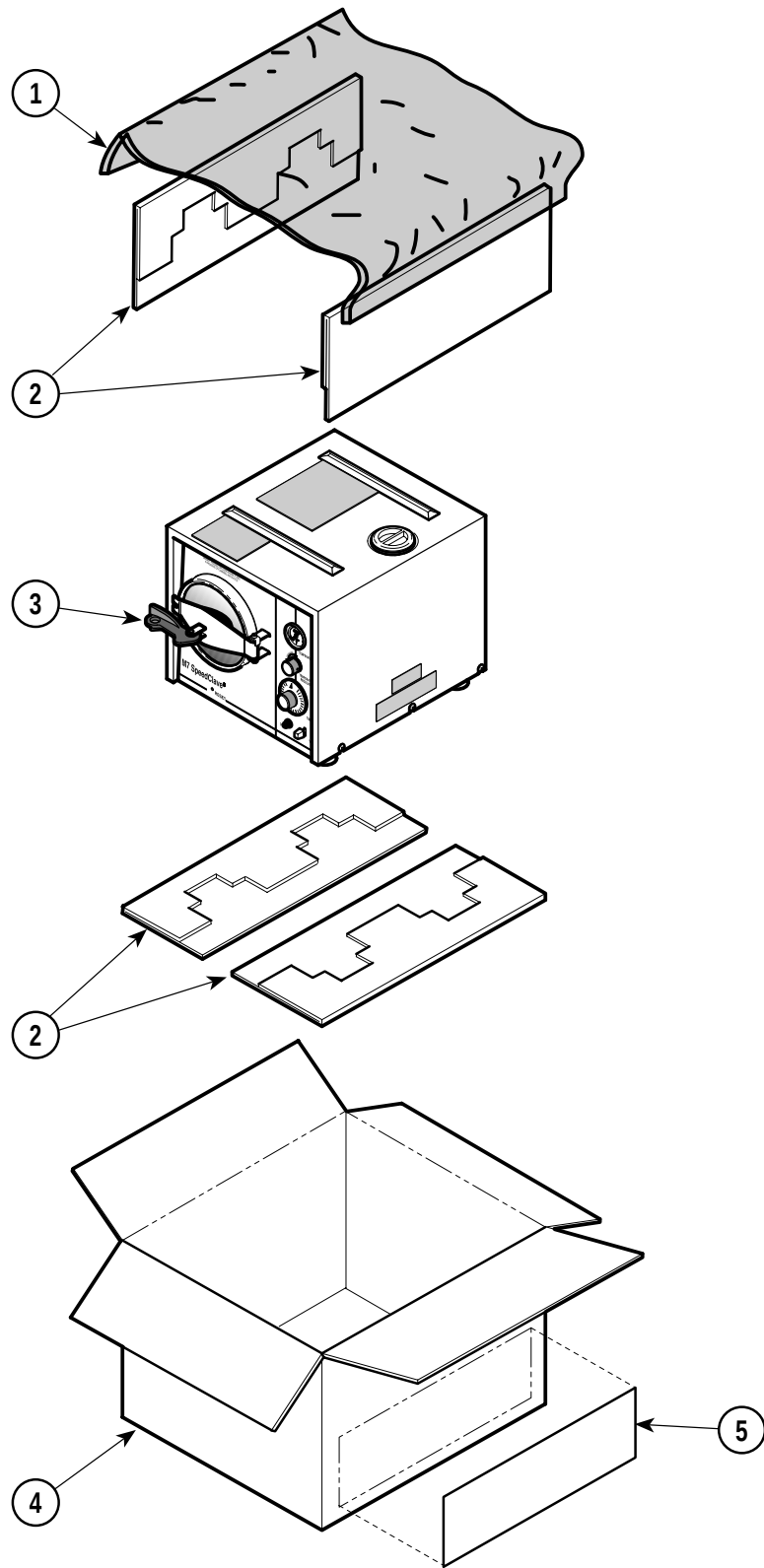

MA246301

| Item | Part No.    | Description         | Qty. | Item | Part No.    | Description          | Qty. |
|------|-------------|---------------------|------|------|-------------|----------------------|------|
| 1    | 053-0061-01 | Plastic .....       | 1    | 4    | 066-0730-00 | Carton .....         | 1    |
| 2    | 066-0240-00 | Insert .....        | 4    | 5    | 061-0719-00 | Label, Midmark ..... | 2    |
| 3    |             | M7 Sterilizer ..... | Ref  |      | 061-0719-01 | Label, Ritter .....  | 2    |

Always Specify Model & Serial Number



## COMMENTS

The Technical Publications Department of Midmark Corporation takes pride in its publications. We are sure that our manuals will fill all of your needs when you are performing scheduled maintenance, servicing, or repairs on a Midmark product.

However, if you find any errors or feel that there should be a change, addition, or deletion to a manual, please let us know!

**Page(s) and Paragraph(s) Needing Changed:**

**Description of Error or Desired Change:**

Please fax or mail a copy of this completed comment sheet to:

Midmark Corporation  
ATTN: Technical Publications Department  
60 Vista Drive  
Versailles, Ohio 45380  
Fax: (937) 526-5542

**(SERVICE PARTS ONLY)**

- ALL **BLOCKED** AREAS MUST BE COMPLETED.
- USE FOR NON-WARRANTY FAX ORDERS ONLY. WARRANTY ORDERS MUST BE TELEPHONED IN (1-800-MIDMARK).

©Midmark Corporation 2000 Rev. 1/07 Page 7-2 Printed in U.S.A.



Midmark Corporation  
60 Vista Drive  
P.O. Box 286  
Versailles, Ohio 45380-0286  
937-526-3662  
Fax 937-526-5542  
midmark.com



Because we care.

Subject to change without notice.  
Refer to [www.Documark.com](http://www.Documark.com) for latest revision.